Efficia External Paddles

The Efficia External Paddles (989803196431) are used with all models of the Efficia DFM100 and HeartStart Intrepid monitor/defibrillator. See the Efficia DFM100 or HeartStart Intrepid *Instructions for Use* for all cleaning techniques along with warnings, cautions, installation, transport and storage environment conditions, and disposal of residues applicable to the Efficia External Paddles.

Intended Use


The Efficia External Paddles are intended to be used with the Efficia DFM100 and HeartStart Intrepid devices by applying the paddles to the patient's chest to obtain an ECG and/or deliver defibrillation therapy. The primary users of the Efficia External Paddles are qualified medical personnel in a prehospital or hospital setting who are trained in the operation of the device and qualified by training in either Advanced Life Support (ALS), Advanced Cardiac Life Support (ACLS), or defibrillation.

The Efficia External Paddles are contraindicated for use in AED Mode, open-chest defibrillation, and performing non-invasive pacing in either Demand or Fixed Modes.

Features

The Efficia External Paddles (Figure 1), which fit into your existing paddle tray, can be used on both adult/child ($\geq 10\text{kg}$) and infant ($< 10\text{kg}$) patients. The apex paddle has a yellow button to remotely charge the defibrillator. Both paddles have orange shock buttons that flash when the defibrillator is charged. Press both orange buttons simultaneously to administer a shock. The sternum paddle contains a Patient Contact Indicator (PCI) with PCI icons  . Orange or red lights on the PCI indicate poor patient contact. To turn the PCI green, adjust paddle pressure and placement to optimize patient contact.

Figure 1 Paddle Features

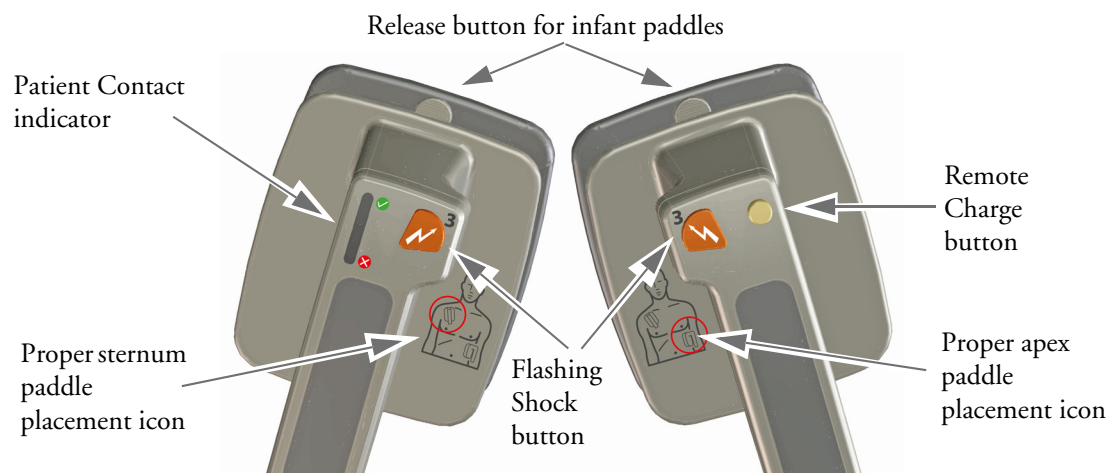NOTE: Reasonable effort should be made to obtain at least one green PCI light. This might not be possible for some patients. Orange lights may be the best that can be achieved.

Accessing Infant Paddles

To access the 989803196431 infant paddles (See Figure 2):

- 1 Press down and hold the release buttons located on the front of the Efficia External Paddles.
- 2 Slide the adult electrode clip off and away from the paddle, exposing the infant-sized surface underneath.

Figure 2 Infant Paddles

NOTE: The 989803196431 Efficia External Paddles have a solids/water resistance rating of IP54.

Specifications

The Efficia External Paddles are a Class IIb device. They comply with the requirements of the council directive 93/42/EEC concerning medical devices and carry CE-marking accordingly.

Product Model: 989803196431

Product Name: Efficia External Paddles

Paddle Weight: 2.0 lbs

Energy Range: 1-200 J

Adult Defibrillator Electrode Surface Area: 80 cm²

Infant/Child Defibrillator Electrode Surface Area: 20 cm²

WARNING: No modification of this equipment is allowed.

Manufacturer

Philips Goldway (Shenzhen) Industrial Inc.

Registered/Production Address:

No. 2 Keji North 3rd Road, Nanshan District, Shenzhen, P. R. China 518057

Tel: +86-755-26980999; Fax: +86-755-26980222

Authorized EU Representative

Shanghai International Holding Corp. GmbH (Europe)

Eiffestraße 80, 20537 Hamburg, Germany

Tel: 0049-40-2513175; Fax: 0049-40-255726

Medical Device Directive

The Efficia External Paddles comply with the requirements of the Medical Device Directive 93/42/EEC and carry the CE₀₁₂₃ mark accordingly.

Electromagnetic Compatibility

The Efficia External Paddles are intended for use in the electromagnetic environments specified for the Efficia DFM100 or HeartStart Intrepid. The Efficia External Paddles meet IEC 60601-1-2 Edition 4.0. Refer to the “Electromagnetic Compatibility” sections of the Efficia DFM100 and HeartStart Intrepid *Instructions for Use* for more information.

Service and Maintenance

The Efficia External Paddles shall be returned for replacement. No parts shall be serviced or maintained. For product support, contact your local Philips representative.

Calling for Service

To download the latest documentation, go to <https://incenter.medical.philips.com>.

Before calling for service, note the following information:

- Serial number
- Problem description

Symbol Definitions

Symbol	Definition	Symbol	Definition
	Serial number		Manufacturer
	Consult <i>Instructions for Use</i>		Caution - See operating instructions in <i>Instructions for Use</i> .
	Class II device		RoHS exempt. Environmentally friendly for a use period of 50 years.
	Manufacture Date		Dispose of in accordance with your country's requirements.

Symbol	Definition	Symbol	Definition
	Device complies with the requirements of the Medical Device Directive 93/42/EEC		Meets IEC type BF leakage current requirements and is defibrillator protected. (Patient Applied Part is isolated and defib-proof suitable for direct patient contact, except the heart or major arteries).
	Indicates hazards arising from dangerous voltages		This end up
	Fragile, handle with care		Keep dry
	Atmospheric pressure limit		Humidity limit
	Temperature limit		Recyclable
	Authorized Representative in the European Community		Refer to instruction manual/booklet

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Copyright © 2019
Shanghai International Holding Corp. GmbH (Europa)