
SERVICE PARTS LIST

ISSUED APR. 2004

OPTION FOR SL-D SERIES

TOPCON CORPORATION

NOTICE

This document contains proprietary and confidential information of Topcon Corporation, and its description is protected under copyright law and international conventions.

Any unauthorized use, duplication, reproduction, modification, disclosure distribution or transfer of any part of this document is strictly prohibited.

Copyright © 2003 Topcon Corporation, All rights reserved.

CONTENTS

ILLUSTRATION (FOR FINDING THE REQUIRED PART NUMBER AND NAME)

SR-52 (STILL CAMERA ATTACHMENT) (QUICK MIRROR TYPE).....	2
SR-53 (STILL CAMERA ATTACHMENT) (BEAM SPLITTER TYPE)	6
TL54 (TV RELAY LENS, FOR 1/2 TYPE TV CAMERA)	8
TL55 (TV RELAY LENS, FOR 1/3 TYPE TV CAMERA)	12
TL56 (TV ATTACHMENT, FOR DSC)	16
TL57 (TV ATTACHMENT, FOR PEN TYPE CCD)	18
BG4 (BACKGROUND ILLUMINATION)	20
SO-TM1 (TONOMETER MOUNT, FOR R900)	22
SO-TM2 (TONOMETER MOUNT, FOR 870)	24
SO-TM3 (TONOMETER MOUNT, FOR R900)	26
SO-TM4 (TONOMETER MOUNT, FOR 870)	28
PG-1 (PATIENT HANDLE)	30
XR-1 (XENON RELAY LENS)	32
CABLE SUPPORT	34
BG3 (BACKGROUND ILLUMINATION)	36
NIKON MOUNT	38
FD-21 (FLASH DEVICE) VERTICAL TYPE	40
FD-21 (FLASH DEVICE) HORIZONTAL TYPE	42
DC-1 (DIGITAL CAMERA UNIT) USB1.1	44
DC-1 (DIGITAL CAMERA UNIT) USB2.0	48
PB-2 (PARALLEL BINOCULAR TUBES)	52

PARTS LIST (FOR FINDING THE REQUIRED ILLUSTRATION FROM THE PARTS NUMBER)

GENERAL PARTS	53
GLASS PARTS	63
SCREW AND WASHERS, E RING	65

Using service list

Description of parts list

Illust. No.	Parts No.	Name of Parts	Q' ty	News No.	Remarks
2-1	42031 20010	Cap	1		
2-2	42031 20200	Body unit	1		
2-3a	42031 20080	Cover	1		Light gray
2-3b	42033 20050	Cover	1		Black
2-6a	G123 M4567	Scale	1		For Cross target
2-6b	G234 M5678	Scale	1		For Corona target
2-10a	42031 2060A	Eye piece assembly	1		For Cross target
2-10b	42031 2060B	Eye piece assembly	1		For Corona target
2-11a	43500 60150	Spacer	4		S/N 310001 ~ 31032
2-11b	42031 20610	Spacer	4		S/N 310323
2-12	42031 60100	PC board unit	1		
2-13	CR3 × 20S	Screw	4		Cr
2-14a	42031 20070	Name plate	1		LH-1
2-14b	42032 20010	Name plate	1		LH-1E
2-14c	42035 20010	Name plate	1		LH-2

Note: This is for reference only, and differs in content from that in the service parts list.

Illustration No. : Used for indexing disassembly drawing and parts list.

Parts code:

Parts description. :

Quantity: Quantity per item.

Remarks: A supplementary explanation of exploded illustration and Illust. No.
(EX. -)

Model name: Name of model in question.

Paint color: Paint colors of parts which have two or more different colors applied.

Specification: Specifications of parts which have two or more specifications or function.

Surface treatment:

Manufacture No. :

Description of index

Index covers general parts, screws, washers, E-rings and glass parts by items.

For ordering


- (1) Be sure to specify model name, Illustration No., parts code, quantity and parts name when placing orders.

(Ex.)

Model	Illust. No.	Parts No.	Qt' y	Name of Parts	Remarks
LH-1E	2-3B	42033 20050	1	Cover	Black

- (2) Those parts indicated only by unit and assembly parts diagrams will not be supplied as separate parts to maintain performance.
- (3) Those parts indicated by the exploded illustration of assembly parts will be supplied as separate parts.
- (4) Place orders for screws and washers by units of 10.
- (5) Name plates will not be supplied. Damaged name plate, however, will be replaced.

Description of exploded illustration

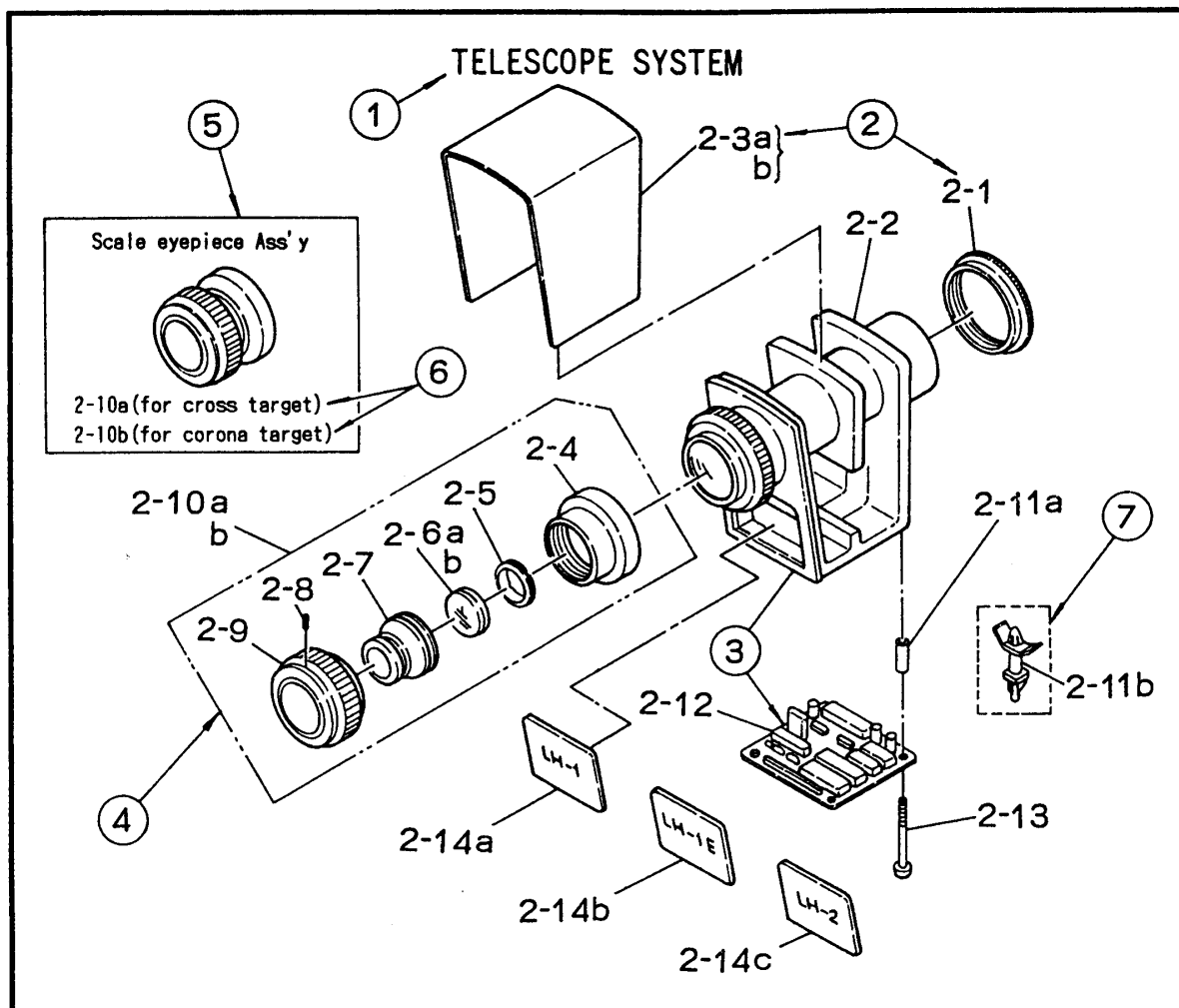

Note: This illustration is for reference and differs in content from that in the service parts list.

Block description

Illustration No.

For indexing exploded illustration and parts list.

See relevant remarks in the parts list for parts with an alphabet letter at the end of part number.
(Ex.2-3a)

Unit parts

Unit parts are described as they are fitted.

Exploded illustration of assembly parts

The exploded illustration of assembly parts are surrounded by two-dotted lines.

Assembly parts diagram

Assembly parts diagram is surrounded by full lines.

Supplementary explanation

A supplementary explanation may be given to explain the composition.

Changed parts


Diagrams of parts which change shape considerably in the course of production are surrounded by dotted lines. Those parts which are not covered by single page for space reasons, see also on the next page.

SR-52 (STILL CAMERA ATTACHMENT) (QUICK MIRROR TYPE)

Illust. Page 2

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44790 10010	Body	1		
2	44790 10020	Mount	1		
3	44790 10030	Mask	1		
4	44790 10040	Cover	1		
5	44790 10050	Cap	1		
6	44790 10060	Lens holder	1		
7	44790 10070	Lens barrel	1		
8	44790 10080	Retainer ring -1	1		
9	44790 10090	Retainer ring -2	1		
10	44691 10210	Mirror base	1		
11	44691 10420	Washer	2		
12	44691 10350	Spring	1		
13	44691 10220	Mirror holder	1		
14	44691 10240	Mirror clamp	1		
15	44691 10260	Plate	2		
16	44790 10170	Mirror link	1		
17	44790 10180	Mirror frame	1		
18	44790 10190	Solenoid arm	1		
19	44691 10250	Mirror clamp	1		
20	44790 10220	Solenoid holder	1		
21	44790 10230	Rotary solenoid	1		
22	44691 10270	Plate	1		
23	44942 31520	Receptacle	1		
24	40452 24030	Wire sticker wiring parts	1		
25	44691 15050	Connector	1		
26	44691 10130	Solenoid plate pin	1		
27	44691 10190	Cover	1		
28	44691 10230	Pin	1		
29	44691 10380	Spring retainer	1		
30	44691 10390	Spring	1		
31	44670 10220	Screw	2		
32	44670 15230	Metal	1		
33	45520 10310	Screw	2		
34	45520 11120	Plate	1		
35	44801 40080	Spacer	2		
36	44801 40510	Spacer	2		
37	44801 40780	Spacer	1		
38	G604M 72080	Lens	1		
39	G234M 72100	Lens	1		
40	G444M 48960	Mirror	1		

SR-52 (STILL CAMERA ATTACHMENT) (QUICK MIRROR TYPE)

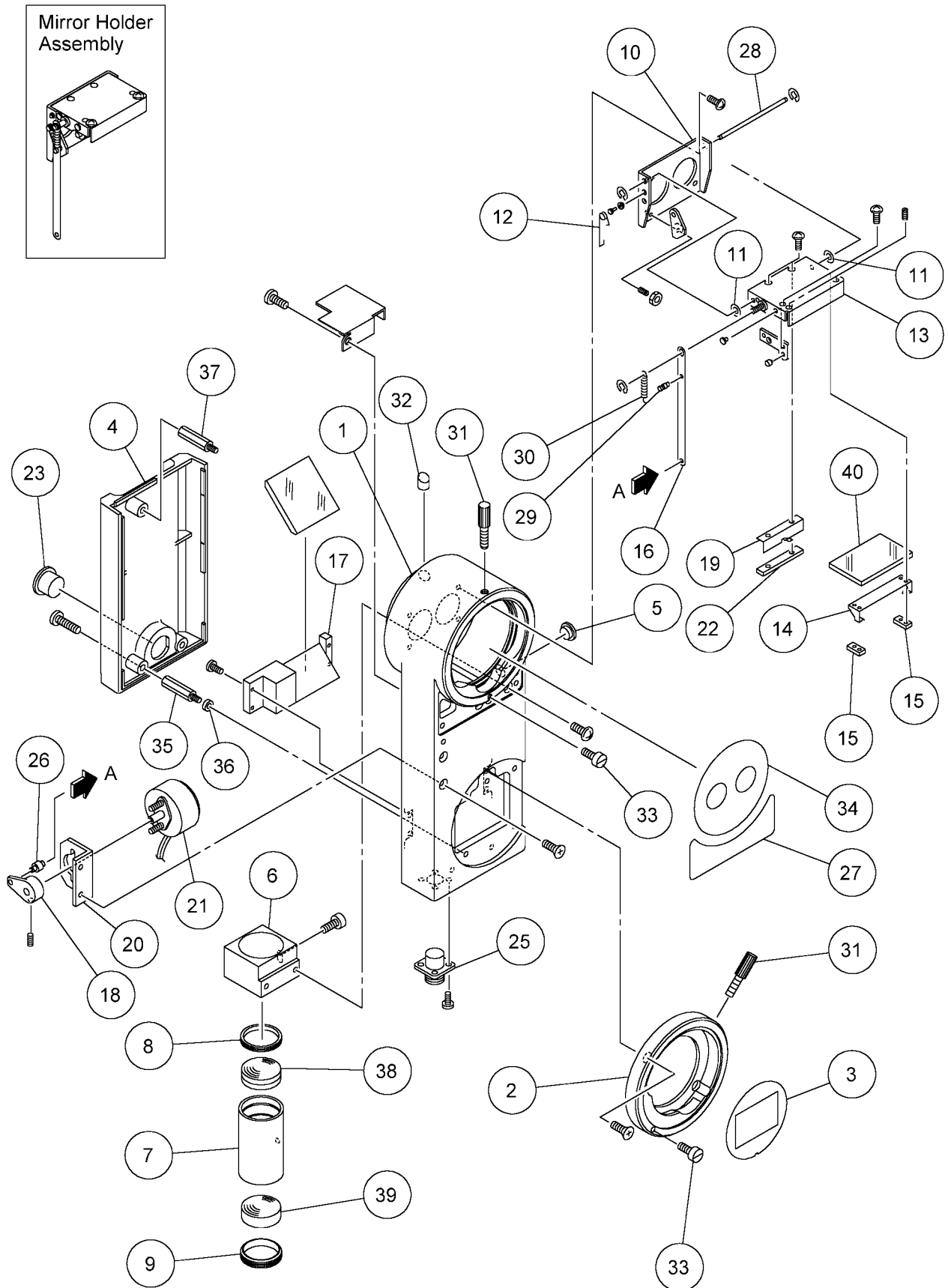


SR-52 (STILL CAMERA ATTACHMENT) (QUICK MIRROR TYPE)

Illust. Page 4

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
41	G444M 49000	Mirror	1		
42	CR2x3S	CR-screw (BZn)	3		
43	CR2x5S	CR-screw (Ni)	4		
44	CR2x4S	CR-screw (BZn)	4		
45	CR2x8S	CR-screw (BZn)	4		
46	CR3x5S	CR-screw (BZn)	3		
47	CR3x8SUS	CR-screw (SUS)	3		
48	CQ3x5SUS	CQ-screw (SUS)	2		
49	CQ3x8SUS	CQ-screw (SUS)	2		
50	6SU3x5S	6SU-set screw (BZn)	1		
51	1N2.6S	Nut (BZn)	1		
52	6S3x5S	Socket bolt (BZn)	1		
53	U2.6x6S	U-set screw (BZn)	1		
54	EJ1.5S	E-ring (BZn)	2		
55	EJ2S	E-ring (BZn)	2		
56	U2x4S	U-set screw (BZn)	2		
57	44691 10280	Rivet	2		
58	44691 10290	Stopper	1		
59	44691 10310	Mirror stopper	2		
60	44691 10320	Stopper holder	1		
61	44691 10330	Stopper shaft	1		
62	44691 10340	Stopper shaft holder	1		
63	44790 10100	Mirror holder assembly	1		
64	44790 10260	Plate	1		

SR-52 (STILL CAMERA ATTACHMENT) (QUICK MIRROR TYPE)

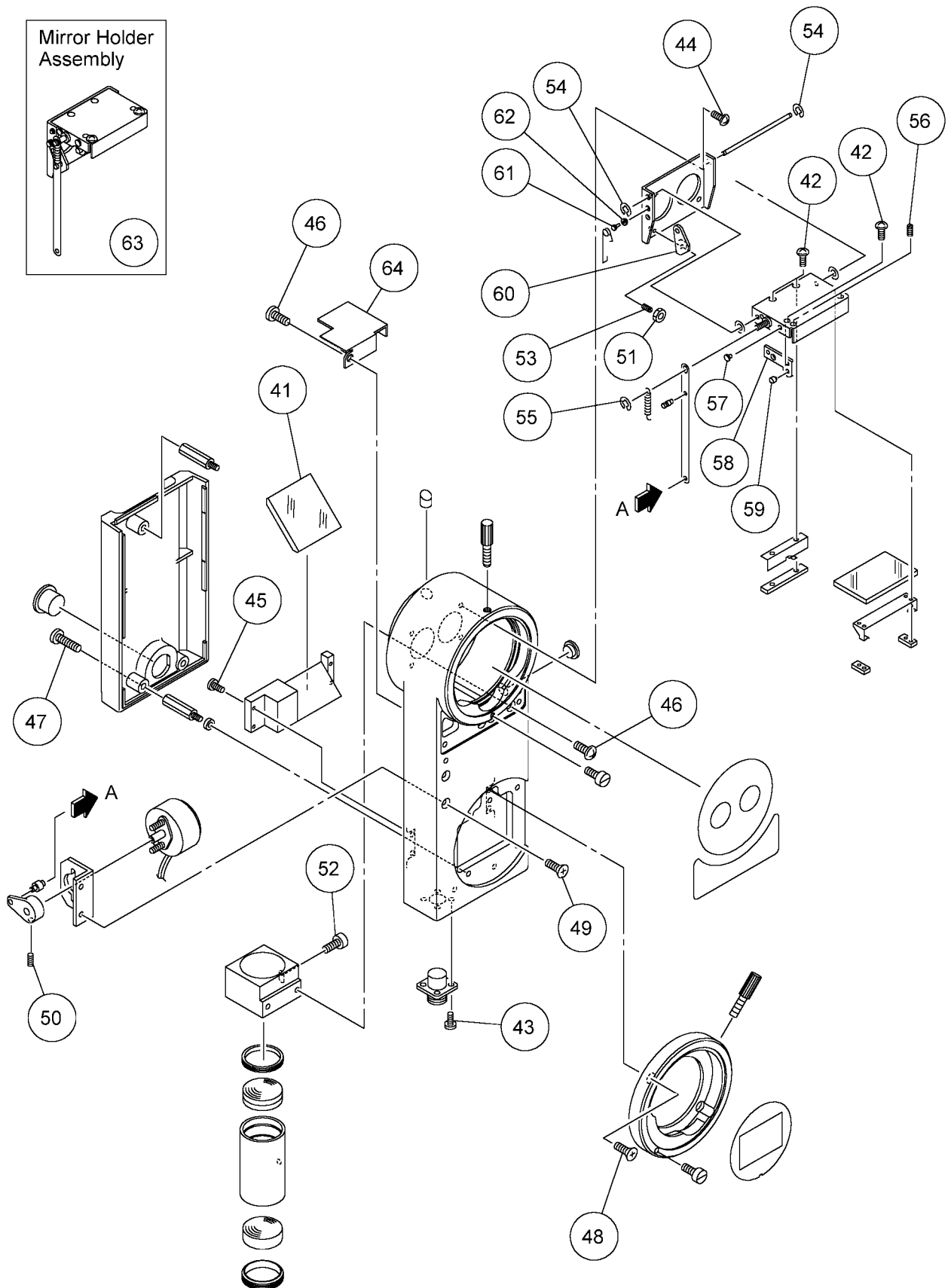


SR-53 (STILL CAMERA ATTACHMENT) (BEAM SPLITTER TYPE)

Illust. Page 6

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44790 10010	Body	1		
2	44790 10020	Mount	1		
3	44790 10030	Mask	1		
4	44790 10040	Cover	1		
5	44790 10050	Cap	1		
6	44790 10060	Lens holder	1		
7	44790 10070	Lens barrel	1		
8	44790 10080	Retainer ring -1	1		
9	44790 10090	Retainer ring -2	1		
10	44790 10180	Mirror base plate	1		
11	44694 17020	Holder	1		
12	44801 40080	Spacer	2		
13	44801 40510	Spacer	2		
14	44801 40780	Spacer	1		
15	44691 10190	Cover	1		
16	44670 10220	Screw	2		
17	44670 15230	Metal	1		
18	45520 10310	Screw	2		
19	45520 11120	Plate	1		
20	44694 13030	Wedge prism cell	1		
21	44694 13040	Wedge prism cell	1		
22	G614M 69920	Prism	1		
23	G334M 66170	Prism	1		
24	G444M 49000	Mirror	1		
25	G604M 72080	Lens	1		
26	G234M 72100	Lens	1		
27	CR2 × 8S	CR-screw (BZn)	4		
28	CR3 × 5S	CR-screw (BZn)	3		
29	CR3 × 8SUS	CR-screw (SUS)	3		
30	CR3 × 16SUS	CR-screw (SUS)	3		
31	CQ3 × 5SUS	CQ-screw (SUS)	4		
32	6S3 × 5S	Socket bolt (BZn)	1		
33	6SU3 × 6S	6SU-set screw (BZn)	3		
34	44790 10260	Plate	1		

SR-53 (STILL CAMERA ATTACHMENT) (BEAM SPLITTER TYPE)

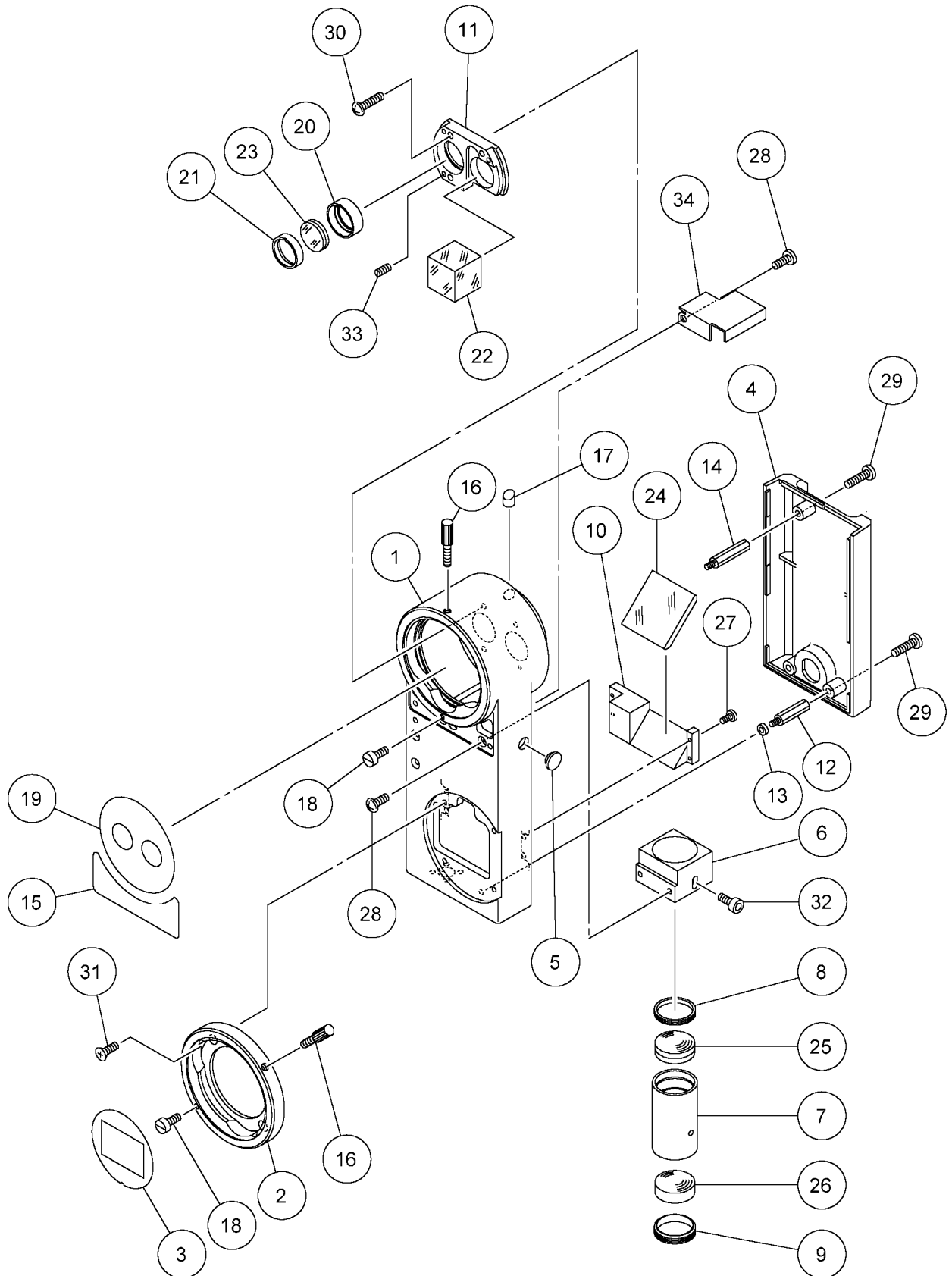


TL54 (TV RELAY LENS, FOR 1/2 TYPE TV CAMERA)

Illust. Page 8

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	G134M 71590	Lens	1		
2	G143M 71610	Lens	1		
3	CR3×8S SUS	CR-screw	2		
4	FW3S BZn	Washer	2		
5	6SV3×4S BZn	6SV-set screw	7		
6	6S3×5S BZn	Socket-bolt	7		
7	G614M 69920	Prism	1		
8	44693 10140	Knob	1		
9	G444M 58030	Mirror	1		
10	G234M 71610	Lens	1		
11	45520 10310	Screw	2		
12	44693 10170	Screw	1		
13	44670 15230	Metal	1		
14	44699 35040	Cell	1		
15	44699 35050	Cell	1		
16	44793 10010	Body	1		
17	44793 10020	Mount	1		
18	44793 10030	Slider	1		
19	44793 10050	Pin	1		
20	44793 10070	Plate	1		
21	44793 10080	Shaft	1		
22	44793 10090	Shaft	2		
23	44793 10110	Cell	1		
24	44793 10120	Spacer	1		
25	44793 10130	Spacer	1		
26	44793 10140	Ring	1		
27	44793 10150	Cover	1		
28	44793 10160	Cover	1		
29	44802 40040	Plunger	1		
30	44802 60160	Clamp	1		
31	45520 11120	Plate	1		
32	G334M 66170	Prism	2		
33	G133M 71620	Lens	2		
34	G603M 71640	Lens	2		
35	44793 10060	Ring	1		
36	44793 10170	Body	1		
37	44793 10180	Cell	1		
38	44793 10190	Spacer	2		
39	44793 10210	Cam	1		
40	44793 10220	Ring	1		

TL54 (TV RELAY LENS, FOR 1/2 TYPE TV CAMERA)

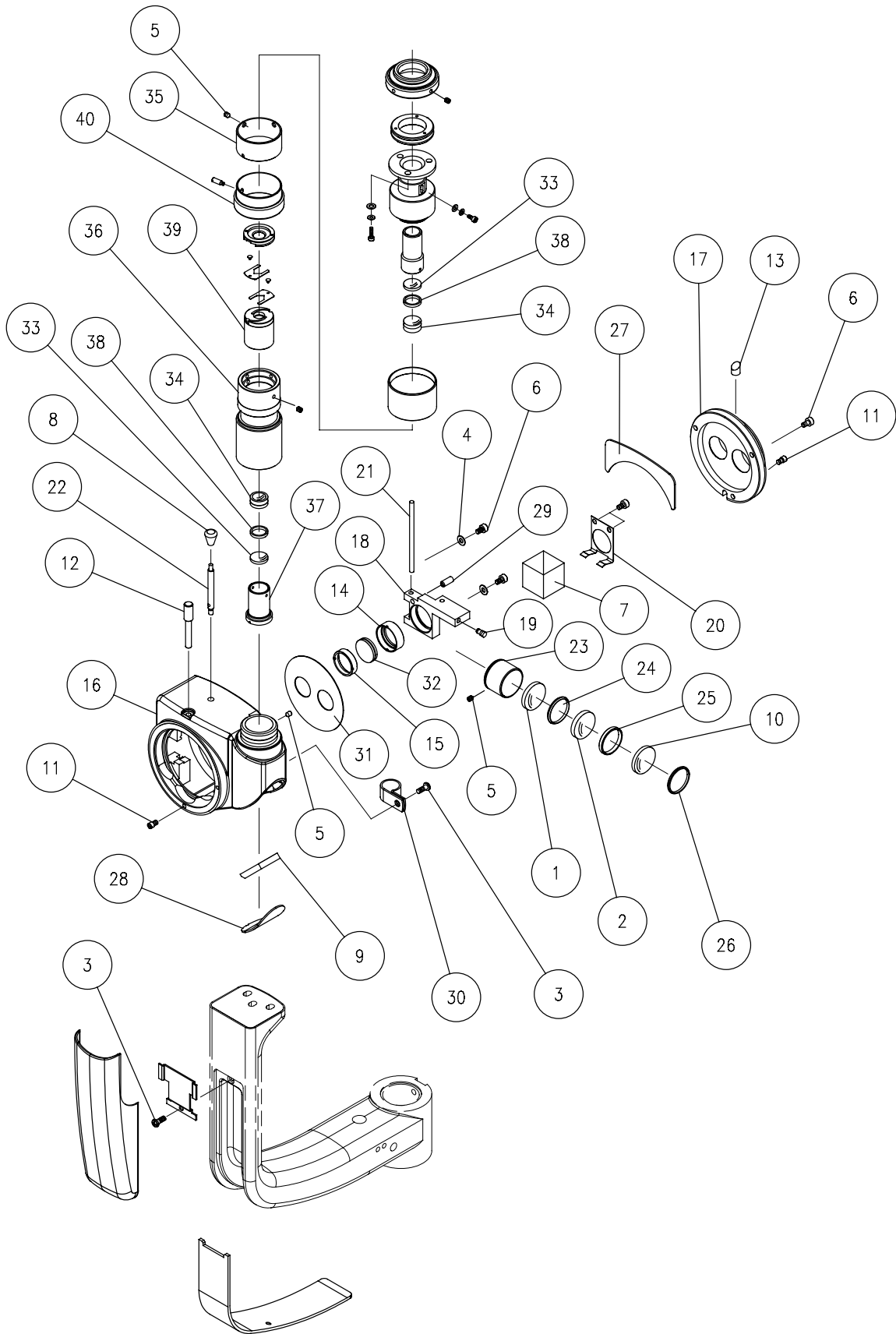


TL54 (TV RELAY LENS, FOR 1/2 TYPE TV CAMERA)

Illust. Page 10

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
41	44793 10230	Plate	2		
42	44793 10240	Pin	2		
43	44793 10250	Pin	1		
44	44793 10260	Holder	1		
45	44793 10040	Plate	1		
46	44793 10270	Cover	1		
47	44793 10280	Cover	1		
48	SW2S BZn	Spring-washer	1		
49	6S2×5 BZn	Socket-bolt	1		
50	6SV3×4S BZn	6SV-set screw	2		
51	6S2×8S BZn	Socket-bolt	3		
52	FWs3S BZn	Washer	3		
53	FW2S BZn	Washer	4		
54	44793 30010	Adapter	1		
55	44793 30020	Lens cell	1		
56	44793 30030	Cover	1		
57	44793 30040	Mount	1		
58	44793 30050	Base	1		

TL54 (TV RELAY LENS, FOR 1/2 TYPE TV CAMERA)

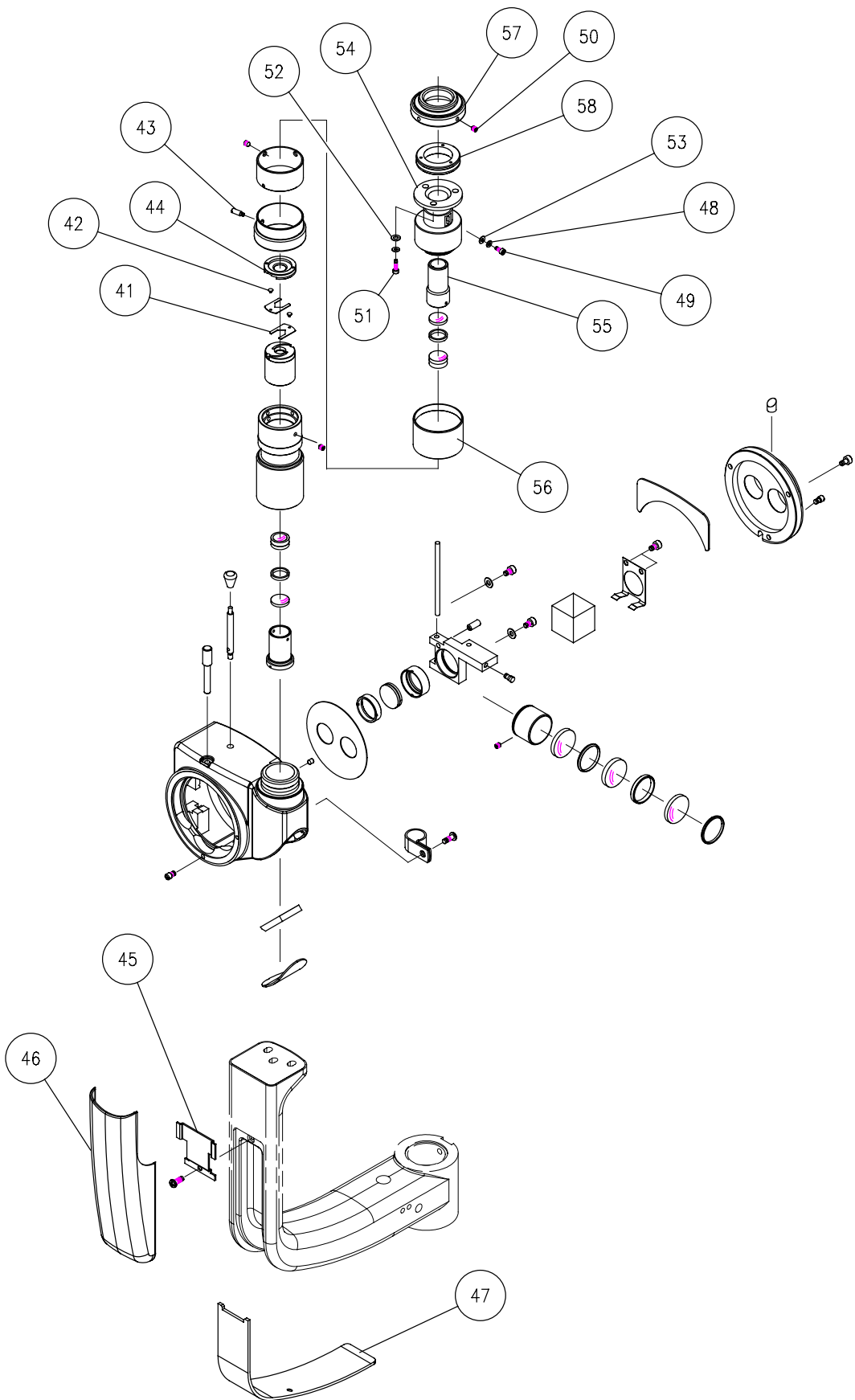


TL55 (TV RELAY LENS, FOR 1/3 TYPE TV CAMERA)

Illust. Page 12

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	G134M 71590	Lens	1		
2	G143M 71610	Lens	1		
3	CR3×8S SUS	CR-screw	2		
4	FW3S BZn	Washer	2		
5	6SV3×4S BZn	6SV-set screw	7		
6	6S3×5S BZn	Socket-bolt	7		
7	G614M 69920	Prism	1		
8	44693 10140	Knob	1		
9	G444M 58030	Mirror	1		
10	G234M 71610	Lens	1		
11	45520 10310	Screw	2		
12	44693 10170	Screw	1		
13	44670 15230	Metal	1		
14	44699 35040	Cell	1		
15	44699 35050	Cell	1		
16	44793 10010	Body	1		
17	44793 10020	Mount	1		
18	44793 10030	Slider	1		
19	44793 10050	Pin	1		
20	44793 10070	Plate	1		
21	44793 10080	Shaft	1		
22	44793 10090	Shaft	1		
23	44793 10110	Cell	1		
24	44793 10120	Spacer	1		
25	44793 10130	Spacer	1		
26	44793 10140	Ring	1		
27	44793 10150	Cover	1		
28	44793 10160	Cover	1		
29	44802 40040	Plunger	1		
30	44802 60160	Clamp	1		
31	45520 11120	Plate	1		
32	G334M 66170	Prism	2		
33	G133M 71620	Lens	1		
34	G603M 71640	Lens	1		
35	44793 10060	Ring	1		
36	44793 10170	Body	1		
37	44793 10180	Cell	1		
38	44793 10190	Spacer	1		
39	44793 10210	Cam	1		
40	44793 10220	Ring	1		

TL55 (TV RELAY LENS, FOR 1/3 TYPE TV CAMERA)

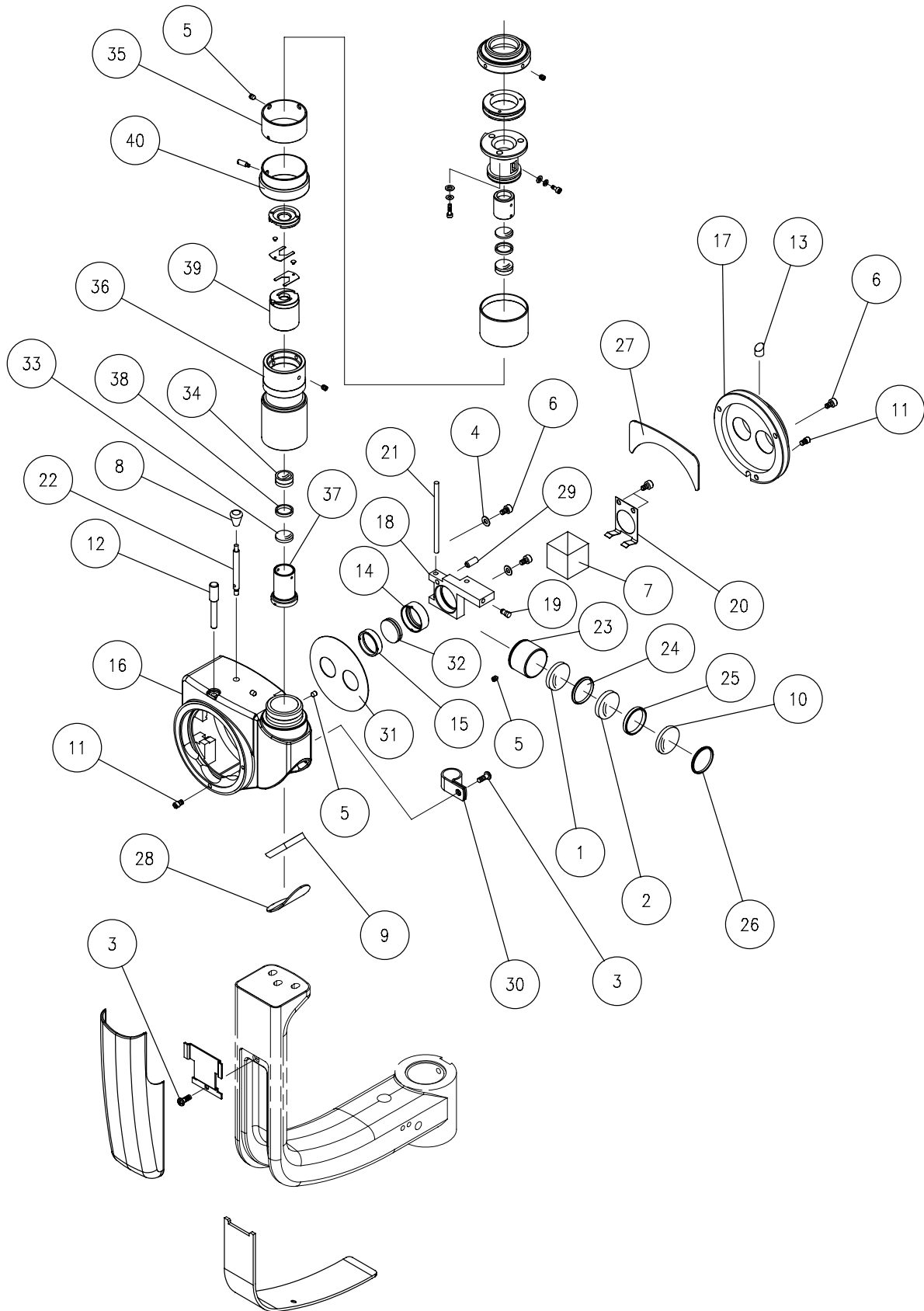


TL55 (TV RELAY LENS, FOR 1/3 TYPE TV CAMERA)

Illust. Page 14

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
41	44793 10230	Plate	2		
42	44793 10240	Pin	2		
43	44793 10250	Pin	1		
44	44793 10260	Holder	1		
45	44793 10040	Plate	1		
46	44793 10270	Cover	1		
47	44793 10280	Cover	1		
48	SW2S BZn	Spring-washer	1		
49	6S2×5 BZn	Socket-bolt	1		
50	6SV3×4S BZn	6SV-set screw	2		
51	6S2×8S BZn	Socket-bolt	3		
52	FWs3S BZn	Washer	3		
53	FW2S BZn	Washer	4		
54	44793 30030	Cover	1		
55	44793 30040	Mount	1		
56	44793 30050	Base	1		
57	44793 31010	Adapter	1		
58	44793 31020	Cell	1		
59	44793 31030	Spacer	1		
60	G603M 71690	Lens	1		
61	G133M 71670	Lens	1		

TL55 (TV RELAY LENS, FOR 1/3 TYPE TV CAMERA)

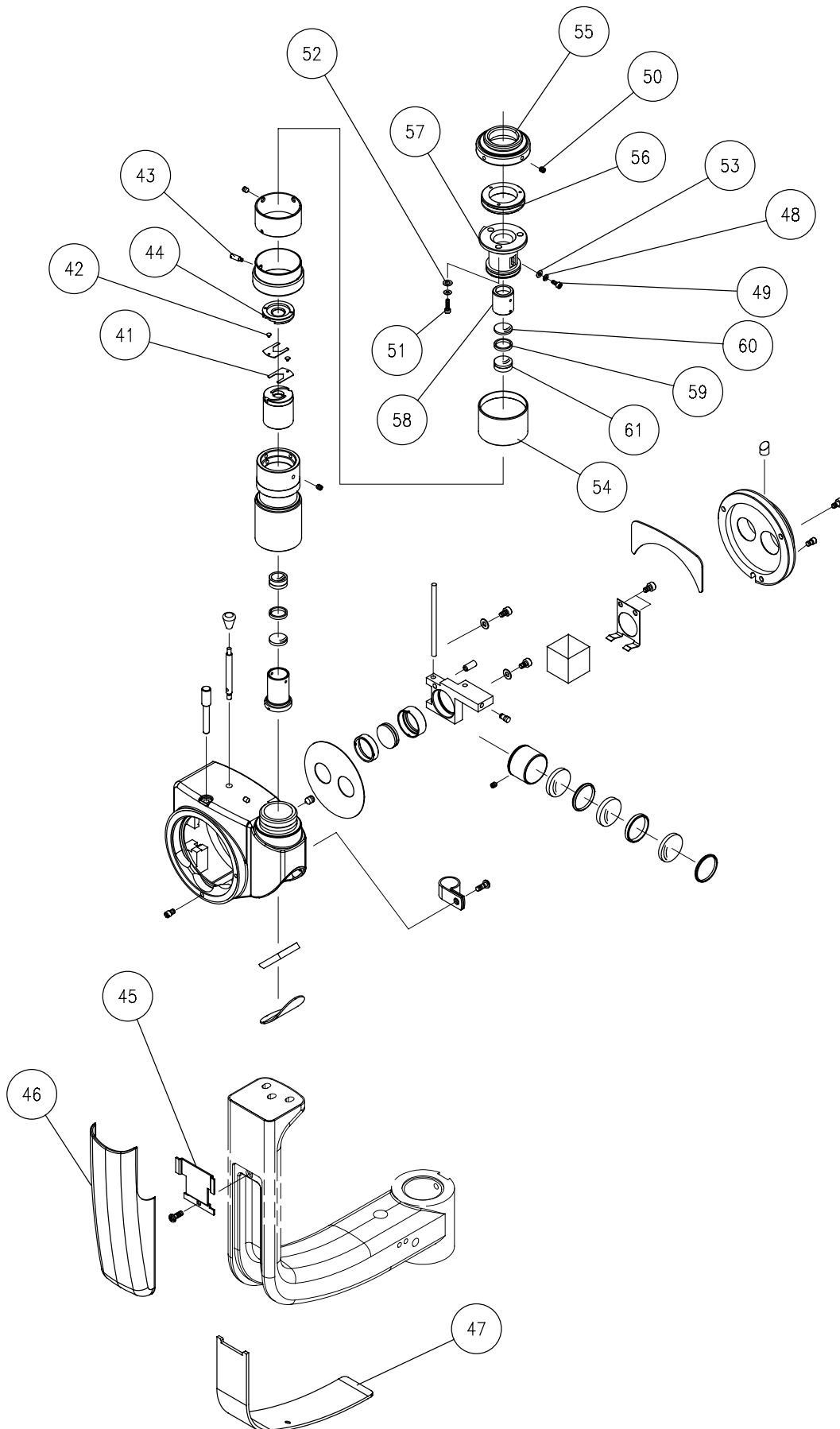


TL56

Illust. Page 16

(TV ATTACHMENT, FOR DSC)

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44793 11010	Body	1		
2	44793 11020	Prism mount	1		
3	44793 11030	Mirror box cover	1		
4	44793 11040	Mirror box	1		
5	44793 11050	Mirror mount	1		
6	44793 11060	Lens holder	1		
7	44793 11070	Mount	1		
8	44793 11080	Cover	1		
9	44793 11090	Cam screw	1		
10	44793 11110	Adjust ring	1		
11	45520 11120	Plate	1		
12	44694 13090	Spacer ring	1		
13	44694 13120	Retainer ring	1		
14	44694 13160	Fixed ring	1		
15	44670 10220	Screw	1		
16	44670 15230	Metal	1		
17	45520 10310	Screw	1		
18	44694 13030	Wedge prism cell	1		
19	44694 13040	Wedge prism cell	1		
20	G134M 66160	Lens	1		
21	G604M 66140	Lens	1		
22	G614M 69920	Prism	1		
23	G334M 66170	Prism	2		
24	G444M 66120	Mirror	1		
25	CR2 × 4SUS	CR-screw (SUS)	1		
26	CR3 × 5S	CR-screw (BZn)	3		
27	6S3 × 8SUS	Socket bolt (SUS)	2		
28	6SU2 × 3S	6SU-set screw (BZn)	3		
29	6SV3 × 4S	6SV-set screw (BZn)	3		

TL56 (TV ATTACHMENT, FOR DSC)

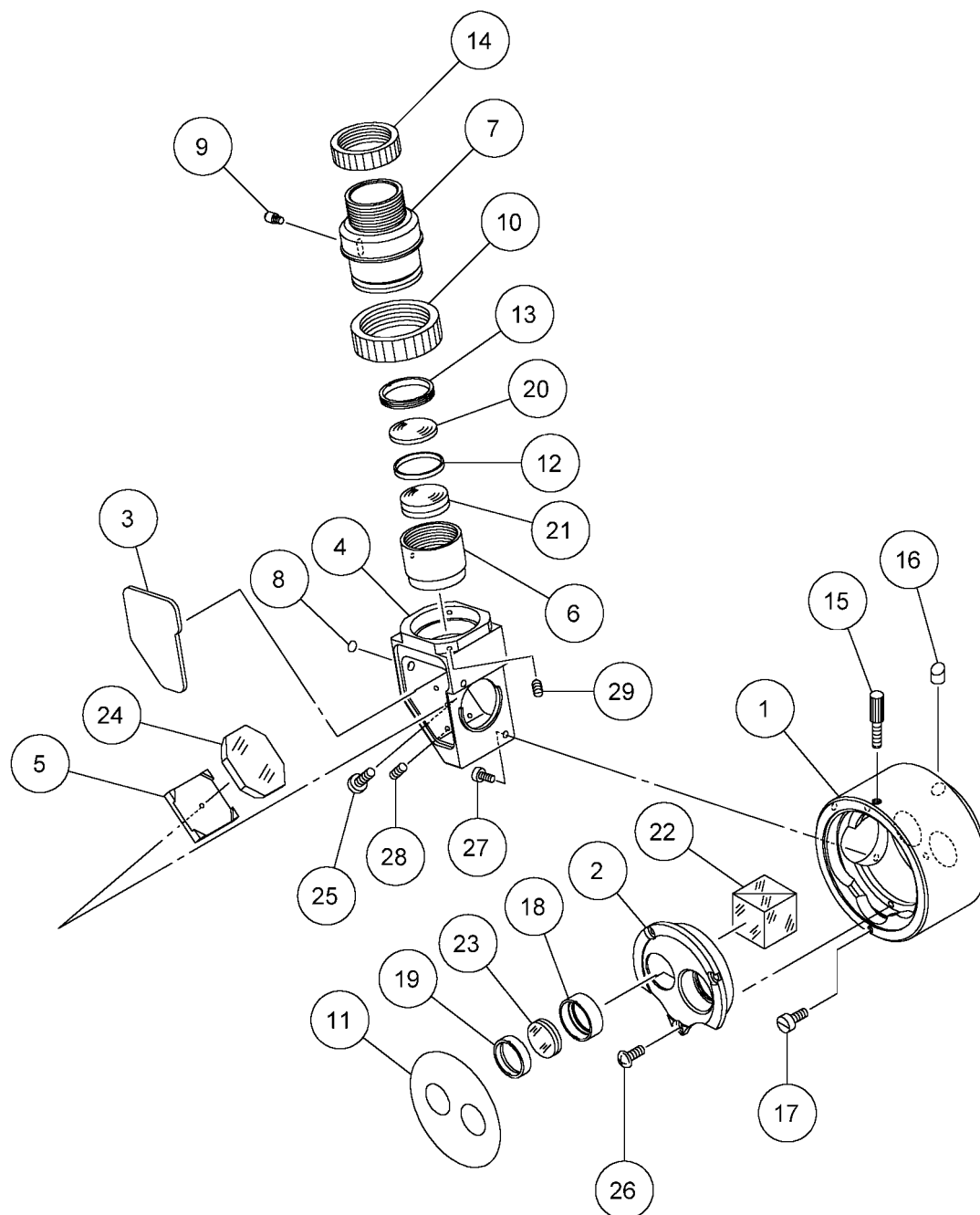


TL57 (TV ATTACHMENT, FOR PEN TYPE CCD)

Illust. Page 18

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44793 12010	Body	1		
2	44793 12020	Arm cover	1		
3	44793 12030	Cord bush	1		
4	44694 15020	Prism holder	1		
5	44694 15030	Spacer	1		
6	44694 15040	Mask plate	1		
7	44694 15050	Mount	1		
8	45520 10310	Screw	1		
9	45520 11120	Plate	1		
10	44670 10220	Screw	1		
11	44670 15230	Metal	1		
12	44694 13030	Wedge prism cell	1		
13	44694 13040	Wedge prism cell	1		
14	G614M 69920	Prism	1		
15	G334M 66170	Prism	2		
16	G604M 69890	Lens	1		
17	G444M 69910	Mirror	1		
18	CR3x8S	CR-screw (BZn)	1		
19	6SU3x5S	6SU-set screw (BZn)	4		
20	CQ3x5SUS	CQ-screw (SUS)	2		

TL57 (TV ATTACHMENT, FOR PEN TYPE CCD)

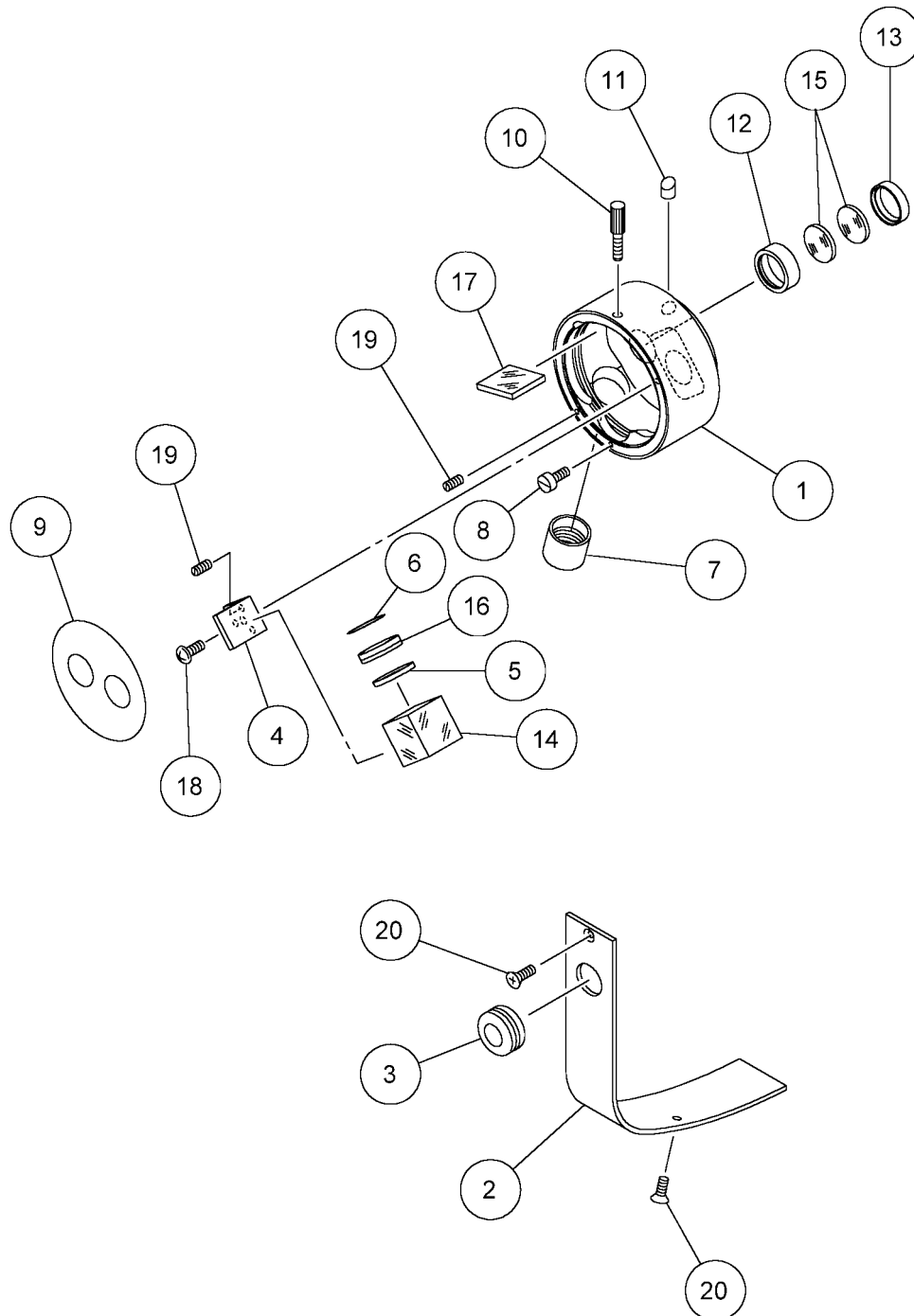


BG4

(BACKGROUND ILLUMINATION)

Illust. Page 20

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	1N3S Ni	Nut	1		
2	TPs2×4S BZn	TPs-screw	2		
3	SW3S Ni	Spring washer	1		
4	6SV3×4S BZn	6SV-set screw	2		
5	CR4×8S BZn	CR-screw	2		
6	SW4S BZn	Spring washer	2		
7	SW4S Ni	Spring washer	2		
8	6SU3×6S BZn	6SU-set screw	2		
9	30305 20590	Bis	1		
10	40340 20450	Ring	1		
11	40340 20460	Ring	1		
12	40340 20540	Spring	1		
13	40340 20550	Knob	1		
14	44670 25650	Plug Pin	1		
15	44670 25660	Screw	2		
16	44670 25780	Knob	2		
17	44670 25940	Chip	2		
18	44670 25970	Plug pin	1		
19	44680 25930	Terminal	1		
20	44680 25940	Connector Post	1		
21	44680 25950	Plug Base	1		
22	44694 30060	Cover	1		
23	44694 40010	Lens Cell	1		
24	44694 40020	Holder	1		
25	44694 40050	Lamp House	1		
26	44770 25150	Filter	1		
27	44770 25230	Spring	1		
28	44793 40010	Lamp Cover	1		
29	G123M 20700	Lens	1		
30	G124M 20710	Lens	1		
31	49790 41010	Case	1		
32	49790 41020	Lever	1		
33	49790 41030	Plate	1		
34	49790 41040	Pin	2		
35	40350 70030	Spring	1		
36	44694 31010	Light Guide	1		
37	44691 70080	Adapter	1		
38	CR2×5S BZn	CR-screw	2		
39	6SV2×3S BZn	6SV-set screw	3		
40	Ball Sei 3	Steel Ball	1		
41	44670 25980	Connector	2		

BG4 (BACKGROUND ILLUMINATION)

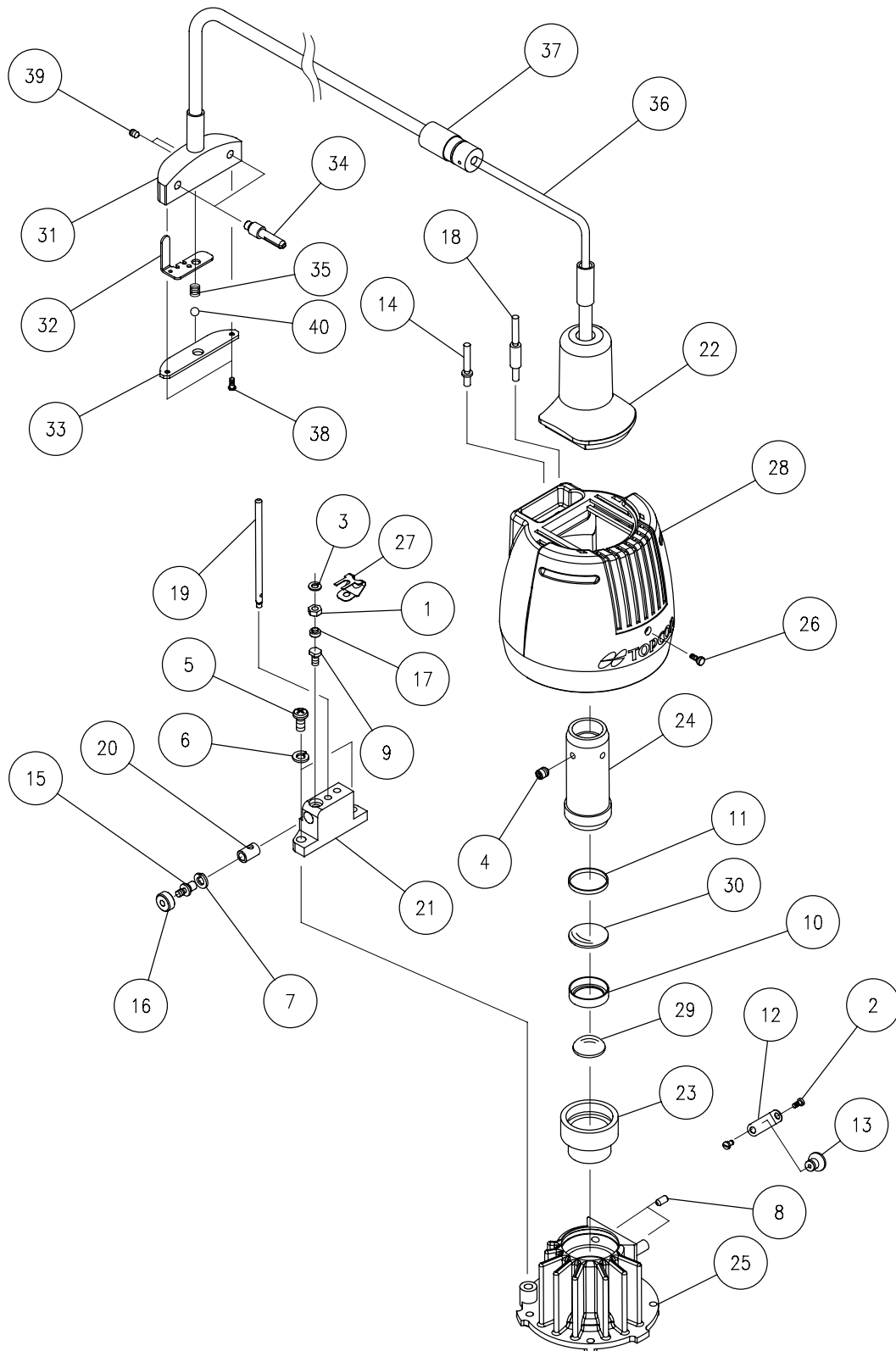


SO-TM1 (TONOMETER MOUNT, FOR R900)

Illust. Page 22

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44795 10010	Tonometer mount base	1		
2	CT4x20SUS	CT-screw (SUS)	1		
3	44795 10030	Mount knob	1		
4	44795 10040	Shaft	1		
5	40391 12020	Pin	1		
6	40391 12030	Pin	1		
7	44680 10151	Pin	1		
8	44680 10161	Pin	2		
9	6S4x12S	Socket bolt (BZn)	1		
10	FW4S	Washer (BZn)	1		

SO-TM1 (TONOMETER MOUNT, FOR R900)

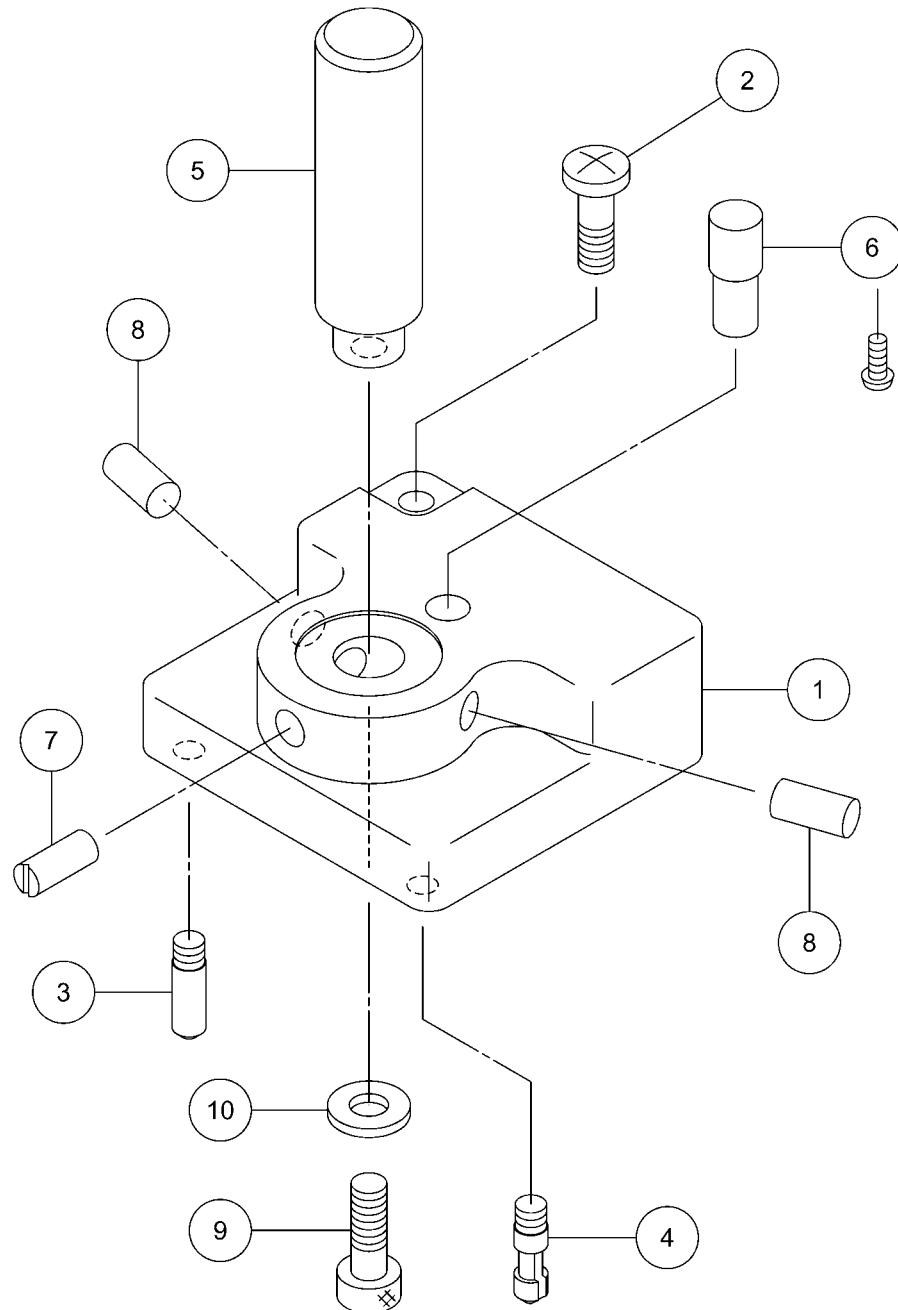


SO-TM2 (TONOMETER MOUNT, FOR 870)

Illust. Page 24

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44795 11010	Base	1		
2	44795 11020	Screw	1		
3	44795 11030	Bearing plate	1		
4	44795 10030	Pin	1		
5	44795 10040	Pin	1		
6	40349 10060	Pin	1		
7	40349 10070	Screw	1		
8	41109 13820	Key	1		
9	SP2x10S	Pin (SUS)	1		
10	6S4x12S	Socket bolt (BZn)	2		

SO-TM2 (TONOMETER MOUNT, FOR 870)

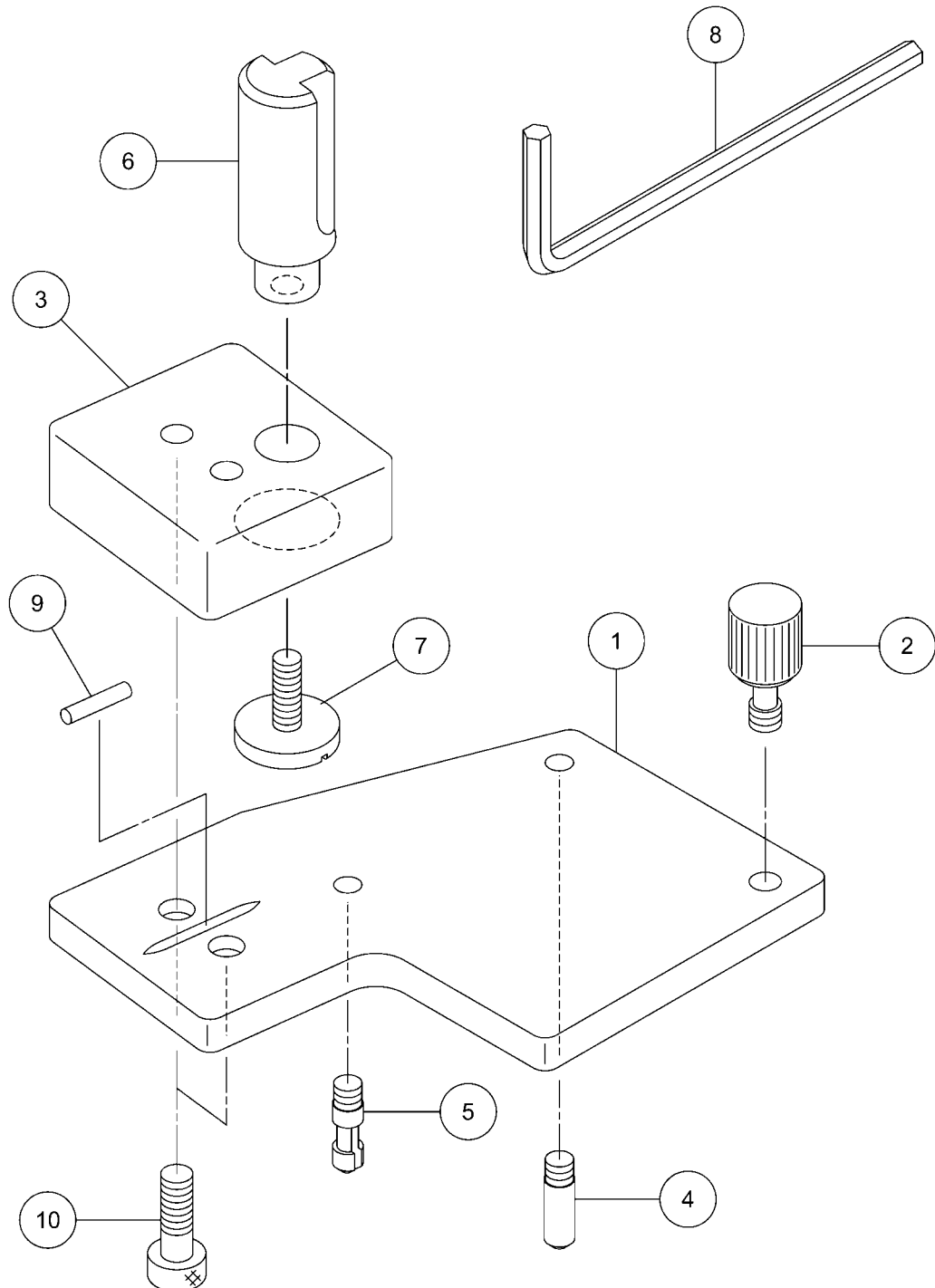


SO-TM3 (TONOMETER MOUNT, FOR R900)

Illust. Page 26

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44780 10070	Mount	1		
2	44780 10090	Pin	2		
3	44680 10150	Pin	1		
4	44680 10160	Pin	2		
5	44680 10170	Screw	1		
6	40391 12020	Shaft	1		
7	40391 12030	Pin	1		
8	44800 30030	Wrench	1		
9	6S4x10S	Socket bolt (BZn)	2		

SO-TM3 (TONOMETER MOUNT, FOR R900)

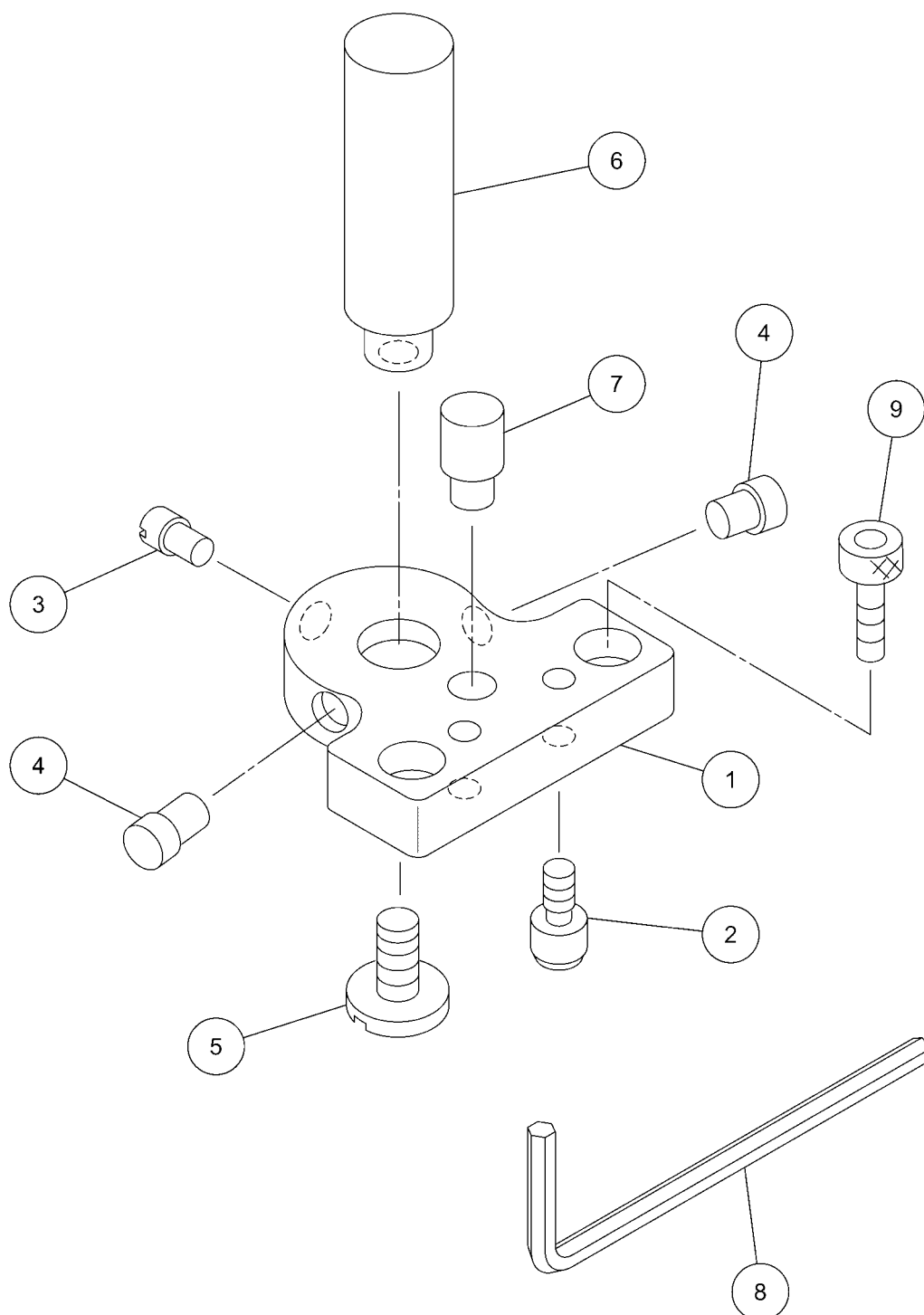


SO-TM4 (TONOMETER MOUNT, FOR 870)

Illust. Page 28

Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44795 13010	Base	1		
2	44795 13020	Bearing plate	1		
3	44795 13030	Shaft	1		
4	44780 10090	Pin	2		
5	40349 10070	Screw	1		
6	SP2x10S	Pin (SUS)	1		
7	6S4x8S	Socket bolt (SUS)	4		
8	41109 13820	Wrench	1		


SO-TM4 (TONOMETER MOUNT, FOR 870)

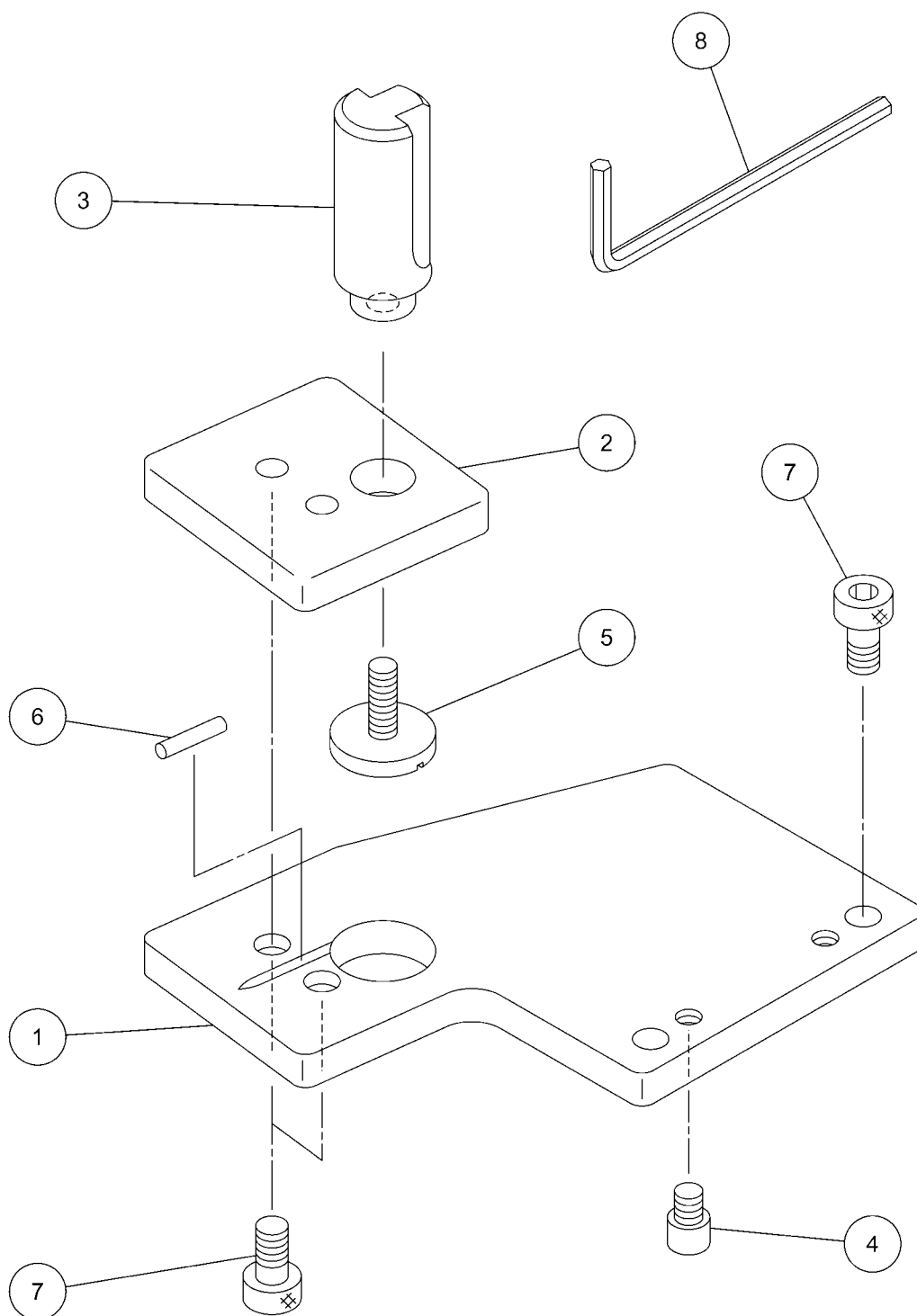

PG-1 (PATIENT HANDLE)

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	44795 30010	Arm	2		
2	44795 30020	Spacer	2		
3	44801 00280	Grip	2		
4	CQ4×6 SUS	CQ-screw	2		


PG-1 (PATIENT HANDLE)

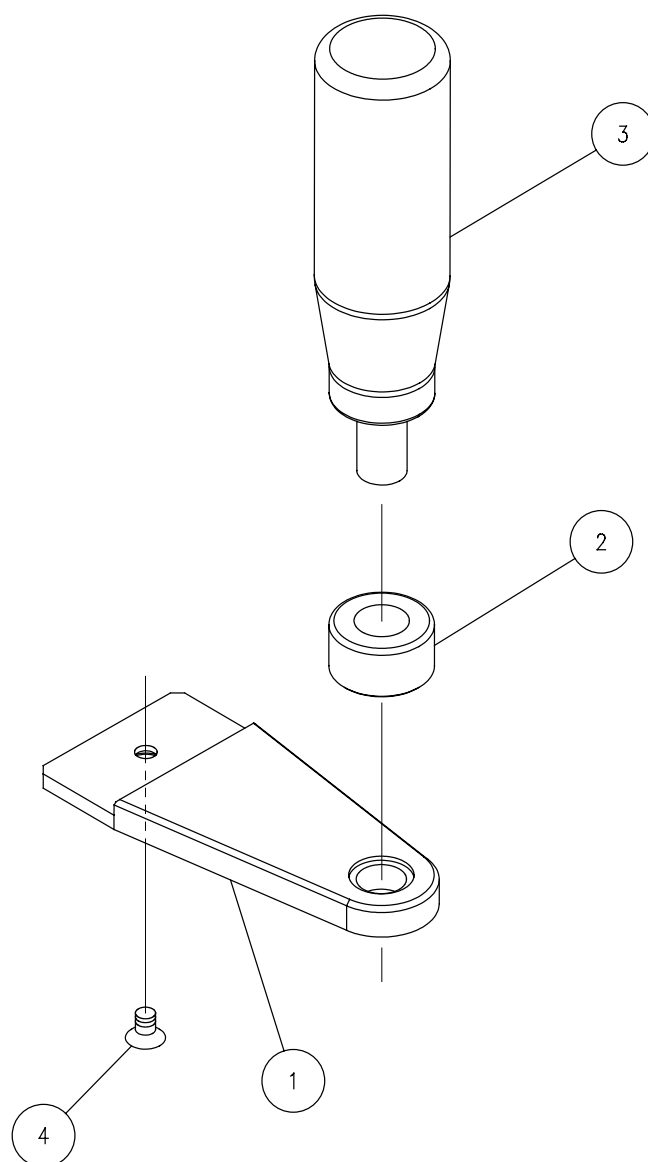

XR-1 (XENON RELAY LENS)

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	44691 21020	Lens Holder	1		
2	G524M 48310	Lens	2		
3	44691 21030	Spacer	1		
4	G604M 64580	Lens	2		
5	44691 21040	Spacer	1		
6	44691 21050	Spacer	1		
7	44691 21060	Ring	1		
8	CR3×12S BZn	CR-screw	1		
9	44790 40010	Lens Holder	1		
10	44691 21070	Knob	1		
11	6SV3×6S BZn	6SV-set screw	1		
12	44691 22200	Lens Assembly	1		1 ~ 7


XR-1 (XENON RELAY LENS)

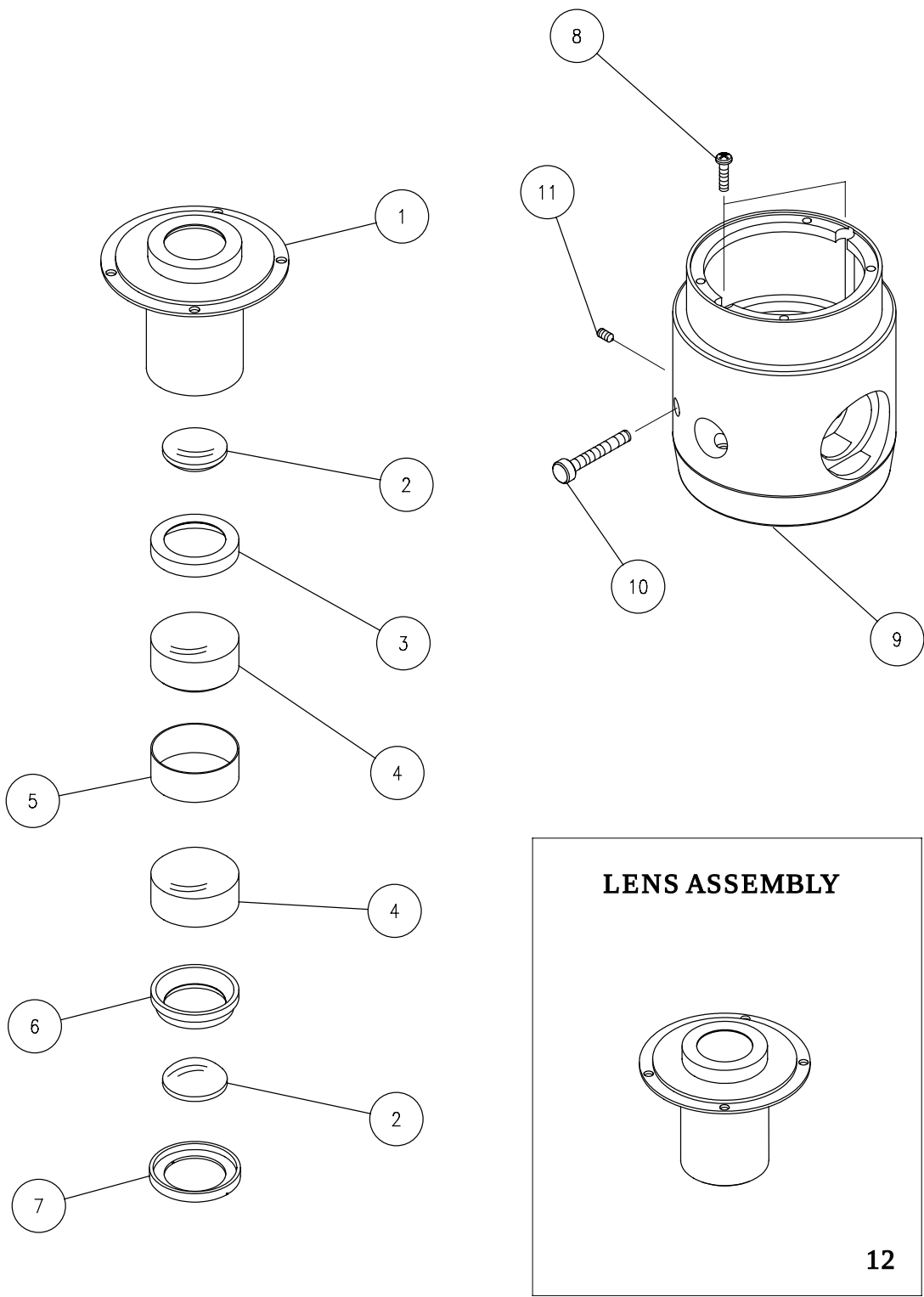

CABLE SUPPORT

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	44795 89010	Pipe	1		
2	44795 89020	Bush	1		
3	44795 89030	Plate	2		
4	CR3×5 SUS	CR-screw	2		

CABLE SUPPORT

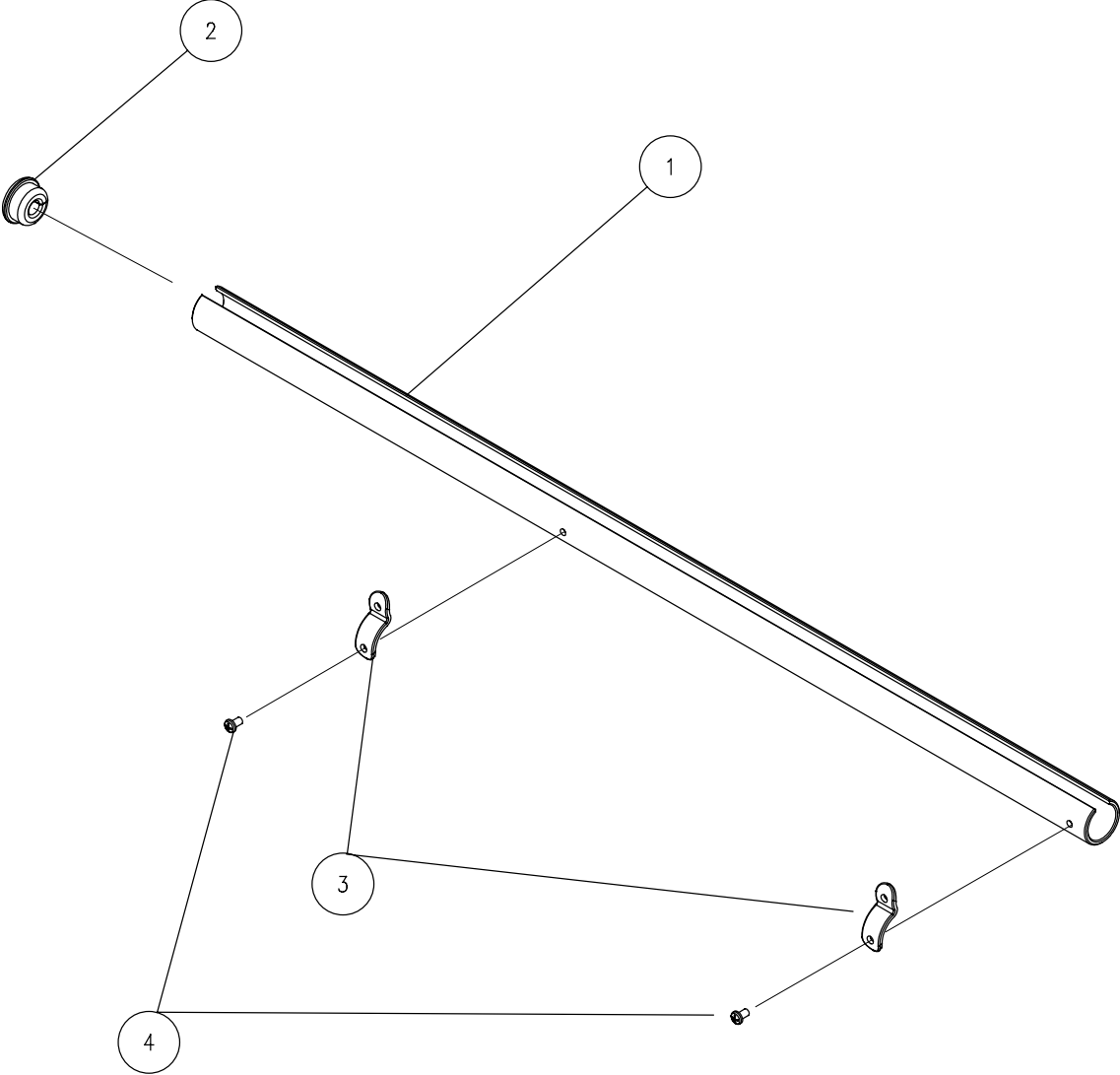


BG3

Illust. Page 36

(BACKGROUND ILLUMINATION)


Illust. No.	Parts No.	Name of Parts	Q'ty	News No.	Remarks
1	44790 41010	BGI case	1		
2	44790 41020	Lever	1		
3	44790 41030	Bottom plate	1		
4	44790 41040	BGI fixed pin	2		
5	40350 70030	Spring	1		
6	40350 70060	Light guide	1		
7	44691 70080	Adapter	1		
8	CR2x5S	CR-screw (BZn)	2		
9	6SV2x3S	6SV-set screw (BZn)	3		
10	Ball Sei3	Steel ball	1		

BG3 (BACKGROUND ILLUMINATION)

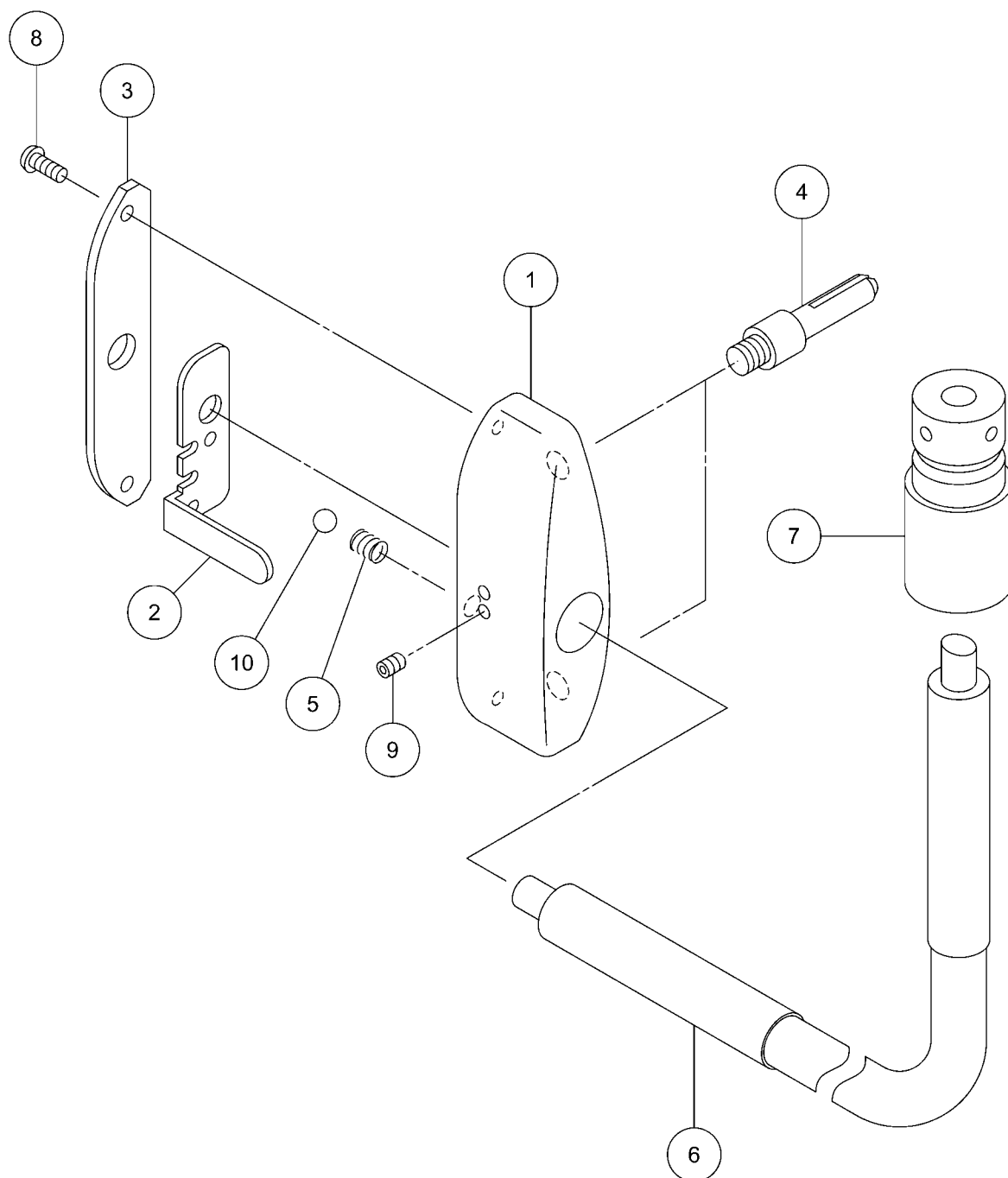

Confidential : duplication and reproduction prohibited


NIKON MOUNT

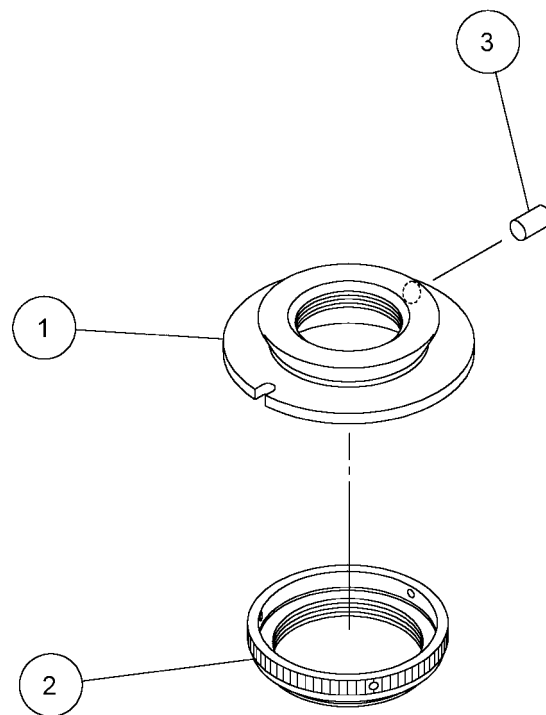

Illust. Page 40

Confidential : duplication and reproduction prohibited


FD-21 (FLASH DEVICE) VERTICAL TYPE

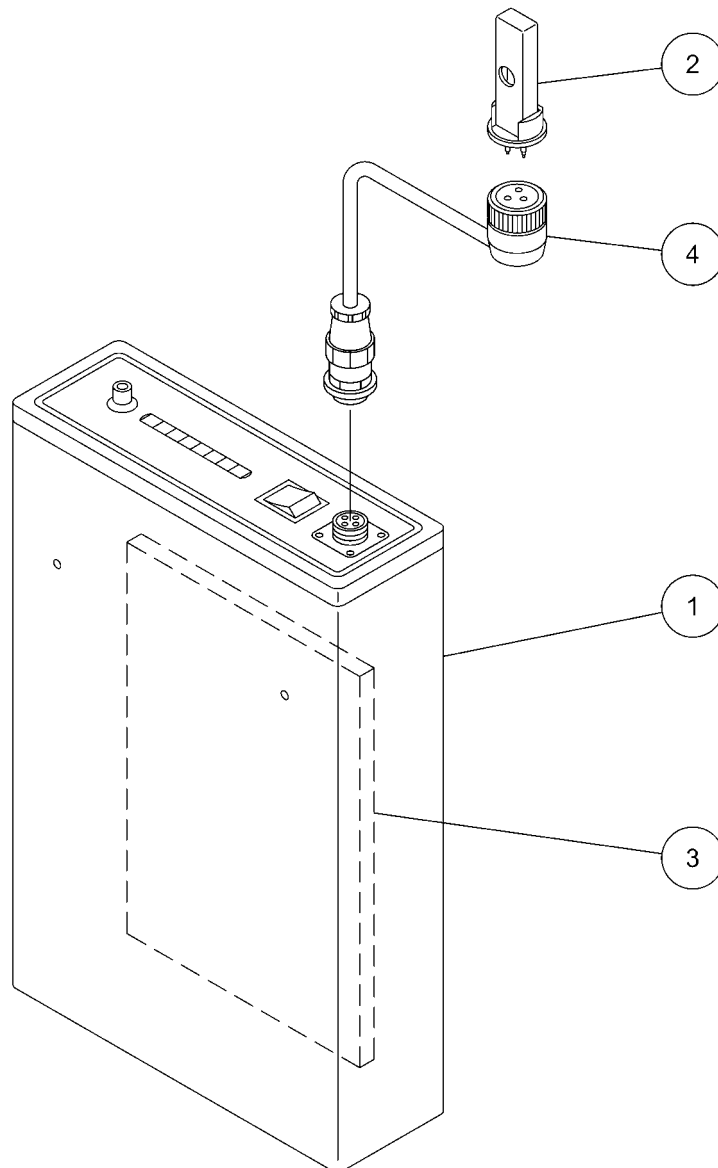

Illust. Page 42

Confidential : duplication and reproduction prohibited


FD-21 (FLASH DEVICE) HORIZONTAL TYPE

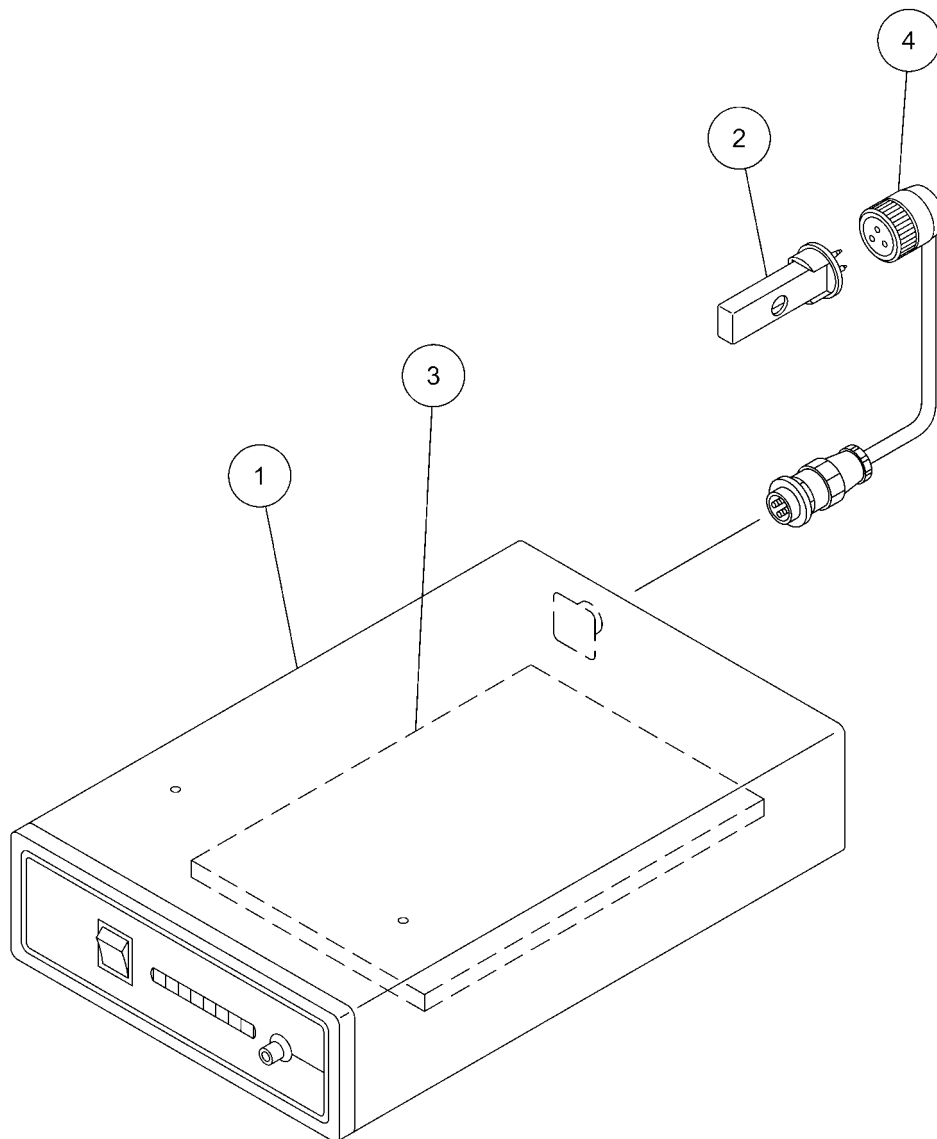

DC-1 (DIGITAL CAMERA UNIT) USB1.1

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	31012 10650	O Ring	1		
2	41820 85140	Cap	2		
3	44670 15230	Metal	1		
4	44693 60060	Holder	1		
5	44693 13030	Cell	1		
6	44693 13040	Cell	1		
7	44793 60010	Body	1		
8	44793 60020	Cover	1		
9	44793 60040	Box	1		
10	44793 60050	Holder	1		
11	44793 60150	Plate	1		
12	44793 60120	Cover	1		
13	44793 60140	Plate	1		
14	44793 60110	Cover	1		
15	44793 60170	Knob	1		
16	44793 60180	Cover	1		
17	44793 60190	Guide	1		
18	44793 60210	Cover	1		
19	44793 60260	Cover	1		
20	44793 70000	Main PCB	1		
21	44793 71000	Power PCB	1		
22	44793 60130	Sheeld Plate	1		
23	44793 72000	Keyboard PCB	1		
24	44800 90170	Screw	3		
25	44801 50120	Key Top	8		
26	45520 10312	Screw	1		
27	45520 11120	Plate	1		
28	G134M 71590	Lens	1		
29	G134M 71610	Lens	1		
30	G234M 71600	Lens	1		
31	G334M 66170	Prism	2		
32	G444M 72150	Mirror	1		
33	G434M 70470	IR Cut Filter	1		
34	G614M 69920	Prism	1		
35	44793 10110	Cell	1		
36	44793 10120	Spacer	1		
37	44793 10130	Spacer	1		
38	44793 10140	Ring	1		
39	44801 40570	Spacer	1		
40	44793 60290	Shutter	1		


DC-1 (DIGITAL CAMERA UNIT) USB1.1

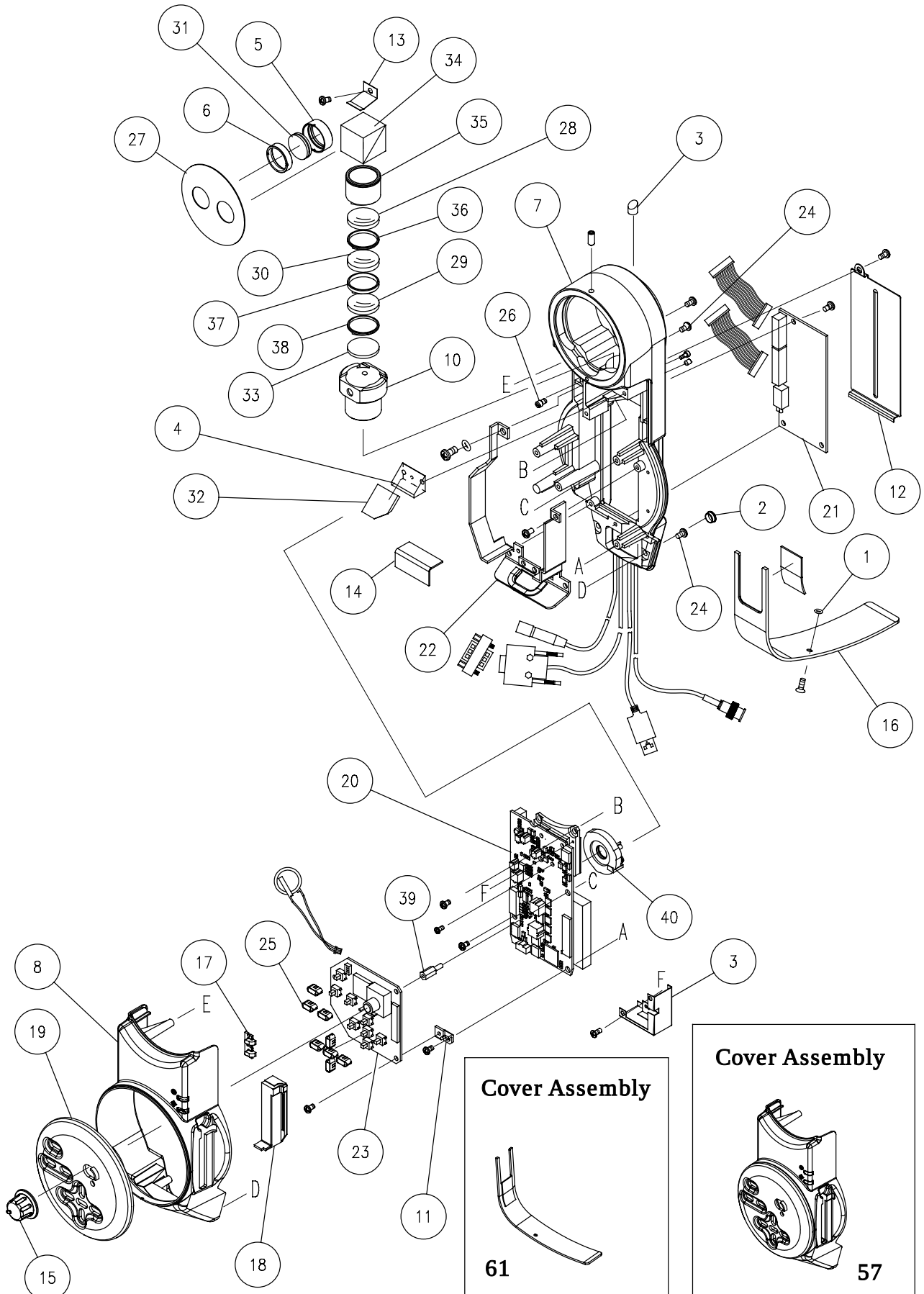

DC-1 (DIGITAL CAMERA UNIT) USB1.1

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
41	CR2×5S BZn	CR-screw	1		
42	6SU3×4S BZn	6SU-set screw	4		
43	6SV4×12 SUS	6SV-set screw	1		
44	CR2×3S BZn	CR-screw	2		
45	CR3×5S BZn	CR-screw	18		
46	CQ3×8 SUS	CQ-screw	1		
47	CR4×8S Ni	CR-screw	1		
48	CR2.5×5S BZn	CR-screw	3		
49	TBW 4S Ni	Washer	1		
50	44793 82000	Power Cable	1		
51	44793 83000	IF Cable	1		
52	44793 84000	USB Cable	1		
53	44793 85000	Video Cable	1		
54	44793 86000	Pmain Cable1	1		
55	44793 87000	Pmain Cable2	1		
56	44793 88000	Battery Unit	1		
57	44793 60100	Cover Assembly	1	8.17.18	
58	44793 83500	Dsub Adapter	1		
59	44801 70480	Rubber	1		
60	44793 60200	Cover Assembly	1	16.59	


DC-1 (DIGITAL CAMERA UNIT) USB1.1

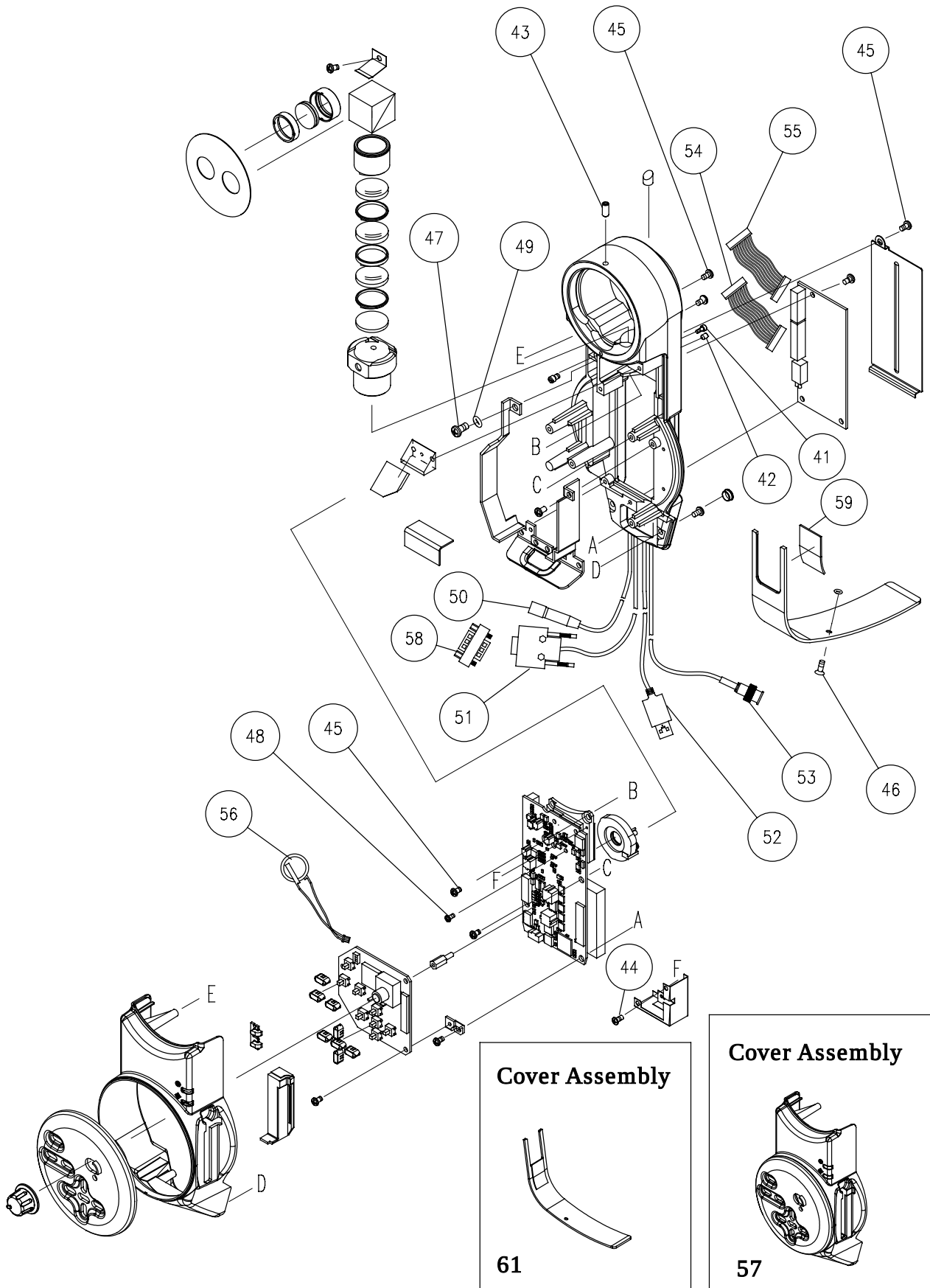

DC-1 (DIGITAL CAMERA UNIT) USB2.0

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	31012 10650	O Ring	1		
2	41820 85140	Cap	2		
3	44670 15230	Metal	1		
4	44693 60060	Holder	1		
5	44693 13030	Cell	1		
6	44693 13040	Cell	1		
7	44793 61010	Body	1		
8	44793 60020	Cover	1		
9	44793 60040	Box	1		
10	44793 60050	Holder	1		
11	44793 60150	Plate	1		
12	44793 60120	Cover	1		
13	44793 60140	Plate	1		
14	44793 60110	Cover	1		
15	44793 60170	Knob	1		
16	44793 60180	Cover	1		
17	44793 60190	Guide	1		
18	44793 60210	Cover	1		
19	44793 60260	Cover	1		
20	44793 73000	Main PCB	1		
21	44793 74000	Power PCB	1		
22	44793 61130	Sheeld Plate	1		
23	44793 72000	Keyboard PCB	1		
24	44800 90170	Screw	3		
25	44801 50120	Key Top	8		
26	45520 10312	Screw	1		
27	45520 11120	Plate	1		
28	G134M 71590	Lens	1		
29	G134M 71610	Lens	1		
30	G234M 71600	Lens	1		
31	G334M 66170	Prism	2		
32	G444M 72150	Mirror	1		
33	G434M 70470	IR Cut Filter	1		
34	G614M 69920	Prism	1		
35	44793 10110	Cell	1		
36	44793 10120	Spacer	1		
37	44793 10130	Spacer	1		
38	44793 10140	Ring	1		
39	44801 40570	Spacer	1		
40	44793 60290	Shutter	1		


DC-1 (DIGITAL CAMERA UNIT) USB2.0

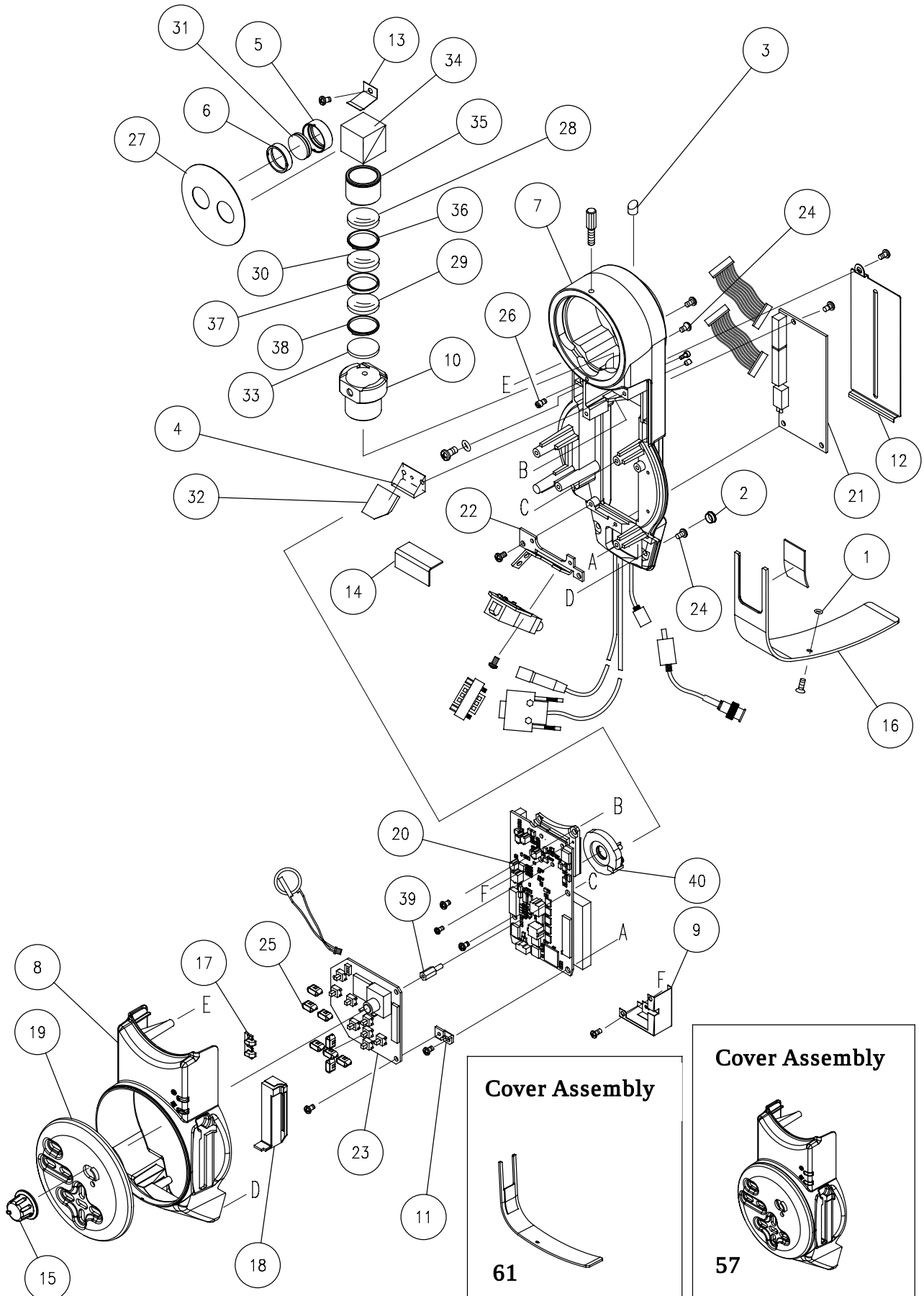

DC-1 (DIGITAL CAMERA UNIT) USB2.0

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
41	CR2×5S BZn	CR-screw	1		
42	6SU3×4S BZn	6SU-set screw	4		
43	44670 10220	Screw	1		
44	CR2×3S BZn	CR-screw	2		
45	CR3×5S BZn	CR-screw	20		
46	CQ3×8 SUS	CQ-screw	1		
47	CR4×8S Ni	CR-screw	1		
48	CR2.5×5S BZn	CR-screw	3		
49	TBW 4S Ni	Washer	1		
50	44793 82500	Power Cable	1		
51	44793 84500	IF Cable	1		
52	44793 86500	Video CHK Cable	1		
53	44793 85500	Video1 Cable	1		
54	44793 89500	Pmain Cable3	1		
55	44793 87000	Pmain Cable2	1		
56	44793 88000	Battery Unit	1		
57	44793 60100	Cover Assembly	1	8.17.18	
58	44793 87500	USB CHK PCB	1		
59	44793 83500	Dsub Adapter	1		
60	44801 70480	Rubber	1		
61	44793 60200	Cover Assembly	1	16.60	

DC-1 (DIGITAL CAMERA UNIT) USB2.0

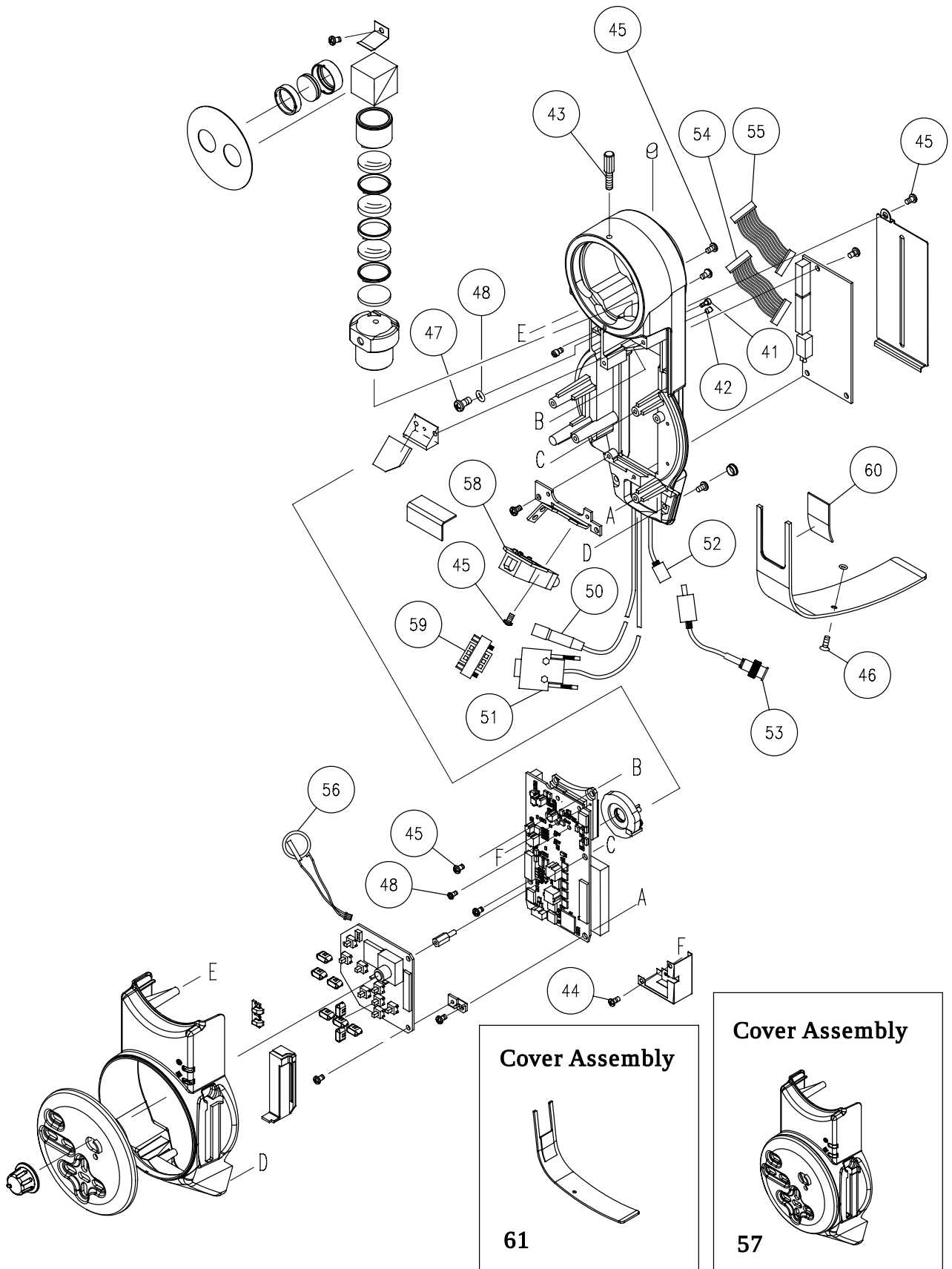


PB-2

Illust. Page 52

(PARALLEL BINOCULAR TUBES)

Illust. NO.	Part NO.	Name of Parts	Q' ty	News NO.	Remarks
1	44670 15160	Prism Holder	2		
2	44670 15180	Gum Plate	2		
3	G335M 53750	Prism	2		
4	G335M 48650	Prism	2		
5	CR3×5S BZn	CR-screw	4		
6	CR2×4S BZn	CR-screw	4		
7	44670 15190	Nut	2		
8	44670 15210	Holder	2		
9	CR2.5 ×6 SUS	CR-screw	4		
10	45562 10170	Prism Holder	2		
11	45562 10160	Gum	2		
12	45562 10010	Box(R)	1		
13	45562 10030	Cap(R)	1		
14	45562 10120	Spacer	4		
15	6SV3×5S BZn	SV-set screw	12		
16	CR2.6×15S BZn	CR-screw	6		
17	45562 10150	Spacer	6		
18	44670 15050	Gear	2		
19	44795 40010	Plate	2		
20	45562 10020	Box(L)	1		
21	45562 10040	Cap(L)	1		
22	6SV2.5×8S BZn	SV-set screw	1		
23	44670 15020	Lens Holder	2		
24	44670 15010	Lens Cell	2		
25	44670 15150	Ring	2		
26	G604M 48630	Lens	2		
27	45562 10140	Receptacle	1		
28	44670 15130	Washer	1		
29	44670 15230	Metal	1		
30	CR2.6×8S BZn	CR-screw	4		
31	44670 15140	Spring	1		
32	45562 10110	Washer	1		
33	45562 10130	Mount	1		
34	44795 40020	Washer	1		

PB-2 **(PARALLEL BINOCULAR TUBES)**

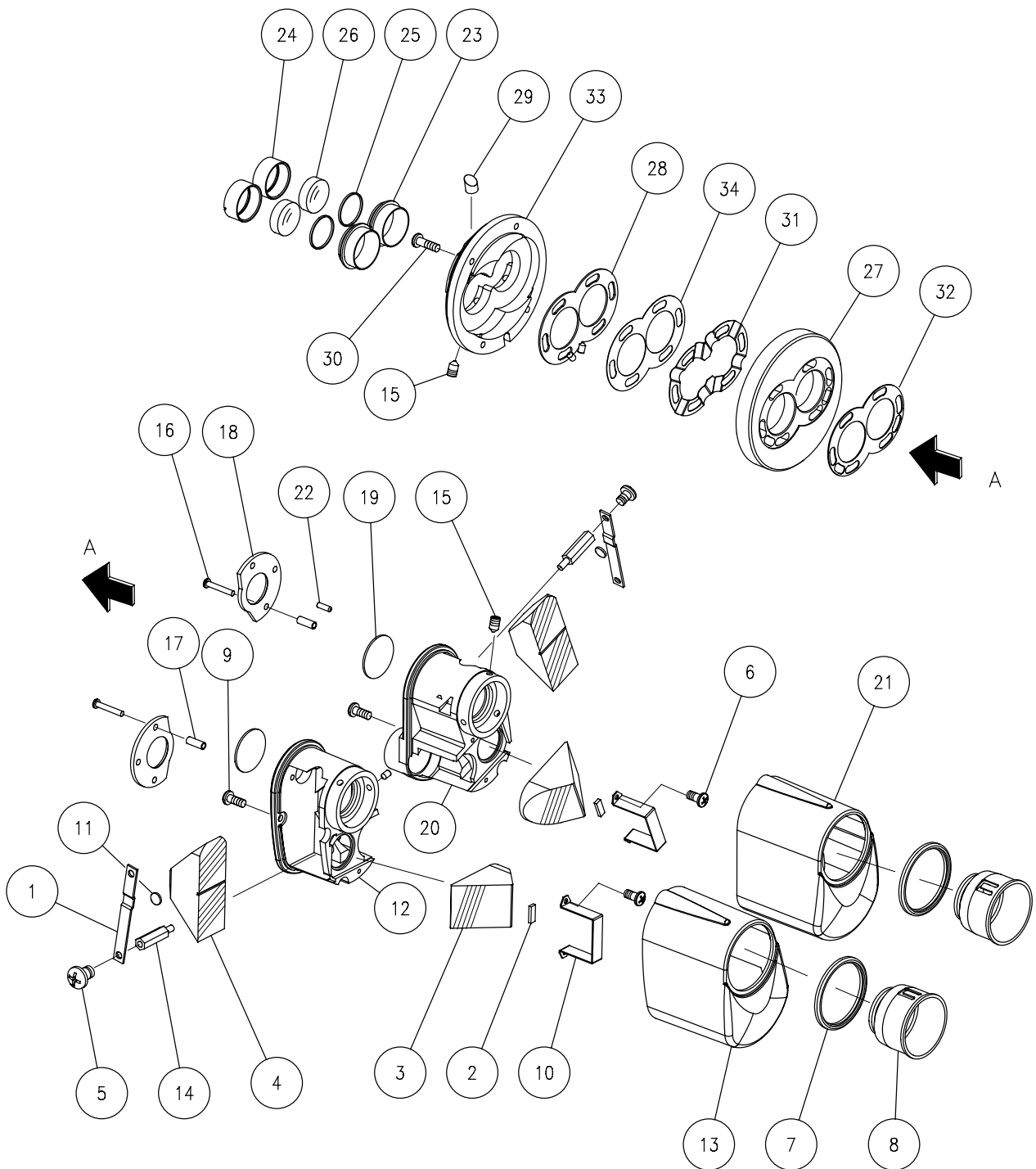

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
Ball Sei 3	20-40	Steel Ball	1	
Ball Sei3	36-10	Steel Ball	1	
30305 20590	20-9	Bis	1	
31012 10650	44-1	O Ring	1	
31012 10650	48-1	O Ring	1	
40340 20450	20-10	Ring	1	
40340 20460	20-11	Ring	1	
40340 20540	20-12	Spring	1	
40340 20550	20-13	Knob	1	
40349 10060	24-6	Pin	1	
40349 10070	24-7	Screw	1	
40349 10070	28-5	Screw	1	
40350 70030	20-35	Spring	1	
40350 70030	36-5	Spring	1	
40350 70060	36-6	Light Guide	1	
40365 40000	40-2	Strobo Iuminous Assembly	1	
40365 40000	42-2	Strobo Iuminous Assembly	1	
40391 12020	22-5	Pin	1	
40391 12020	26-6	Shaft	1	
40391 12030	22-6	Pin	1	
40391 12030	26-7	Pin	1	
40452 24030	2-24	Wire Sticker Wiring Parts	1	
41109 13820	24-8	Key	1	
41109 13820	28-8	Wrench	1	
41820 85140	44-2	Cap	2	
41820 85140	48-2	Cap	2	
44670 10220	16-15	Screw	1	
44670 10220	18-10	Screw	1	
44670 10220	2-31	Screw	2	
44670 10220	50-43	Screw	1	
44670 10220	6-16	Screw	2	
44670 15010	52-24	Lens Cell	2	
44670 15020	52-23	Lens Holder	2	
44670 15050	52-18	Gear	2	
44670 15130	52-28	Washer	1	
44670 15140	52-31	Spring	1	
44670 15150	52-25	Ring	2	
44670 15160	52-1	Prism Holder	2	
44670 15180	52-2	Gum Plate	2	
44670 15190	52-7	Nut	2	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
44670 15210	52-8	Holder	2	
44670 15230	12-13	Metal	1	
44670 15230	16-16	Metal	1	
44670 15230	18-11	Metal	1	
44670 15230	2-32	Metal	1	
44670 15230	44-3	Metal	1	
44670 15230	48-3	Metal	1	
44670 15230	52-29	Metal	1	
44670 15230	6-17	Metal	1	
44670 15230	8-13	Metal	1	
44670 15230	38-3	Pin	1	
44670 25650	20-14	Plug Pin	1	
44670 25660	20-15	Screw	2	
44670 25780	20-16	Knob	2	
44670 25940	20-17	Chip	2	
44670 25970	20-18	Plug Pin	1	
44670 25980	20-41	Connector	2	
44680 10150	26-3	Pin	1	
44680 10151	22-7	Pin	1	
44680 10160	26-4	Pin	2	
44680 10161	22-8	Pin	2	
44680 10170	26-5	Screw	1	
44680 25930	20-19	Terminal	1	
44680 25940	20-20	Connector Post	1	
44680 25950	20-21	Plug Base	1	
44691 10130	2-26	Solenoid Plate Pin	1	
44691 10190	2-27	Cover	1	
44691 10190	6-15	Cover	1	
44691 10210	2-10	Mirror Base	1	
44691 10220	2-13	Mirror Holder	1	
44691 10230	2-28	Pin	1	
44691 10240	2-14	Mirror Clamp	1	
44691 10250	2-19	Mirror Clamp	1	
44691 10260	2-15	Plate	2	
44691 10270	2-22	Plate	1	
44691 10280	4-57	Rivet	2	
44691 10290	4-58	Stopper	1	
44691 10310	4-59	Mirror Stopper	2	
44691 10320	4-60	Stopper Holder	1	
44691 10330	4-61	Stopper Shaft	1	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
44691 10340	4-62	Stopper Shaft Holder	1	1 ~ 7
44691 10350	2-12	Spring	1	
44691 10380	2-29	Spring Retainer	1	
44691 10390	2-30	Spring	1	
44691 10420	2-11	Washer	2	
44691 15050	2-25	Connector	1	
44691 21020	32-1	Lens Holder	1	
44691 21030	32-3	Spacer	1	
44691 21040	32-5	Spacer	1	
44691 21050	32-6	Spacer	1	
44691 21060	32-7	Ring	1	
44691 21070	32-10	Knob	1	
44691 22200	32-12	Lens Assembly	1	
44691 70080	20-37	Adapter	1	
44691 70080	36-7	Adapter	1	
44693 10140	12-8	Knob	1	
44693 10140	8-8	Knob	1	
44693 10170	12-12	Screw	1	
44693 10170	8-12	Screw	1	
44693 13030	44-5	Cell	1	
44693 13030	48-5	Cell	1	
44693 13040	44-6	Cell	1	
44693 13040	48-6	Cell	1	
44693 60060	44-4	Holder	1	
44693 60060	48-4	Holder	1	
44694 13030	16-18	Wedge Prism Cell	1	
44694 13030	18-12	Wedge Prism Cell	1	
44694 13030	6-20	Wedge Prism Cell	1	
44694 13040	16-19	Wedge Prism Cell	1	
44694 13040	18-13	Wedge Prism Cell	1	
44694 13040	6-21	Wedge Prism Cell	1	
44694 13090	16-12	Spacer Ring	1	
44694 13120	16-13	Retainer Ring	1	
44694 13160	16-14	Fixed Ring	1	
44694 15020	18-4	Prism Holder	1	
44694 15030	18-5	Spacer	1	
44694 15040	18-6	Mask Plate	1	
44694 15050	18-7	Mount	1	
44694 17020	6-11	Holder	1	
44694 30060	20-22	Cover	1	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
44694 31010	20-36	Light Guide	1	
44694 40010	20-23	Lens Cell	1	
44694 40020	20-24	Holder	1	
44694 40050	20-25	Lamp House	1	
44699 35040	12-14	Cell	1	
44699 35040	8-14	Cell	1	
44699 35050	12-15	Cell	1	
44699 35050	8-15	Cell	1	
44770 25150	20-26	Filter	1	
44770 25230	20-27	Spring	1	
44780 10070	26-1	Mount	1	
44780 10090	26-2	Pin	2	
44780 10090	28-4	Pin	2	
44790 10010	2-1	Body	1	
44790 10010	6-1	Body	1	
44790 10020	2-2	Mount	1	
44790 10020	6-2	Mount	1	
44790 10030	2-3	Mask	1	
44790 10030	6-3	Mask	1	
44790 10040	2-4	Cover	1	
44790 10040	6-4	Cover	1	
44790 10050	2-5	Cap	1	
44790 10050	6-5	Cap	1	
44790 10060	2-6	Lens Holder	1	
44790 10060	6-6	Lens Holder	1	
44790 10070	2-7	Lens Barrel	1	
44790 10070	6-7	Lens Barrel	1	
44790 10080	2-8	Retainer Ring -1	1	
44790 10080	6-8	Retainer Ring -1	1	
44790 10090	2-9	Retainer Ring -2	1	
44790 10090	6-9	Retainer Ring -2	1	
44790 10100	4-63	Mirror Holder Assembly	1	
44790 10170	2-16	Mirror Link	1	
44790 10180	6-10	Mirror Base Plate	1	
44790 10180	2-17	Mirror Frame	1	
44790 10190	2-18	Solenoid Arm	1	
44790 10220	2-20	Solenoid Holder	1	
44790 10230	2-21	Rotary Solenoid	1	
44790 10260	4-64	Plate	1	
44790 10260	6-34	Plate	1	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
44790 40010	32-9	Lens Holder	1	For Japan, TMS, TS, GEN
44790 41010	36-1	BGI Case	1	
44790 41020	36-2	Lever	1	
44790 41030	36-3	Bottom Plate	1	
44790 41040	36-4	BGI Fixed Pin	2	
44790 50000	40-1	Flash Drive	1	
44790 50100	40-3	Control PCB	1	
44790 50100	42-3	Control PCB	1	
44790 50200	40-4	Xenon Cable	1	For TE
44790 50200	42-4	Xenon Cable	1	
44790 54000	42-1	Flash Drive	1	
44790 80010	38-1	Mount Adapter	1	
44790 80020	38-2	Mount Nikon	1	
44793 10010	12-16	Body	1	
44793 10010	8-16	Body	1	
44793 10020	12-17	Mount	1	
44793 10020	8-17	Mount	1	
44793 10030	12-18	Slider	1	
44793 10030	8-18	Slider	1	
44793 10040	10-45	Plate	1	
44793 10040	14-45	Plate	1	
44793 10050	12-19	Pin	1	
44793 10050	8-19	Pin	1	
44793 10060	12-35	Ring	1	
44793 10060	8-35	Ring	1	
44793 10070	12-20	Plate	1	
44793 10070	8-20	Plate	1	
44793 10080	12-21	Shaft	1	
44793 10080	8-21	Shaft	1	
44793 10090	12-22	Shaft	1	
44793 10090	8-22	Shaft	2	
44793 10110	12-23	Cell	1	
44793 10110	44-35	Cell	1	
44793 10110	48-35	Cell	1	
44793 10110	8-23	Cell	1	
44793 10120	12-24	Spacer	1	
44793 10120	44-36	Spacer	1	
44793 10120	48-36	Spacer	1	
44793 10120	8-24	Spacer	1	
44793 10130	12-25	Spacer	1	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
44793 10130	44-37	Spacer	1	
44793 10130	48-37	Spacer	1	
44793 10130	8-25	Spacer	1	
44793 10140	12-26	Ring	1	
44793 10140	44-38	Ring	1	
44793 10140	48-38	Ring	1	
44793 10140	8-26	Ring	1	
44793 10150	12-27	Cover	1	
44793 10150	8-27	Cover	1	
44793 10160	12-28	Cover	1	
44793 10160	8-28	Cover	1	
44793 10170	12-36	Body	1	
44793 10170	8-36	Body	1	
44793 10180	12-37	Cell	1	
44793 10180	8-37	Cell	1	
44793 10190	12-38	Spacer	1	
44793 10190	8-38	Spacer	2	
44793 10210	12-39	Cam	1	
44793 10210	8-39	Cam	1	
44793 10220	12-40	Ring	1	
44793 10220	8-40	Ring	1	
44793 10230	10-41	Plate	2	
44793 10230	14-41	Plate	2	
44793 10240	10-42	Pin	2	
44793 10240	14-42	Pin	2	
44793 10250	10-43	Pin	1	
44793 10250	14-43	Pin	1	
44793 10260	10-44	Holder	1	
44793 10260	14-44	Holder	1	
44793 10270	10-46	Cover	1	
44793 10270	14-46	Cover	1	
44793 10280	10-47	Cover	1	
44793 10280	14-47	Cover	1	
44793 11010	16-1	Body	1	
44793 11020	16-2	Prism Mount	1	
44793 11030	16-3	Mirror Box Cover	1	
44793 11040	16-4	Mirror Box	1	
44793 11050	16-5	Mirror Mount	1	
44793 11060	16-6	Lens Holder	1	
44793 11070	16-7	Mount	1	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
44793 11080	16-8	Cover	1	
44793 11090	16-9	Cam screw	1	
44793 11110	16-10	Adjust Ring	1	
44793 12010	18-1	Body	1	
44793 12020	18-2	Arm Cover	1	
44793 12030	18-3	Cord Bush	1	
44793 30010	10-54	Adapter	1	
44793 30020	10-55	Lens cell	1	
44793 30030	10-56	Cover	1	
44793 30030	14-54	Cover	1	
44793 30040	10-57	Mount	1	
44793 30040	14-55	Mount	1	
44793 30050	10-58	Base	1	
44793 30050	14-56	Base	1	
44793 31010	14-57	Adapter	1	
44793 31020	14-58	Cell	1	
44793 31030	14-59	Spacer	1	
44793 40010	20-28	Lamp Cover	1	
44793 60010	44-7	Body	1	
44793 60020	44-8	Cover	1	
44793 60020	48-8	Cover	1	
44793 60040	44-9	Box	1	
44793 60040	48-9	Box	1	
44793 60050	44-10	Holder	1	
44793 60050	48-10	Holder	1	
44793 60100	46-57	Cover Assembly	1	
44793 60100	50-57	Cover Assembly	1	
44793 60110	44-14	Cover	1	
44793 60110	48-14	Cover	1	
44793 60120	44-12	Cover	1	
44793 60120	48-12	Cover	1	
44793 60130	44-22	Shield Plate	1	
44793 60140	44-13	Plate	1	
44793 60140	48-13	Plate	1	
44793 60150	44-11	Plate	1	
44793 60150	48-11	Plate	1	
44793 60170	44-15	Knob	1	
44793 60170	48-15	Knob	1	
44793 60180	44-16	Cover	1	
44793 60180	48-16	Cover	1	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
44793 60190	44-17	Guide	1	
44793 60190	48-17	Guide	1	
44793 60200	46-60	Cover Assembly	1	
44793 60200	50-61	Cover Assembly	1	
44793 60210	44-18	Cover	1	
44793 60210	48-18	Cover	1	
44793 60260	44-19	Cover	1	
44793 60260	48-19	Cover	1	
44793 60290	44-40	Shutter	1	
44793 60290	48-40	Shutter	1	
44793 61010	48-7	Body	1	
44793 61130	48-22	Sheeld Plate	1	
44793 70000	44-20	Main PCB	1	
44793 71000	44-21	Power PCB	1	
44793 72000	44-23	Keyboard PCB	1	
44793 72000	48-23	Keyboard PCB	1	
44793 73000	48-20	Main PCB	1	
44793 74000	48-21	Power PCB	1	
44793 82000	46-50	Power Cable	1	
44793 82500	50-50	Power Cable	1	
44793 83000	46-51	IF Cable	1	
44793 83500	46-58	Dsub Adapter	1	
44793 83500	50-59	Dsub Adapter	1	
44793 84000	46-52	USB Cable	1	
44793 84500	50-51	IF Cable	1	
44793 85000	46-53	Video Cable	1	
44793 85500	50-53	Video1 Cable	1	
44793 86000	46-54	Pmain Cable1	1	
44793 86500	50-52	Video CHK Cable	1	
44793 87000	46-55	Pmain Cable2	1	
44793 87000	50-55	Pmain Cable2	1	
44793 87500	50-58	USB CHK PCB	1	
44793 88000	46-56	Battery Unit	1	
44793 88000	50-56	Battery Unit	1	
44793 89500	50-54	Pmain Cable3	1	
44795 10010	22-1	Tonometer Mount Base	1	
44795 10030	22-3	Mount Knob	1	
44795 10030	24-4	Pin	1	
44795 10040	24-5	Pin	1	
44795 10040	22-4	Shaft	1	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
44795 11010	24-1	Base	1	
44795 11020	24-2	Screw	1	
44795 11030	24-3	Bearing Plate	1	
44795 13010	28-1	Base	1	
44795 13020	28-2	Bearing Plate	1	
44795 13030	28-3	Shaft	1	
44795 30010	30-1	Arm	2	
44795 30020	30-2	Spacer	2	
44795 40010	52-19	Plate	2	
44795 40020	52-34	Washer	1	
44795 89010	34-1	Pipe	1	
44795 89020	34-2	Bush	1	
44795 89030	34-3	Plate	2	
44800 30030	26-8	Wrench	1	
44800 90170	44-24	Screw	3	
44800 90170	48-24	Screw	3	
44801 00280	30-3	Grip	2	
44801 40080	2-35	Spacer	2	
44801 40080	6-12	Spacer	2	
44801 40510	2-36	Spacer	2	
44801 40510	6-13	Spacer	2	
44801 40570	44-39	Spacer	1	
44801 40570	48-39	Spacer	1	
44801 40780	2-37	Spacer	1	
44801 40780	6-14	Spacer	1	
44801 50120	44-25	Key Top	8	
44801 50120	48-25	Key Top	8	
44801 70480	46-59	Rubber	1	
44801 70480	50-60	Rubber	1	
44802 40040	12-29	Plunger	1	
44802 40040	8-29	Plunger	1	
44802 60160	12-30	Clamp	1	
44802 60160	8-30	Clamp	1	
44942 31520	2-23	Receptacle	1	
45520 10310	12-11	Screw	2	
45520 10310	16-17	Screw	1	
45520 10310	18-8	Screw	1	
45520 10310	2-33	Screw	2	
45520 10310	6-18	Screw	2	
45520 10310	8-11	Screw	2	

GENERAL PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
45520 10312	44-26	Screw	1	
45520 10312	48-26	Screw	1	
45520 11120	12-31	Plate	1	
45520 11120	16-11	Plate	1	
45520 11120	18-9	Plate	1	
45520 11120	2-34	Plate	1	
45520 11120	44-27	Plate	1	
45520 11120	48-27	Plate	1	
45520 11120	6-19	Plate	1	
45520 11120	8-31	Plate	1	
45562 10010	52-12	Box(R)	1	
45562 10020	52-20	Box(L)	1	
45562 10030	52-13	Cap(R)	1	
45562 10040	52-21	Cap(L)	1	
45562 10110	52-32	Washer	1	
45562 10120	52-14	Spacer	4	
45562 10130	52-33	Mount	1	
45562 10140	52-27	Receptacle	1	
45562 10150	52-17	Spacer	6	
45562 10160	52-11	Gum	2	
45562 10170	52-10	Prism Holder	2	
49790 41010	20-31	Case	1	
49790 41020	20-32	Lever	1	
49790 41030	20-33	Plate	1	
49790 41040	20-34	Pin	2	

GLASS PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
G123M 20700	20-29	Lens	1	
G124M 20710	20-30	Lens	1	
G133M 71620	12-33	Lens	1	
G133M 71620	8-33	Lens	2	
G133M 71670	14-61	Lens	1	
G134M 66160	16-20	Lens	1	
G134M 71590	12-1	Lens	1	
G134M 71590	44-28	Lens	1	
G134M 71590	48-28	Lens	1	
G134M 71590	8-1	Lens	1	
G134M 71610	44-29	Lens	1	
G134M 71610	48-29	Lens	1	
G143M 71610	12-2	Lens	1	
G143M 71610	8-2	Lens	1	
G234M 71600	44-30	Lens	1	
G234M 71600	48-30	Lens	1	
G234M 71610	12-10	Lens	1	
G234M 71610	8-10	Lens	1	
G234M 72100	2-39	Lens	1	
G234M 72100	6-26	Lens	1	
G334M 66170	12-32	Prism	2	
G334M 66170	16-23	Prism	2	
G334M 66170	18-15	Prism	2	
G334M 66170	44-31	Prism	2	
G334M 66170	48-31	Prism	2	
G334M 66170	6-23	Prism	1	
G334M 66170	8-32	Prism	2	
G335M 48650	52-4	Prism	2	
G335M 53750	52-3	Prism	2	
G434M 70470	44-33	IR Cut Filter	1	
G434M 70470	48-33	IR Cut Filter	1	
G444M 48960	2-40	Mirror	1	
G444M 49000	4-41	Mirror	1	
G444M 49000	6-24	Mirror	1	
G444M 58030	12-9	Mirror	1	
G444M 58030	8-9	Mirror	1	
G444M 66120	16-24	Mirror	1	
G444M 69910	18-17	Mirror	1	
G444M 72150	44-32	Mirror	1	
G444M 72150	48-32	Mirror	1	

GLASS PARTS

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
G524M 48310	32-2	Lens	2	
G603M 71640	12-34	Lens	1	
G603M 71640	8-34	Lens	2	
G603M 71690	14-60	Lens	1	
G604M 48630	52-26	Lens	2	
G604M 64580	32-4	Lens	2	
G604M 66140	16-21	Lens	1	
G604M 69890	18-16	Lens	1	
G604M 72080	2-38	Lens	1	
G604M 72080	6-25	Lens	1	
G614M 69920	12-7	Prism	1	
G614M 69920	16-22	Prism	1	
G614M 69920	18-14	Prism	1	
G614M 69920	44-34	Prism	1	
G614M 69920	48-34	Prism	1	
G614M 69920	6-22	Prism	1	
G614M 69920	8-7	Prism	1	

SCREW AND WASHERS,E RING

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
CR2 × 3S BZn	4-42	CR-screw	3	
CR2 × 3S BZn	46-44	CR-screw	2	
CR2 × 3S BZn	50-44	CR-screw	2	
CR2 × 4S BZn	4-44	CR-screw	4	
CR2 × 5S BZn	36-8	CR-screw	2	
CR2 × 5S Ni	4-43	CR-screw	4	
CR2 × 4S BZn	52-6	CR-screw	4	
CR2 × 4 SUS	16-25	CR-screw	1	
CR2 × 5S BZn	20-38	CR-screw	2	
CR2 × 5S BZn	46-41	CR-screw	1	
CR2 × 5S BZn	50-41	CR-screw	1	
CR2 × 8S BZn	4-45	CR-screw	4	
CR2 × 8S BZn	6-27	CR-screw	4	
CR2.5 × 5S BZn	46-48	CR-screw	3	
CR2.5 × 5S BZn	50-48	CR-screw	3	
CR2.5 × 6 SUS	52-9	CR-screw	4	
CR2.6 × 8S BZn	52-30	CR-screw	4	
CR2.6 × 15S BZn	52-16	CR-screw	6	
CR3 × 5 SUS	34-4	CR-screw	2	
CR3 × 5S BZn	6-28	CR-screw	3	
CR3 × 5S BZn	16-26	CR-screw	3	
CR3 × 5S BZn	46-45	CR-screw	18	
CR3 × 5S BZn	50-45	CR-screw	20	
CR3 × 5S BZn	52-5	CR-screw	4	
CR3 × 8S SUS	8-3	CR-screw	2	
CR3 × 8S SUS	12-3	CR-screw	2	
CR3 × 8SUS	6-29	CR-screw	3	
CR3 × 5S BZn	4-46	CR-screw	3	
CR3 × 8S BZn	18-18	CR-screw	1	
CR3 × 8SUS	4-47	CR-screw	3	
CR3 × 12S BZn	32-8	CR-screw	1	
CR3 × 16SUS	6-30	CR-screw	3	
CR4 × 8S BZn	20-5	CR-screw	2	
CR4 × 8S Ni	46-47	CR-screw	1	
CR4 × 8S Ni	50-47	CR-screw	1	
CQ3 × 5 SUS	4-48	CQ-screw	2	
CQ3 × 5 SUS	6-31	CQ-screw	4	
CQ3 × 5 SUS	18-20	CQ-screw	2	
CQ3 × 8 SUS	4-49	CQ-screw	2	
CQ3 × 8 SUS	46-46	CQ-screw	1	

SCREW AND WASHERS,E RING

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
CQ3 × 8 SUS	50-46	CQ-screw	1	
CQ4 × 6 SUS	30-4	CQ-screw	2	
CT4 × 20SUS	22-2	CT-screw	1	
TPs2 × 4S BZn	20-2	TPs-screw	2	
6S2 × 5 BZn	10-49	Socket-bolt	1	
6S2 × 5 BZn	14-49	Socket-bolt	1	
6S2 × 8S BZn	10-51	Socket-bolt	3	
6S2 × 8S BZn	14-51	Socket-bolt	3	
6S3 × 5S BZn	6-32	Socket bolt	1	
6S3 × 5S BZn	8-6	Socket-bolt	7	
6S3 × 5S BZn	12-6	Socket-bolt	7	
6S3 × 5S BZn	4-52	Socket bolt	1	
6S3 × 8 SUS	16-27	Socket bolt	2	
6S4 × 8S SUS	28-7	Socket bolt	4	
6S4 × 10S BZn	26-9	Socket bolt	2	
6S4 × 12S BZn	22-9	Socket bolt	1	
6S4 × 12S BZn	24-10	Socket bolt	2	
U2.6 × 6S BZn	4-53	U-set screw	1	
U2 × 4S BZn	4-56	U-set screw	2	
6SU2 × 3S BZn	16-28	6SU-set screw	3	
6SU3 × 4S BZn	46-42	6SU-set screw	4	
6SU3 × 4S BZn	50-42	6SU-set screw	4	
6SU3 × 5S BZn	4-50	6SU-set screw	1	
6SU3 × 5S BZn	18-19	6SU-set screw	4	
6SU3 × 6S BZn	6-33	6SU-set screw	3	
6SU3 × 6S BZn	20-8	6SU-set screw	2	
6SV2 × 3S BZn	20-39	6SV-set screw	3	
6SV2 × 3S BZn	36-9	6SV-set screw	3	
6SV2.5 × 8S BZn	52-22	SV-set screw	1	
6SV3 × 4S BZn	8-5	6SV-set screw	7	
6SV3 × 4S BZn	10-50	6SV-set screw	2	
6SV3 × 4S BZn	12-5	6SV-set screw	7	
6SV3 × 4S BZn	14-50	6SV-set screw	2	
6SV3 × 4S BZn	16-29	6SV-set screw	3	
6SV3 × 4S BZn	20-4	6SV-set screw	2	
6SV3 × 5S BZn	52-15	SV-set screw	12	
6SV3 × 6S BZn	32-11	6SV-set screw	1	
6SV4 × 12 SUS	46-43	6SV-set screw	1	
FW2S BZn	10-53	Washer	4	
FW2S BZn	14-53	Washer	4	

SCREW AND WASHERS,E RING

Parts No.	Illust.No.	Name of Parts	Q'ty	Remarks
FW3S BZn	12-4	Washer	2	
FW3S BZn	8-4	Washer	2	
FW4S BZn	22-10	Washer	1	
FWs3S BZn	10-52	Washer	3	
FWs3S BZn	14-52	Washer	3	
SW2S BZn	10-48	Spring-washer	1	
SW2S BZn	14-48	Spring-washer	1	
SW3S Ni	20-3	Spring washer	1	
SW4S BZn	20-6	Spring washer	2	
SW4S Ni	20-7	Spring washer	2	
1N2.6S BZn	4-51	Nut	1	
1N3S Ni	20-1	Nut	1	
EJ1.5S BZn	4-54	E-Ring	2	
EJ2S BZn	4-55	E-Ring	2	
SP2 × 10S SUS	28-6	Pin	1	
SP2 × 10S SUS	24-9	Pin	1	
TBW 4S Ni	46-49	Washer	1	
TBW 4S Ni	50-49	Washer	1	

TOPCON MEDICAL SYSTEMS, INC.

37 West Century Road, Paramus, New Jersey 07652, U.S.A. Phone: 201-261-9450 Fax: 201-387-2710 www.topcon.com

TOPCON CANADA INC.

110 Provencher Avenue, Boisbriand, QC J7G 1N1 CANADA Phone: 450-430-7771 Fax: 450-430-6457 www.topcon.com

TOPCON OMNI SYSTEMS, INC.

Valley Forge Business Center, 2430 Blvd. Of the Generals, Norristown, PA 19403, U.S.A. Phone: 610-630-9200 Fax: 610-630-6428 www.topconomni.com

TOPCON EUROPE B.V. (European Representative)

Esse Baan 11, 2908 LJ Capelle a/d IJssel, The Netherlands. Phone: 010-4585077 Fax: 010-4585045 www.topconeurope.com

ITALY OFFICE : Via Monfalcone 39, 20092 Cinisello B. mo (MI) ITALY Pnone: 02-61-25-583 Fax: 02-61-25-927

TOPCON DEUTCHLAND G.m.b.H.

Weidkamp D-180 45356 Essen, Germany. Phone: 0201-8619-200 Fax: 0201-8619-275 www.topcon.de Med@topcon.de

TOPCON S.A.R.L.

HEAD OFFICE : 89, Rue de Paris, 92585 Clichy, Cedex, France. Phone: 01-4106-9494 Fax: 01-4739-0251

LYON OFFICE : 138, Avenue du 8 mai 1945, 69100 Villeurbanne, France. Phone: 047868-8237 Fax: 047868-1902

TOPCON ESPANA S.A.

HEAD OFFICE : Frederic Mompou 5, ED. Euro 3, 08960, Sant Just Desvem Barcelona, **Spain**. **Phone: 93-4734057 Fax: 93-4733932** www.topcon.com

MADRID OFFICE : Avenida Burgos, 16E, 1 ° 28036, Madrid, Spain.

Phone: 91-302-4129 Fax: 91-383-3890

TOPCON SCANDINAVIA A, B

Neongatan 2, S-43151 Molndal, Sweden. Phone: 031-7109200 Fax: 031-7109249 info@topcon.se

TOPCON (GREAT BRITAIN) LTD

Topcon House, Kennet Side, Bone Lane, Newbury, Berkshire RG14 5PX U.K. Phone: 01635-551120 Fax: 01635-551170

TOPCON SOUTH ASIS PTE. LTD.

Blk 192 Pandan Loop #07-01 Pantech industrial Complex SINGAPORE 128381 Phone: 62780222 Fax: 62733540 web.signet.com.sg/~topconts

TOPCON INSTRUMENTS (MALAYSIA) SDN.BHD.

Excella Bussiness Park Block C, Ground 1st floor Jalan Ampang Putra, Taman Ampang Hillir, 55100 Kuala Lumpur, Malaysia Phone: 03-42701068 Fax: 03-42704508

TOPCON INSTRUMENTS (THAILAND) CO., LTD.

77/162 Sinn Sathom Tower, 37th Fl., Krungdhonburi Rd., Klonglonsai, Klongsam, Bangkok 10600 Thailand. Phone: 440-1152 ~ 7 Fax: 440-1158

TOPCON AUSTRALIA PTY. LTD.

408 Victoria Road, Gladesville, NSW2111, Australia Phone: 02-9817-4666 Fax: 02-9817-4654 www.topcon.com.au

TOPCON KOREA CORPORATION

2F Yooseoung Bldg., 1593-3, Seocho-Dong, Seocho-Gu, Seoul, 137-876 KOREA Phone: 02-2055-0321 Fax+03-2055-0319 www.topcon.co.kr

TOPCON OPTICAL (H.K.) LIMITED

2/F., Meeco Industrial Bldg., NO.53-55 Au Pui Wan Street, Fo Tan Road, Shatin, N.T., Hong Kong Phone: 2690-1328 Fax: 2690-2221 E-mail: sales@topcon.com.hk

TOPCON CORPORATION BEIJING OFFICE

Room No. 962 Poly Plaza Building, 14 Dongzhimen Nandajie, Dongcheng District, Beijing, 100027, China Phone: 10-6501-4191 ~ 2 Fax: 10-6501-4190

TOPCON CORPORATION BEIRUT OFFICE

P.O. BOX 70-1002 Antelias, BEIRUT-LEBANON. Phone: 961-4-523525/523526 Fax: 961-4-521119

TOPCON CORPORATION DUBAI OFFICE

C/O Atlas Medical FZCO., P.O.Box 54304, C-25, Dubai Airport Free Zone, UAE Phone: 971-4-2995900 Fax: 971-4-2995901

TOPCON CORPRATION

75-1 Hasunuma-cho, Itabashi-ku, Tokyo, 174-8580 Japan.

Phone: 3-3558-2520 Fax: 3-3960-4214 www.topcon.co.jp

Printed in Japan 0201-300TO

P-SLD_OPTION-0-0404-13