



umano medical

QUICK REFERENCE CARD

airnest orange HYBRID™

1 Mattress Set-up Instructions

- Place mattress directly on the bed frame.
- If the surface is being used in powered mode, hang the control unit on the footboard and attach the mattress connector to the controller.
- Ensure the CPR valve is closed.
- Connect the power cord to an electrical outlet or to the bed auxiliary outlet.
- The amber light will illuminate.

2 CPR Instructions

- Turn off the control unit and disconnect the CPR deflate connector on patient left foot end of the mattress.



This medical support surface is intended to be used by trained and qualified medical staff. Restrictions apply regarding its use in specific environments. Please refer to the user manual for detailed instructions, safety measures, and warnings.



For immediate assistance, please contact your local account executive.

3 Controller Instructions




Press **POWER KEY**  to green light position.



Press **SELECT KEY**  until **MAX INFLATE**  LED lights up. The compressor will come on and quickly inflate the mattress. During the inflation, the control unit will beep every 3 minutes as a reminder that MAX INFLATE mode is activated.



Using the **SOFT / FIRM** keys  set patient comfort pressure settings by selecting the number corresponding closest to the patient's weight.






To activate **ALTERNATING PRESSURE** mode, press the **MODE KEY**  until the **ALTERNATING PRESSURE**  LED lights up.



To adjust the alternating pressure **CYCLE TIME**, press the **TIME KEY**  to the desired cycle time.

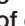

To activate **STATIC** mode, press the **MODE KEY**  until **STATIC**  LED lights up. All air cells in the mattress will be maintained at the same pressure and can be adjusted to the desired level of comfort by using the  keys.





To activate the **LOW AIR LOSS**, press the **SELECT KEY**  until **LOW AIR LOSS**  LED lights up.





To set **ALARM SILENCE**, press the **ALARM SILENCE KEY**  until the indicator light goes on.



To set **LOCK OUT**, press and hold the **LOCK KEY**  until the LED lights up. To unlock, press and hold the **LOCK KEY**  until the LED light goes off.



To activate the **FOWLER** mode, press the **MODE KEY**  until **FOWLER**  LED lights up. When the patient is in a sitting position and this mode is activated, it will inflate the mattress to higher pressures to prevent the patient from “bottoming out”.

Manufactured for:

Umano Medical Inc. 230, boul. Nilus-Leclerc, L'Islet QC G0R 2C0 CANADA
Toll Free: 1 (844) 409-4030 information@umanomedical.com



This medical support surface is intended to be used by trained and qualified medical staff. Restrictions apply regarding its use in specific environments. Please refer to the user manual for detailed instructions, safety measures and warnings.