



umano medical

QUICK REFERENCE CARD

ook«snow»^{MH} BED


BED OVERVIEW

A	Main nurse controls
<ul style="list-style-type: none"> • Siderail motion control lockouts • Bed movements • Scale & bed exit • Battery indicator 	
B	Bed exit side view lights (on foot bumpers)
<ul style="list-style-type: none"> • Green = Bed exit system is armed • Flashing amber = In alarm • Not illuminated = System not in use 	
C	Siderail controls
<ul style="list-style-type: none"> • Outer siderail = nurse controls • Inner = patient controls <p>* Siderail controls can be locked out on the footboard</p>	




REMINDERS


Ensure bed is plugged in to enable scale, bed exit & advanced features.



If bed is operating on battery mode, do not allow the battery to fully discharge.



Apply the brakes.




SAFE-LOCK™ BRAKING SYSTEM



D	Safe-Lock™ Braking System
<ul style="list-style-type: none"> • Turn to red = Brake ON • Turn to green = Steer ON • Turn to N = Neutral position (Bed is fully mobile) 	


E	Siderail release
<ul style="list-style-type: none"> • Lift green handle UP to unlock • Lower siderail down <p><i>Green window indicates safe status</i> <i>Red window indicates unlocked siderail</i></p>	

F	CPR release handle
<ul style="list-style-type: none"> • Pull and hold red handle to completely lower backrest section <p><i>Seat section will automatically retract flat</i> <i>Press any button to cancel</i></p>	

 This medical bed is intended to be used by trained and qualified medical staff. Restrictions apply regarding its use in specific environments. Please refer to the user manual for detailed instructions, safety measures and warnings.

BED EXIT, SCALE AND MASTER LOCKOUT

HOME button on the footboard returns the display to HOME SCREEN / MAIN MENU

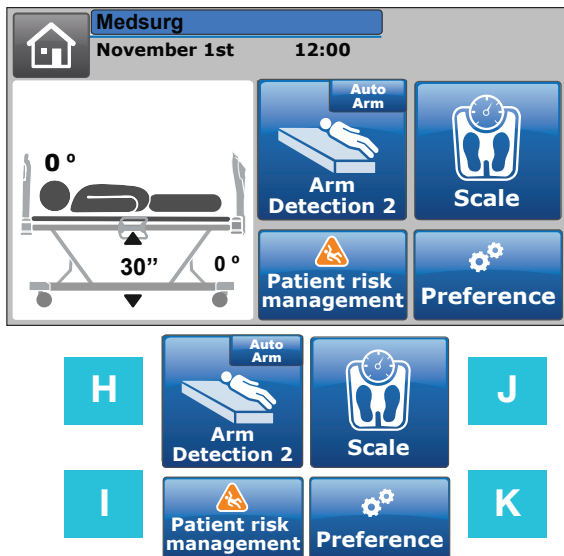

H To arm/disarm BED EXIT system (regular/manual setting)

- ① To arm BED EXIT
Patient must be in bed to arm bed exit system.
From HOME SCREEN
Press 'ARM'
Press 'YES' (To arm bed exit)
Verify that foot end bumper indicators (optional) are illuminated **green**. When they are, bed exit system is armed.
- ② To disarm BED EXIT (if alarm is triggered)
Press 'DISARM' (on HOME SCREEN). This will turn off the system. Patient must be placed back in bed and system must be manually rearmed.

H To arm/disarm BED EXIT system (AUTO Arm)



When Auto Arm is activated and patient is detected in the bed, the bed exit is armed automatically.

To disarm BED EXIT (if alarm is triggered) :
Press "DISARM" (which will appear on the HOME SCREEN). This will turn off the alarm.

If patient is out of bed :
The system will return to stand-by mode.

If patient is still in bed :
The system will apply a 1-minute care PAUSE to allow the caregiver to attend to the patient. If after 1 minute the patient is still in bed, the system will automatically initiate arming procedure.

I To turn ON/OFF AUTO Arm

From HOME SCREEN,
Press Patient risk management
Press 'Auto Arm' to activate or deactivate it.

I To change ALARM settings (volume and tone)


Alarm must be off to change settings.

From the HOME SCREEN
Press 'Patient Risk Mgmt'
Press to enter Alarm Settings menu.
Make desired changes and press 'OK' to save changes.

For immediate assistance, please contact your local sales representative.

I To change BED EXIT detection level

Bed alarm must be off to make changes.
To change or set DETECTION LEVEL
From the HOME SCREEN,
Press Patient risk management
Select one of the three detection level options
Press 'YES' (to change detection).

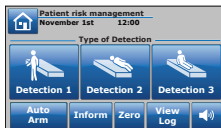

J To ZERO the SCALE

From HOME SCREEN
Press 'SCALE' button
Press 'ZERO' button
Press 'YES' or 'START' to confirm
IMPORTANT: Patient must not be on bed during procedure.



K MOTION CONTROL MASTER LOCKOUT

From HOME SCREEN
Press 'PREFERENCE' button to enter the preference menu
Press to access lock menu
Press each lock to lock desired positions.
When lock turns **orange** this confirms that position is locked.



K Preference menu

Lights configuration, Master Lockouts, ID, Auto-Contour, Minimum height adjustment, Date/time, Languages and Maintenance