




UmanoMedical

QUICK REFERENCE CARD

ook«SNOW» BED

BED OVERVIEW

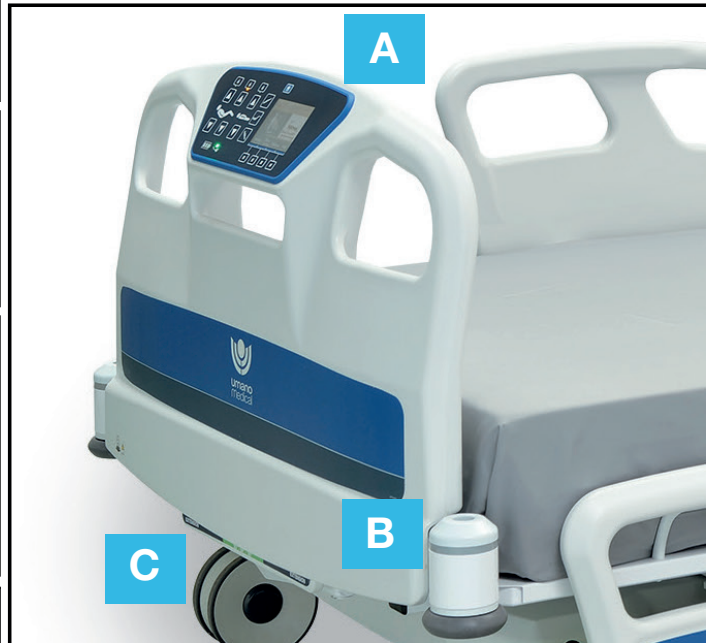

A Main Nurse control

- Lock out
- Bed movements
- Scale & bed exit
- Battery indicator

B Bed exit Sideview Lights (on foot bumpers) (optional)

- Green = Armed
- Flashing red = In alarm
- Not illuminated = Alarm is OFF

C Bed extender (optional)


- Reach & pull on Green handle
- Pull to extend
- Push to retract
- Set to desired length and lock
- Install cushion

D Siderail controls



- Outside of siderail = nurse controls
- Inside of siderail = patient controls
- * Siderail controls can be locked at foot end

Reminders

- Ensure bed is plugged to enable scale, bed exit & advanced features



- Apply the brakes

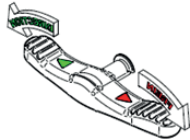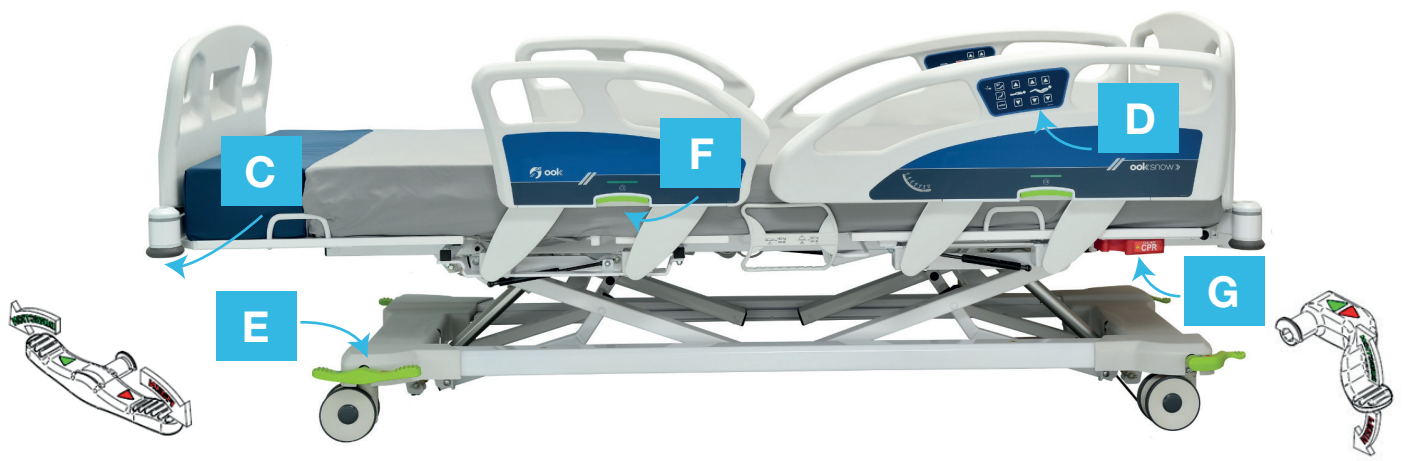E Brake pedals

- Push Red = Brake ON
- Push Green = Steer ON
- Horizontal = Neutral position (Bed is fully mobile)

F Siderail release

- Lift green handle UP to unlock
- Lower siderail down
- * Green window indicates safe status
- * Red window indicates unlocked siderail

G CPR release handle

- Pull red handle
- Lower backrest section down


Please refer to the user manuals for detailed instructions, safety measures and warnings.



SCALE & BED EXIT

HOME Button on the footboard returns the program to HOME SCREEN /MAIN MENU

NAVIGATION Buttons on footboard will activate commands on MENU

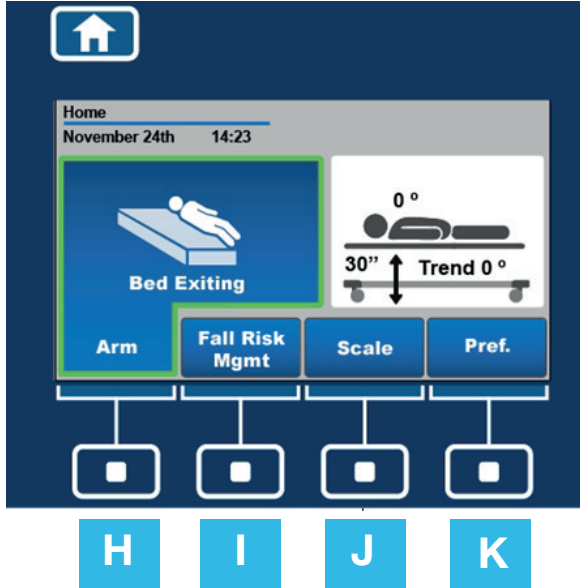

H TO ARM BED EXIT SYSTEM

- 1 Verify that bed Detection Level (Sensitivity zone) and Alarm Settings (Volume & Tone) are appropriate for the patient. (To change, enter: Fall Risk Mgmt menu)
- 2 Patient must be in bed to arm bed exit system.
- 3 From HOME SCREEN, press 'ARM' **H**
- 4 Press 'YES' ('to arm bed exit') or Press 'AUTO', if applicable.
- 5 Verify that foot end bumper indicators (optional) are illuminated **green**. When foot bumpers are illuminated **green**, bed exit system is 'ARMED'.

H IF ALARM IS TRIGGERED

Press 'DISARM' (which will appear on the HOME SCREEN) This will turn 'OFF' the alarm)

TO RESET BED ALARM

- 1 Using Regular/Manual setting, patient must be placed back in bed and alarm must be manually ARMED.
- 2 Using AUTO setting, patient must be placed back in bed and bed exit system will automatically reengage and bumper indicators will illuminate green verifying that the bed alarm has reset and is engaged.

I TO CHANGE ALARM SETTINGS

Alarm must be OFF to change settings (Alarm sensitivity level, Alarm volume, Alarm tone)
 From the HOME SCREEN
 Press 'Fall Risk Mgmt'
 Press ' Arrow Right'
 Press 'Select' area of adjustment, make desired changes, press 'OK' to save changes.


I TO CHANGE BED EXIT SENSITIVITY

Please note bed alarm must be 'OFF' to make changes

To change or set DETECTION LEVEL (sensitivity zone)

From the HOME SCREEN, Press **Fall Risk Mgmt**
 Press **Select**
 (set detection)

Press to scroll through detection level options

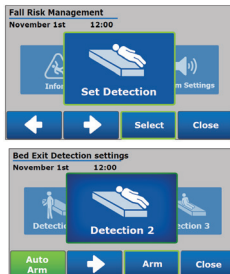



J To 'ZERO' out the SCALE

From the HOME SCREEN
 Press 'SCALE' button (which then becomes 'Select') **J**

Press (Press once to scroll to 'Zero')

Press **Select** to 'ZERO'

Press 'YES' or START to confirm
 Important: Patient should not be on bed

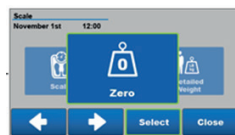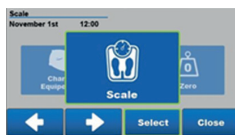

J SCALE - To weigh patient

From the HOME SCREEN
 Press 'SCALE' button (which then becomes 'Select') **J**

Press **Select** and weight will be displayed



K Maintenance menu AND Other bed settings
 CLOSE Button acts as a 'Back' Button and will bring you a step back in the program

For immediate assistance, please contact your local account executive.