244

Barrier-Free™

Bariatric Power Treatment Table

[English](#)

[Español](#)

[Français](#)

Ritter®

User's Guide

Style P TP202 20-42-FO-0014 Rev A1 C2188 003-1456-99 Rev. AA5 5/4/22

Product Identification

(The information below is required when calling for service)

Model Number:

Serial Number:

Date of Purchase:

Name of Owner / Facility:

Name of Dealer:

Dealer's Phone Number:

Midmark Authorized Service Company:

**Serial / Model
Number Location
(on frame)**

MA678900p

Table of Contents

Important Information

Safety Symbols	4
Intended Use	5
Patient Positioning	5
Electromagnetic Interference	5
Transportation / Storage Conditions	5
Authorized Representative	6
Safety Instructions:	
<i>Hand Control Connection</i>	7

Operation

Power Switch	8
Table Up / Down	9
Back Up / Down	10
Tilt Up / Down	11
Paper Roll	12

Maintenance

Cleaning	13
Preventative Maintenance	13
Calling For Service	13

Specifications

Weights, Dimensions, Electrical Specifications	14
Regulatory Compliance Chart	15
Upholstered Top Dimensions	15
Range of Motion	16

Warranty Information

Limited Warranty	17
------------------------	----

Important Information

Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which could result in injury if not avoided.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note

Amplifies a procedure, practice, or condition.

Consult User Guide for important information.

Proper shipping orientation

Fragile

Type B, Applied Part

Protective earth ground

Keep dry

Maximum stacking height (palletted units)

Fuse rating specification

Dangerous voltage / shock hazard

Duty Cycle (motor run time):
1 minute ON, 9 minutes OFF

[ATTENTION: Refer to *Regulatory Compliance Chart* to verify UL compliance for specific models]

This product is Listed by Underwriters Laboratories, Inc. Representative samples of this product have been evaluated by UL and meet the applicable U.S. safety requirements.

This product has been evaluated with respect to electric shock, fire, & mechanical hazards only, in accordance with UL60601-1, IEC60601-1, CAN/CSA C22.2, No. 601.1.

This product has been evaluated with respect to electrical shock, fire, & mechanical hazards only, in accordance with CAN/CSA C22.2, No. 601.1.

This component (or material) is Recognized by Underwriters Laboratories, Inc. Representative samples of this component have been evaluated by UL and meet the applicable U.S. safety requirements.

Intended Use

This product is intended to be used as a chair / table to provide positioning and support of patients during bariatric procedures conducted by qualified medical professionals.

Patient Positioning

The table is designed to accommodate the following examination positions:

- Prone
- Supine
- Lateral Sims

Caution

Never seat a patient on the back section of the table. Doing so may cause table to tip.

Electrical Requirements

Caution

Most exposed metal parts of the table are electrically grounded. When performing a cauterization or similar treatment, the patient must be insulated from the metal portions of the table by nonconductive material. Failure to do so could result in electric shock or burns to patient.

Electromagnetic Interference

This table is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this table, remove interfering device from room and/or plug table into an isolated circuit.

Transportation / Storage Conditions

Ambient Temperature Range: -5°C to +38°C (+23°F to 100°F)

Relative Humidity 10% to 90% (non-condensing)

Atmospheric Pressure 500hPa to 1060hPa (0.49atm to 1.05atm)

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

Authorized Representatives

WARNING

In the event of any serious incident in relation to the device, please contact Midmark and the appropriate competent authority.

Customers in Colombia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Técnica Electromédica S.A.
Calle 47 # 5-26
Bogota Colombia c.p. 110111
Tel. (+ 57 1) 756 87 87
atencionalcliente@tecnicaelectromedica.com

Customers in Costa Rica should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Corporacion Almotec SA
225 metros al sur de oficina de correos
San Jose, Zapote, Costa Rica
Tel: 506 7112 9782

Customers in the Panama should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

TECNIDENTAL PANAMA S.A.
VILLA MAGNA, VILLA ZAITA
OFFICE B8
PANAMA
Tel/Fax: 343-4049/ 6614-8029
tecnidentalpanama@gmail.com

Customers in Mexico should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Sofía Zavala | Asuntos Regulatorios
Carlos Nafarrate SA de CV
Avenida Las Américas 601,
Ladrón de Guevara, 44600,
Guadalajara, Jalisco, México | Tel (33) 3669 9800

Safety Instructions

Hand Control Connection

The hand control connection port is located on the table base at the head end of the table.


Equipment Alert

Never plug the hand control into any other outlet. Doing so may result in damage to the equipment.

Caution

Be sure that all personnel and equipment are clear of the table before activating any function. Failure to do so could result in personal injury.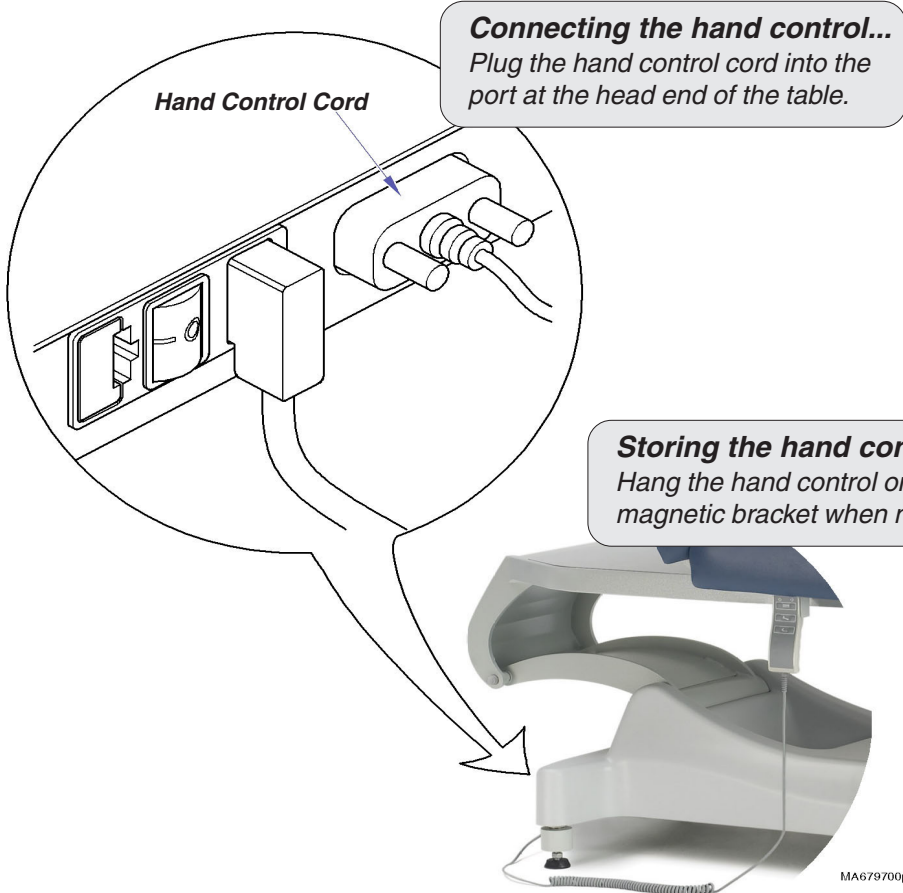

Operation

Power Switch


The power switch must be ON for the table functions to operate.

To turn the table ON...

Move the power switch to ON (I).

To turn the table OFF...

Move the power switch to OFF (O).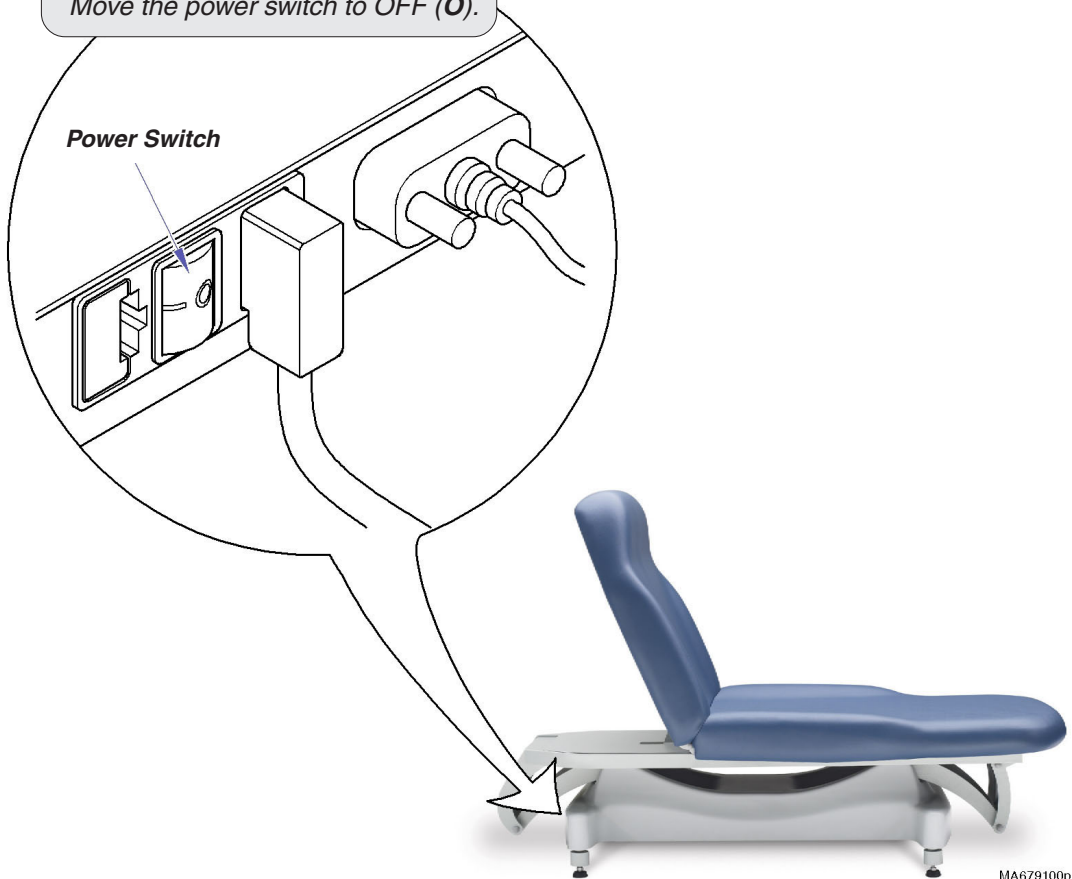

MA679100p

Table Up / Down

To move the table top UP or DOWN...
Press and hold the appropriate button.


Note: Movement automatically stops at the maximum / minimum height.

[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].

Max.
34 in.
(86.4 cm)

Min.
18 in.
(45.7 cm)
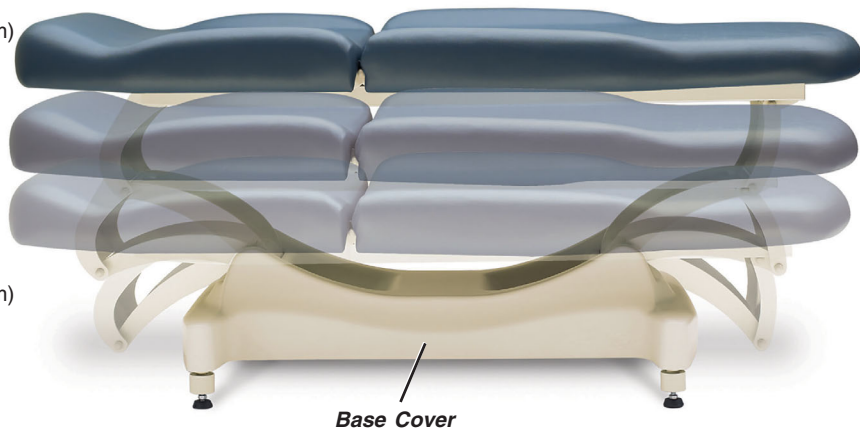


Back Up / Down

To move the Back section UP or DOWN...
Press and hold the appropriate button.

Note: Movement automatically stops at the maximum / minimum angle.

[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].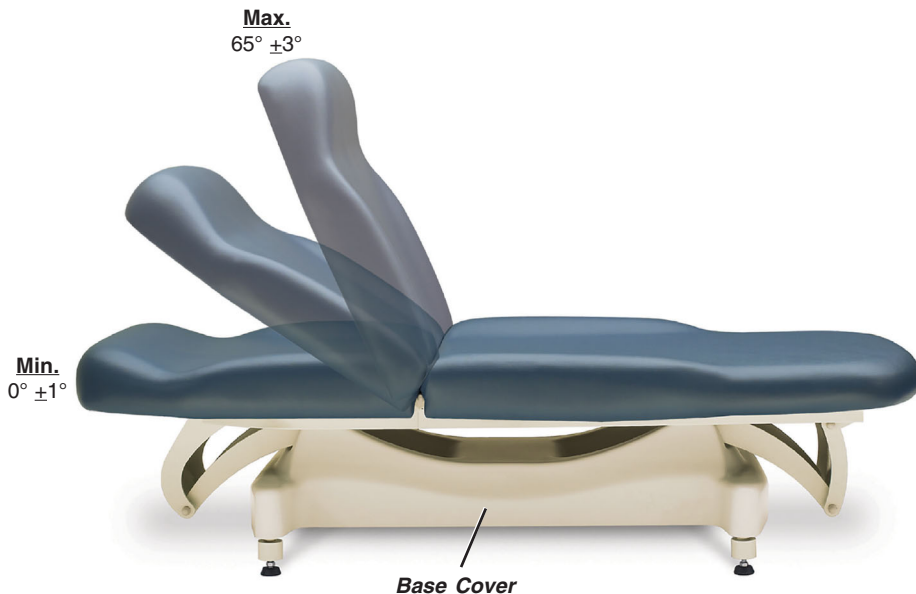
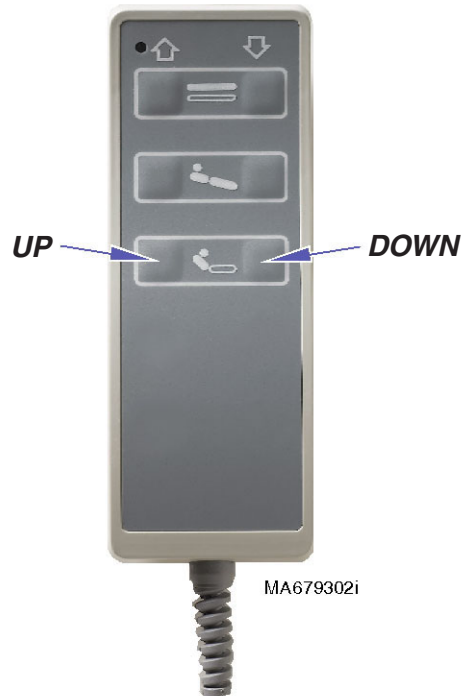


Tilt Up / Down

To Tilt the table top UP or DOWN...
Press and hold the appropriate button.

Note: Movement automatically stops at the maximum / minimum angle.

[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].

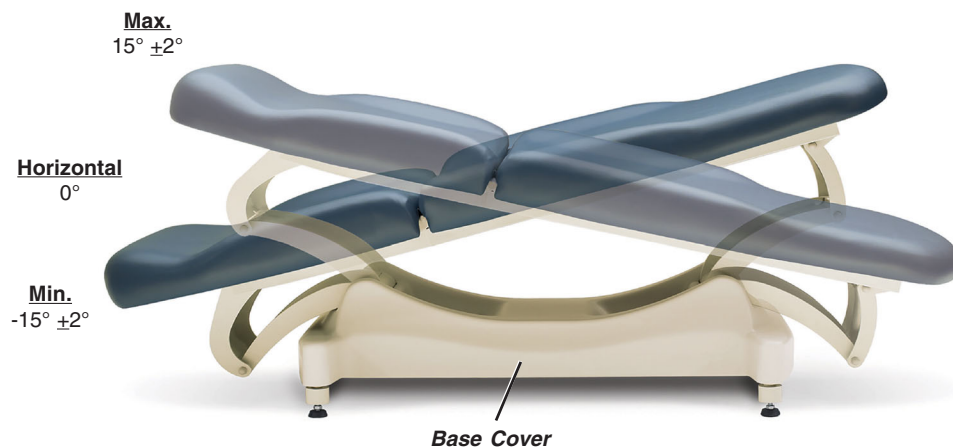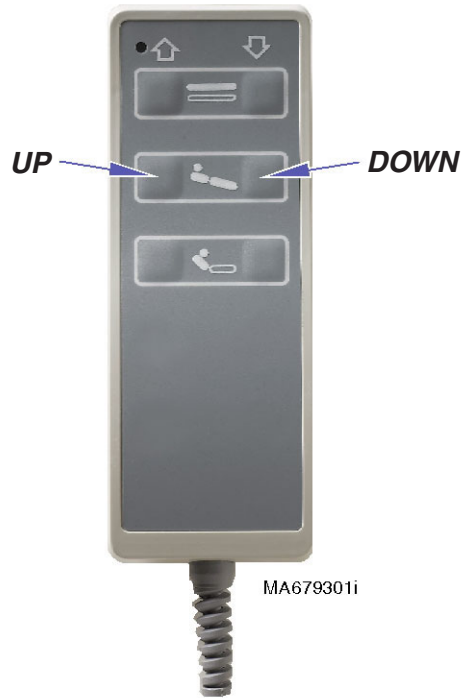
Paper Roll

Paper Roll Size (max.) 24 in. wide x 6 in. diameter
(61 cm x 15.2 cm)

WARNING

To avoid injury, do not put head or extremities under the paper tear strap.

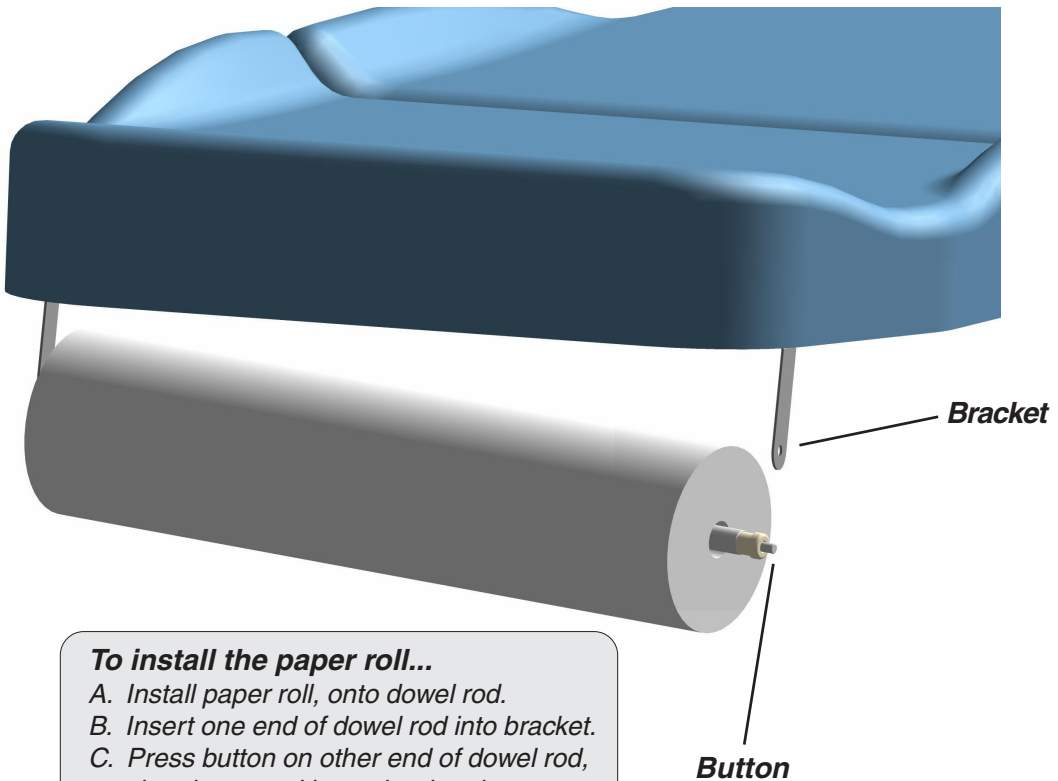To install the paper roll...

- A. Install paper roll, onto dowel rod.*
- B. Insert one end of dowel rod into bracket.*
- C. Press button on other end of dowel rod, then insert rod into other bracket.*

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Preventive Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your equipment every twelve months.

Calling For Service

Note

Model / Serial Number information is required when calling for service.

Contact your Midmark Dealer, or log onto www.midmark.com to locate your nearest service provider. To contact Midmark directly:

1.800.Midmark (1.800.643.6275) or 937.526.3662
8:00 am until 5:00 pm. Monday through Friday (EST)
[excluding standard US holidays]

Specifications

Weights, Dimensions, Electrical Specifications

Maximum Patient Weight: 850 lbs (385 kg)

w/Caster Base Accessory 650 lbs (295 kg)

Paper Roll (maximum size): (See *Paper Roll* page)

Range of Motion &

Dimensions (See *Range of Motion & Dimensions* page)

Weight of Table: 340 lbs (154 kg)

Power Cord: 8 ft. (243.8 cm) long

Fuses:

115 VAC models:

located in IEC inlet (2) 6.3A, 250V, Slo-Blo, 5 x 20 mm

Duty Cycle (motor run time): Intermittent Operation

[1 minute ON / 9 minutes OFF]

Classifications: Class 1, Type B Applied Part,
Ordinary Equipment, Intermittent Operation

Electrical Requirements: [See *Regulatory Compliance Charts*]

WARNING

Equipment is not suitable for use in the presence of a flammable anesthetic Mixture.


Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Regulatory Compliance Chart

Fire Code Ratings: All upholstery complies with California Bureau of Home Furnishing [Technical Bulletin 117](#).
Optional upholstery is available that complies with California Bureau of Home Furnishing [Technical Bulletin 133](#).

Model	Description	Serial Number Prefix	Complies To:					Electrical Ratings:		
			UL 60601-1	CAN/CSA 22.2, #601.1-M90	EN 60601-1-2 (EMC)	EN 60601-1	CE	VAC +/-10%	Amps	Cycles (Hz)
244-001	Bariatric Treatment Table	n/a	X	X				120	2.5	60


Upholstered Top Dimensions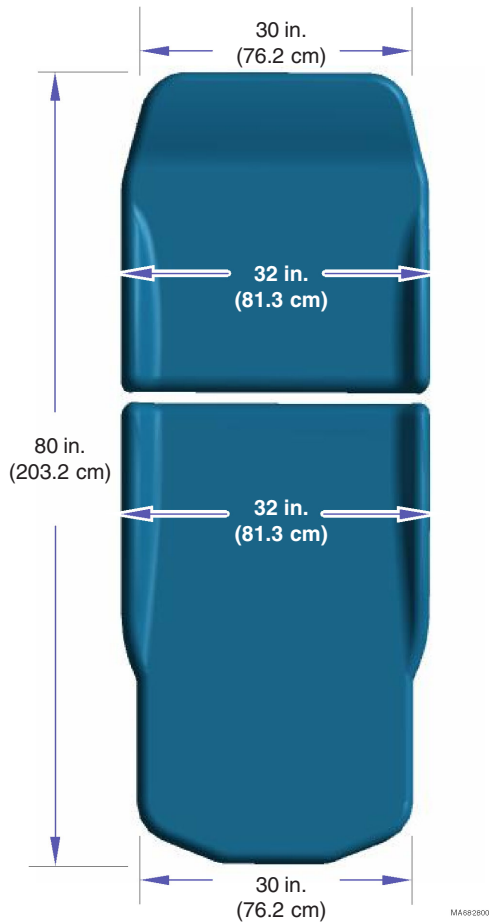

Range of Motion

Max.
34 in.
(86.4 cm)

Min.
18 in.
(45.7 cm)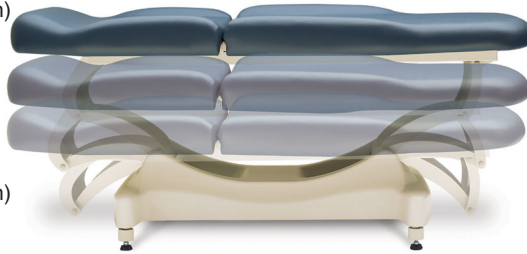

Max.
 $65^{\circ} \pm 3^{\circ}$


Min.
 $0^{\circ} \pm 1^{\circ}$

Max.
 $15^{\circ} \pm 2^{\circ}$

Horizontal
 0°

Min.
 $-15^{\circ} \pm 2^{\circ}$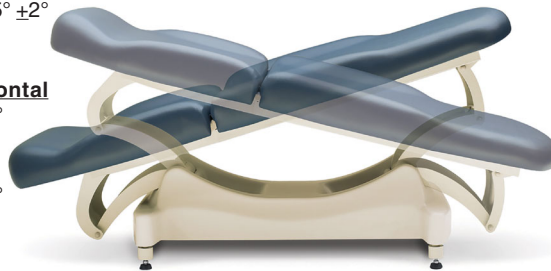

Midmark® + Ritter® Medical Equipment (Tables, Chairs, Physician Seating, Lights, Workstations and Sterilizers) Limited Warranty

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components. QuickClean® Ultrasonic Cleaners are warranted for a period of three (3) years.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at midmark.com; by phone at 1.800.MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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