253

Ritter®

LED Examination Light

[English](#)

[Español](#)

[Français](#)

Ritter 253

MA8209p

User Guide

Product Information



Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

**Model / Serial
Number Location**

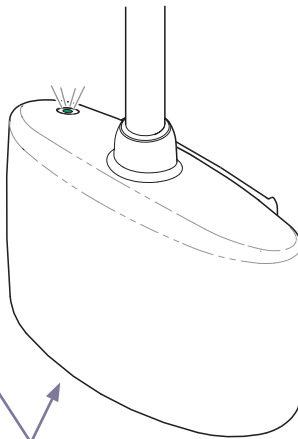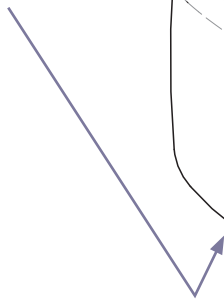MA8173i

Table of Contents

Important Information

<u>Safety Symbols</u>	4
<u>Transportation & Storage Conditions</u>	4
<u>Before First Use</u>	5
<u>Intended Use</u>	5
<u>Electromagnetic Interference</u>	5
<u>Disposal of Equipment</u>	5

Operation

<u>Power ON / OFF</u>	6
<u>Automatic Shutoff Feature</u>	7
<u>Light Pattern Adjustment</u>	8

Maintenance

<u>Fuse Replacement</u>	9
<u>Troubleshooting</u>	10
<u>Calling for Service</u>	10
<u>Cleaning</u>	10
<u>Preventative Maintenance</u>	10

Specifications

<u>Specifications Chart</u>	11
<u>Model Identification / Compliance Chart</u>	12

Warranty Information

<u>Limited Warranty</u>	13
-------------------------------	----

Important Information - Safety Symbols

These symbols may appear on your equipment and/or in the manuals.

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage.

Proper Shipping Orientation

Consult User Guide

Humidity Limit

Fragile

Fuse Rating Specification

Pressure Limit

Keep Dry

Power ON / OFF

Temperature Limit

Type B Applied Part

Bright Light
(Do not look directly into light)

Catalogue Number

Manufacturer


Serial Number

Transportation / Storage Conditions

Ambient Temperature Range..... -22°F to 140°F (-30°C to +60°C)
 Relative Humidity..... 10% to 90% (non-condensing)
 Atmospheric Pressure 500 hPa to 1060 hPa

Before First Use...

Remove protective film from lens...
Pull tab to peel protective film from lens.

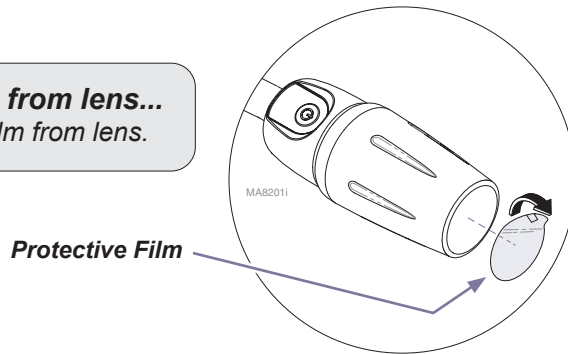Intended Use

The Ritter 253 provides focusable illumination for general examinations and procedures performed by trained professionals in a healthcare environment.

Electromagnetic Interference

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room
- Plug light into isolated circuit
- Increase separation between light and interfering device
- Contact Midmark if interference persists

Disposal of Equipment

At the end of product life, the light, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

WARNING

This product can expose you to chemicals including Antimony Oxide (Antimony Trioxide), which is known to the state of California to cause cancer. For more information go to www.p65warnings.ca.gov

Operation

Power ON / OFF

To connect power to the light...


- A) Connect power cord to light housing.
- B) Plug cord into properly rated receptacle.

Note: When power is connected, the lighthouse will flash several times, and the indicator light will illuminate.


To turn the light ON / OFF...

Press the power button.

Indicator Light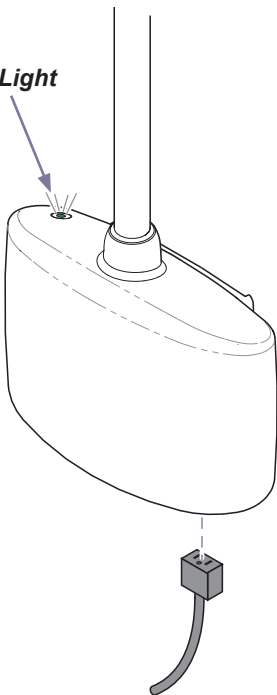

Power Button

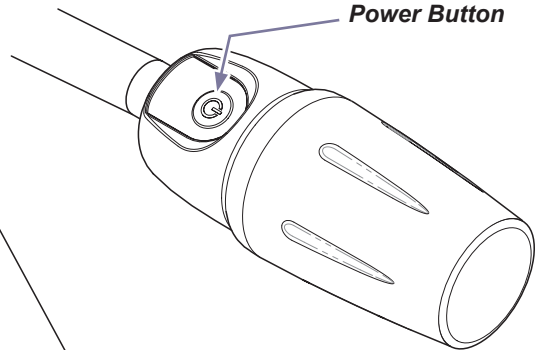MA8171i

Note

The light has an automatic shutoff feature that turns the light OFF after two hours. Refer to **Automatic Shutoff Feature** section for more information.

Automatic Shutoff Feature

The Automatic Shutoff Feature turns the light OFF two hours after the last power button press.


Note

Lights are shipped from the factory with this feature **disabled**.

To enable the Automatic Shutoff Feature...

- A) Press & hold the power button for 5 seconds.
- B) Observe. The lighthouse will flash **three** times indicating the feature has been enabled.


Note: To disable this feature, repeat the procedure.
The lighthouse will flash **four** times when this feature is disabled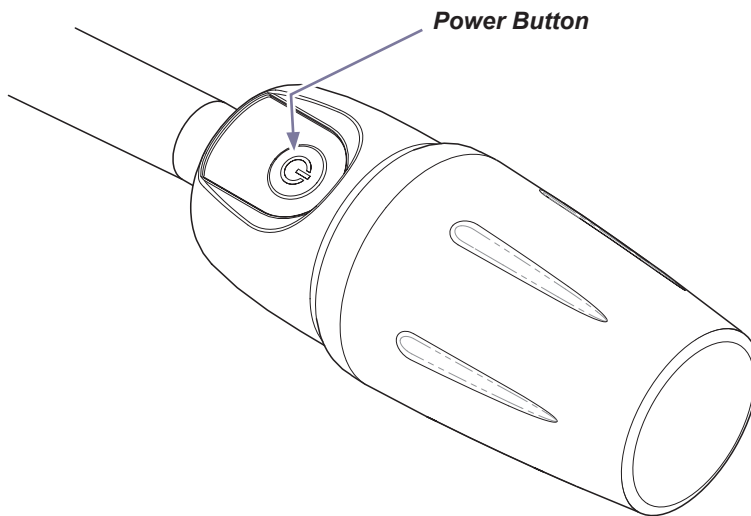

MA8170i

Light Pattern Adjustment

To adjust the size of the light pattern...
Rotate the lighthouse clockwise / counterclockwise.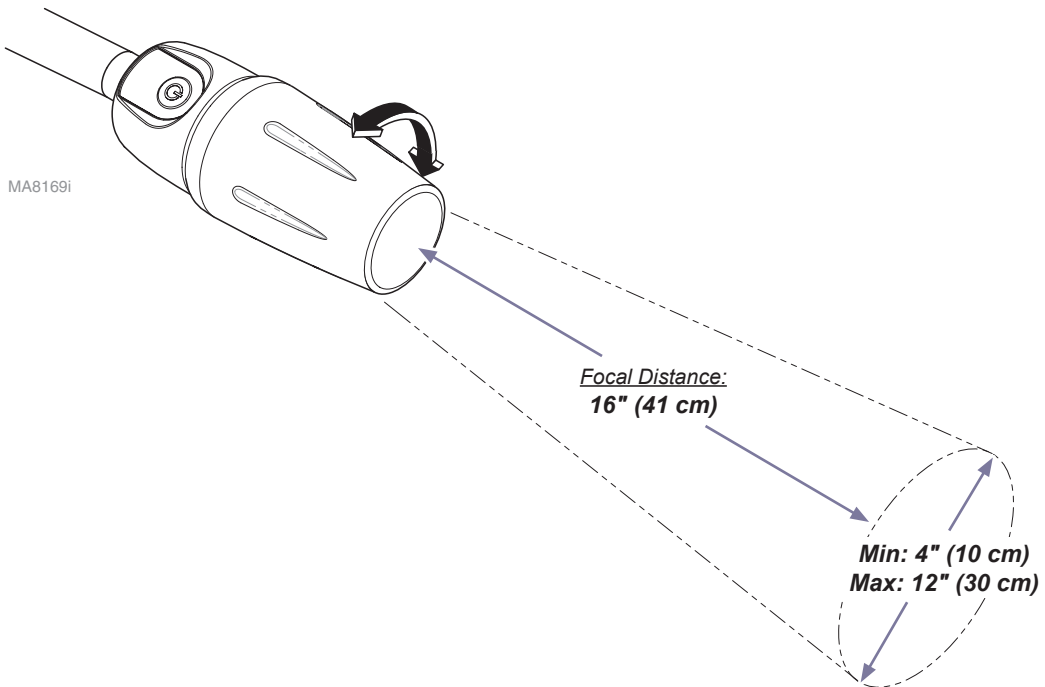


Maintenance

Fuse Replacement

To replace fuses...

- A) Disconnect power cord from light housing.
- B) Press tab & pull fuse holder out of cord inlet.
- C) Replace fuses.
- D) Push fuse holder into inlet until it locks in place.

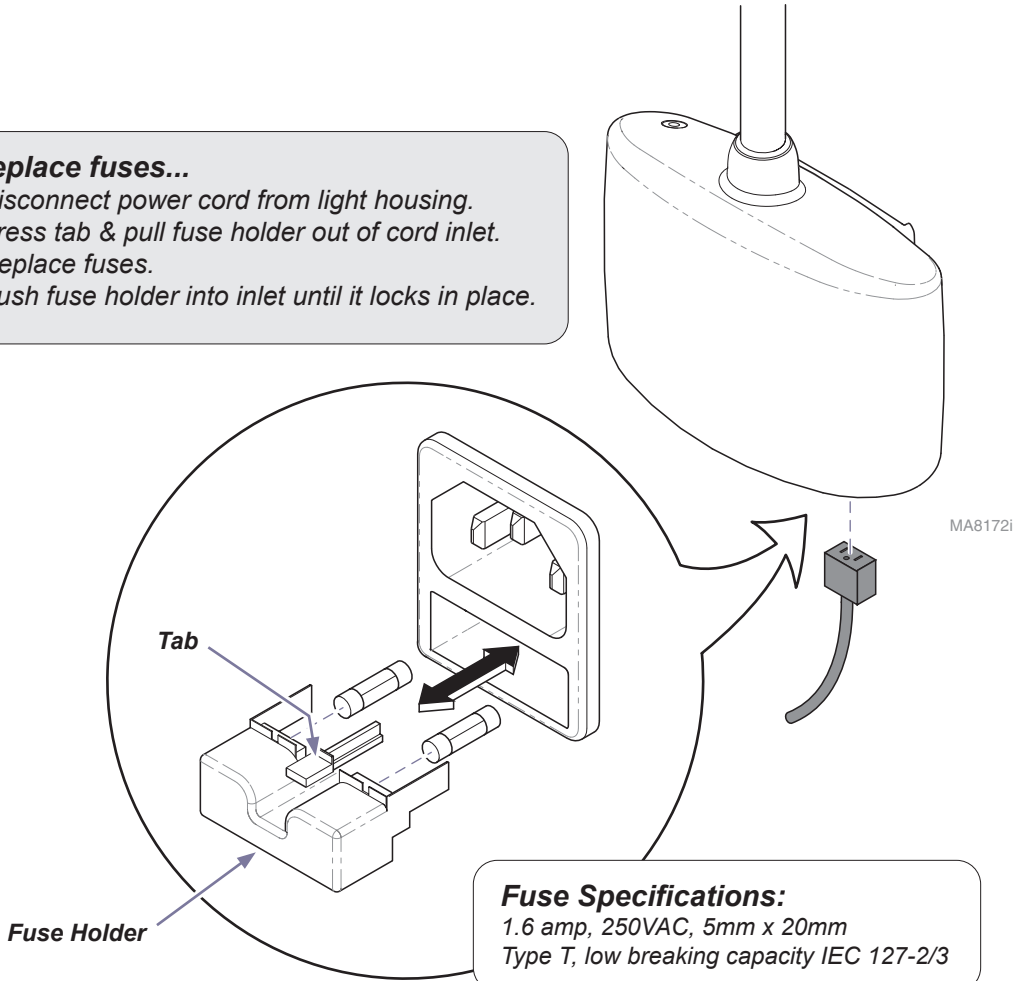Fuse Specifications:

1.6 amp, 250VAC, 5mm x 20mm
Type T, low breaking capacity IEC 127-2/3

Troubleshooting

Problem	Possible Cause	Correction
Light will not turn ON. (Power indicator light: OFF)	Power cord is not connected.	Secure power cord connections.
	Faulty fuse(s).	Replace fuses. (Refer to procedure in this manual)
Light will not turn ON. (Power indicator light: ON)	Lighthouse or flex arm malfunctioning.	Contact authorized Midmark service technician.
Light pattern will not adjust.	Lighthouse malfunctioning.	Contact authorized Midmark service technician.
Light turns OFF by itself.	Automatic Shutoff Feature	Disable automatic shutoff feature. (Refer to procedure in this manual)

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact your authorized Midmark dealer.

To contact Midmark® directly:

1.800.MIDMARK (1.800.643.6275)
8:00 am until 5:00 pm Monday - Friday (EST)
midmark.com

Cleaning

Clean the painted metal and plastic surfaces using a clean soft cloth, and mild cleaner. Disinfect your product with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection and Sterilization in Healthcare Facilities.

Preventative Maintenance

Periodically inspect the following:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.

Have an authorized service technician inspect your equipment periodically.

Specifications

Specifications Chart	
LED	Life: approximately 50,000 hr* (<i>non-replaceable</i>) * <i>per bulb manufacturer</i>
Light Pattern	Circular
Adjustable Spot Size @ <i>Focal distance: 16" (41 cm)</i>	Diameter: 4" (10 cm) to 12" (30 cm)
Illumination @ <i>Focal distance: 16" (41 cm)</i> <i>Spot size: 4" (10 cm)</i>	1,700 foot-candles / 18,298 lux
Color Temperature	4,500K
Power Cord Length	8 ft. (2.5 m)
Flex Arm Reach <i>[from power source enclosure]</i>	54.5" (138 cm)
Flex Arm Rotation <i>[at power source enclosure]</i>	140°
Electrical Rating	115 VAC, 50/60 Hz, 100mA 230 VAC, 50/60 Hz, 60mA
Fuses	1.6 amp, 250V, 5mm x 20mm, Type T low breaking capacity IEC 127-2/3
Regulatory Compliance	UL 60601-1 Rev. 26-Apr-06 CAN/CSA C22.2 No. 60601-1:08 Canadian Medical Device Regulations 5OR/98-282 NFPA99 (<i>applicable to product requirements</i>)
Classification	Class 1, Type B Applied Part

WARNING

Equipment ***is not*** suitable for use in the presence of a ***flammable anesthetic mixture*** with oxygen, air, or nitrous oxide.

Clarification: Equipment ***is*** suitable for use in the presence of oxygen, air, or nitrous oxide.

Model Identification / Compliance Chart

Model	Description	Complies To:				Electrical Ratings:		
		UL 60601-1 Rev 26-Apr-06	CAN / CSA C22.2, #60601.1:08	Canadian Medical Device Regulations 50R/98-282	NFPA 99	VAC +/- 10%	Amps	Cycles (Hz)
253-001	LED Exam Light only	•	•	•	•	115	100mA	50/60
253-002	Includes 253-001 and 9A454001 (Spline Mount)	-	-	-	-	230	60mA	50/60
253-003	Includes 253-001 and 9A455001 (Extended Spline Mount)	-	-	-	-	-	-	-
253-005	Includes 253-001 and 9A457001 (Base Rail Mount)	-	-	-	-	-	-	-
253-006	Includes 253-001 and 9A458001 (Caster Base)	-	-	-	-	-	-	-
253-007	Includes 253-001 and 9A474001 (Wall Mount)	-	-	-	-	-	-	-
253-011	LED Exam Light only	•	•	•	•	115	100mA	50/60
253-012	Includes 253-011 and 9A619001 (Standard Spline Mount)	-	-	-	-	230	60mA	50/60
253-013	Includes 253-011 and 9A620001 (Extended Spline Mount)	-	-	-	-	-	-	-
253-015	Includes 253-011 and 9A622001 (Base Rail Mount)	-	-	-	-	-	-	-
253-016	Includes 253-011 and 9A628001 (Caster Base)	-	-	-	-	-	-	-
253-017	Includes 253-011 and 9A623001 (Wall Mount)	-	-	-	-	-	-	-

Midmark® + Ritter® Medical Equipment (Tables, Chairs, Physician Seating, Lights, Workstations and Sterilizers) Limited Warranty

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components. QuickClean® Ultrasonic Cleaners are warranted for a period of three (3) years.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at midmark.com; by phone at 1.800.MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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TP202 20-42-FO-00014 Rev A1 C2169 © Midmark Corporation 2016